

DAILY SUCTION SYSTEM MAINTENANCE

PULI-JET Gentle 2.0 must never be pre or semi diluted. It must be diluted (according to the instructions) and then used straight away.




- 1** Fill the Cattani Pulse Cleaner with 2.5 litres of ambient temperature water which is halfway between the second and third line of the Pulse Cleaner.



- 2** Place PULI-JET Gentle 2.0 disinfectant on a level and steady surface and slightly unscrew the red cap (1-2 revolutions), do not completely remove the cap.



- 3** Gently squeeze the bottle to force disinfectant into the dosage chamber until the disinfectant reaches the 30ml line. If too much solution has entered the chamber it can be poured back into the main bottle by tipping towards the small tube running down the side.

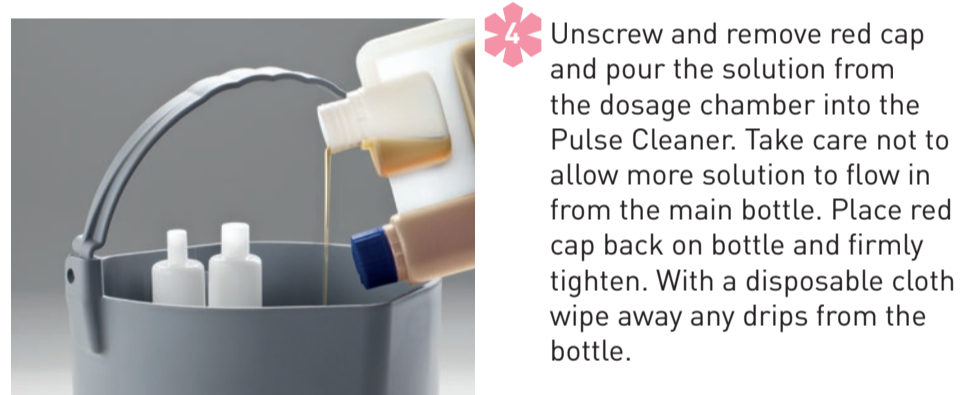

- 4** Unscrew and remove red cap and pour the solution from the dosage chamber into the Pulse Cleaner. Take care not to allow more solution to flow in from the main bottle. Place red cap back on bottle and firmly tighten. With a disposable cloth wipe away any drips from the bottle.



- 5** Connect suction hoses to Pulse Cleaner (adaptors may be required as hose sizes vary). Pulsed aspiration of the solution will happen automatically. Once pulsing has stopped please remove hoses from the Pulse Cleaner. Please note, a small amount of solution will remain in the bottom of the cleaner. This can be aspirated manually. It is highly recommended to continue aspirating air for several minutes before shutting down the suction system. This ensures that the disinfectant reaches the suction unit in the plant room.



STEP-BY-STEP SUCTION SYSTEM MAINTENANCE

Point your smart phone camera at the QR code to view our short instructional videos.

