



CATTANI  
AFTER CARE  
EQUIPMENT SERVICING  
SCHEDULE

# CONTENTS

## 04

### AFTER CARE PROGRAMME

- 5 How to promote to your customers
- 6 8 Ways to promote the After Care Programme to your customers

## 09

### COMPRESSORS ANNUAL SERVICE

- 11 AC 100 - AC 200
- 15 AC 300 - AC 400
- 21 AC 600 - AC 900
- 24 AC 1200 - 1800
- 26 AC 2400
- 27 Blok Jet 9 x 16 Cyl.

## 28

### CAD CAM COMPRESSORS ANNUAL SERVICE

- 28 AC 210 - AC 310
- 32 AC 410 - AC 610
- 38 AC 910

## 39

### SMART SERIES SUCTION SYSTEMS ANNUAL SERVICE

- 41 Micro SMART
- 43 Micro SMART Cube
- 45 Turbo SMART
- 49 Turbo SMART Cube

## 53

### TANDEM SMART SERIES SUCTION SYSTEMS ANNUAL SERVICE

- 53 Tandem Micro SMART Range
- 55 Tandem Turbo SMART Range
- 59 Multi SMART T3 Range
- 61 Multi SMART T4 Range
- 63 Multi SMART T5 Range

# 65

**MAXI SMART SERIES  
SUCTION SYSTEMS  
ANNUAL SERVICE**



65 Maxi Smart

# 67

**BLOK JET LARGE  
VACUUM SYSTEM  
ANNUAL SERVICE**



67 Blok Jet Large Central  
Vacuum System

# 69

**SUCTION SYSTEMS  
ANNUAL SERVICE**




69 Aspi Jet Range  
73 Turbo Jet Range

# 75

**DRY SYSTEMS  
ANNUAL SERVICE**



75 Dry Suction Range

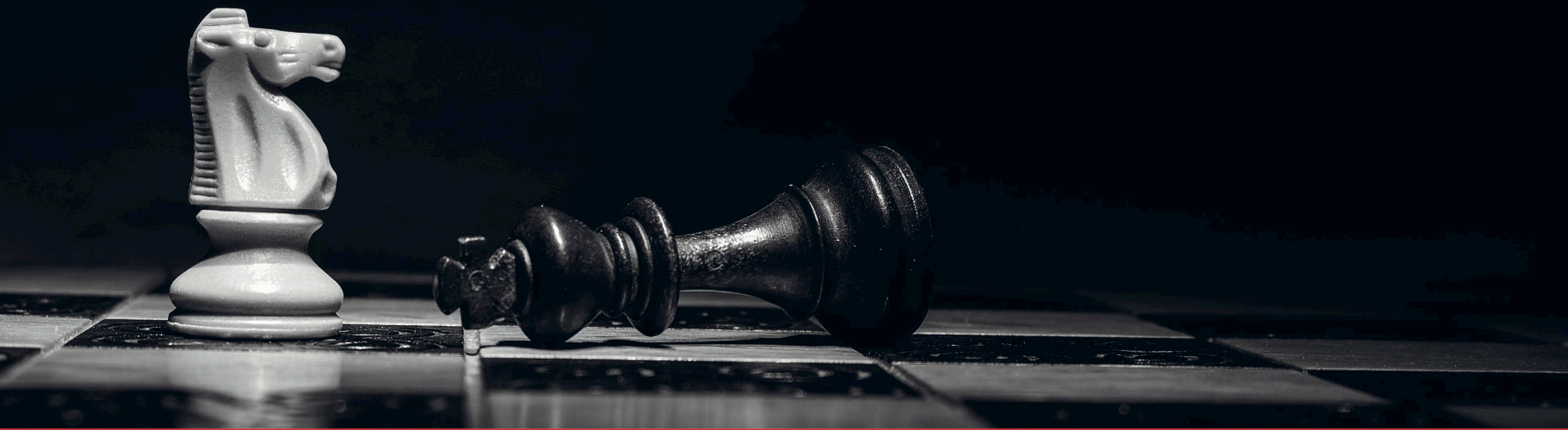

# AFTER CARE PROGRAMME

# CATTANI AFTER CARE – IT'S A SMART MOVE

## How to promote the After Care Programme to your customers

Cattani After Care is a service programme with genuine benefits for dentists. This document lists the key advantages and in simple language and suggests different ways to express those advantages.

You won't need to use all of the arguments presented here, so choose the ones that suit you, the dentist and the practice situation.

And don't hesitate to speak to your friends at Cattani if you need any more information or advice regarding the programme.

## 8 Ways to promote the After Care Programme to your customers

### 1. Your annual check-up is due

Preventive maintenance is a proven concept – even dentists do it

Dentists encourage their patients to have regular check-ups and oral hygiene visits because it helps them keep their natural teeth and delays major work for as long as possible. Preventive machinery maintenance like the Cattani After Care Programme does the same thing. It's like a regular health check for the dentist's plant room equipment.

### 2. Scheduled servicing keeps a motor running

All engines need regular maintenance – it's a mechanical fact of life

A Cattani machine is more like a car than any other piece of equipment in a dental practice, and like a car it's unrealistic to think it will run indefinitely without some TLC. No one buys a Ferrari and runs it into the ground – same with Cattani. It will perform longer, and much more reliably, if it is looked after on a regular basis with the Cattani After Care Programme.

### 3. It's basic risk management

Minimise the risk of breakdowns and their unpleasant consequences

Regular scheduled servicing of Cattani suction units and compressors minimises the risk of them breaking down. This idea can also be framed as “prevent now or pay later” or even as a kind of insurance. Think about reducing the chance of:

- Premature equipment failure
- Unscheduled down time – when is it ever be convenient?
- Expensive, unscheduled technician call-outs
- Significant machinery repair and replacement costs
- Embarrassment in front of patients – and the cost and inconvenience of rescheduling, maybe even losing a patient

### 4. Cattani After Care means no surprises

The Cattani After Care Programme is transparent in concept and cost

Cattani After Care is more than just regular scheduled servicing. Think of receiving an invoice after the annual car service, not knowing whether it's going to cost, say, \$300 or \$1200. That's another reason why Cattani After Care makes such good sense – you know what each service costs, so no bill shock. And every service in the programme has its own unique, pre-packaged kit with all the essential parts.

### 5. Cattani time is productive time

Cattani After Care keeps practice disruption to a minimum

This is a completely proactive service programme, managed by the technician. In many cases the technician can service Cattani equipment at the same time as other surgery equipment like autoclaves, chairs, etc, with visits coordinated to minimise costs and inconvenience for both parties.

### **6. Cattani After Care is in expert hands**

**No other specialist plant room equipment supplier matches Cattani's level of service**

**Cattani's intention is to provide the best level of service to end users (and service training courses for technicians), and dentists can be assured that only technicians fully trained in servicing Cattani products are invited to be part of the After Care Programme.**

### **7. Everyone wins**

**Quality, documented service every time with Cattani After Care**


**Cattani After Care is a fully laid out programme with dates and terms and conditions, all in line with factory-recommended service intervals. After Care comes with a simple log book that is kept up to date by the technician. This log book leaves no doubt about where things are at with a particular machine.**

### **8. Quality equipment, quality service, always**

**Optimal performance and professionalism is good for everyone**

**No one can afford to be casual about their professional reputations. Cattani After Care is an easy way to protect and even enhance the reputations of the dentist, the technician and Cattani itself. We all take a hit if something goes wrong, so let's do everything we can to prevent that, and in fact go the extra mile. Grow our reputations and we grow our businesses.**



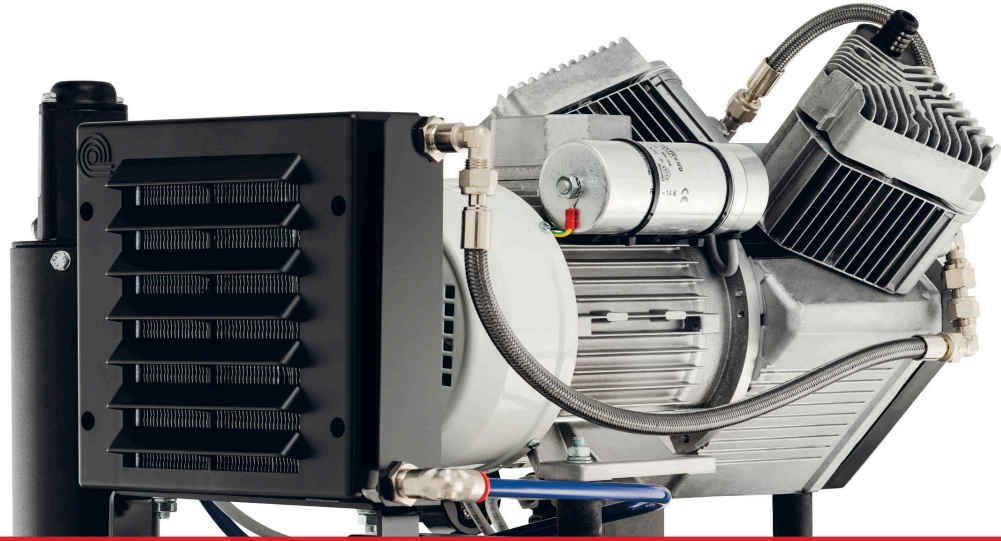

# AC SERIES COMPRESSORS

# AC SERIES COMPRESSOR ANNUAL SERVICING SCHEDULE

## Cattani Compressors are serviced annually

- 1x Service Kit required per head unit
- Kits are available for head units with 1,2, 3 & 6 cylinder(s)
- All Service Kits available with a standard filter or HEPA filter for the drying column

### AC COMPRESSOR (~1)

#### Annual Servicing Schedule

##### Year 1 & Year 2 - Minor Service Kit

- Air inlet filters (1 filter per cylinder), drying column filter and o-ring seal
- HEPA filter option for drying column is available.

##### Year 3 - Intermediate Service Kit

- Air inlet filters (1 filter per cylinder), drying column filter, o-ring seal and a motor start capacitor
- HEPA filter option for drying column is available.

### AC COMPRESSOR (~3)

#### Annual Servicing Schedule

##### Every Year - Minor Service Kit

- Air inlet filters (1x filter per cylinder), drying column filter and o-ring seal
- HEPA filter option for drying column is available.

Refer to Page 80 to view  
Aftercare Compressor Service Report.

# ANNUAL SERVICING SCHEDULE

## AC100 Compressor (~1)

(with HEPA Filter for drying column)



\* Servicing schedule also applies to "E" version of this compressor

Year	Minor Service Kits	Intermediate Service Kits
1	(x1) K-SK1013B Inlet filters & O-ring	
2	(x1) K-SK1013B Inlet filters & O-ring	
3		(x1) K-SK101B Capacitor, Inlet filters & O-ring
4	(x1) K-SK1013B Inlet filters & O-ring	
5	(x1) K-SK1013B Inlet filters & O-ring	
6		(x1) K-SK101B Capacitor, Inlet filters & O-ring
7	(x1) K-SK1013B Inlet filters & O-ring	
8	(x1) K-SK1013B Inlet filters & O-ring	
9		(x1) K-SK101B Capacitor, Inlet filters & O-ring
10	(x1) K-SK1013B Inlet filters & O-ring	

# ANNUAL SERVICING SCHEDULE

## AC100Q Compressor (~1)

(with HEPA Filter for drying column)



Year	Minor Service Kits	Intermediate Service Kits
1	(x1) K-SK1013B Inlet filters & O-ring	
2	(x1) K-SK1013B Inlet filters & O-ring	
3		(x1) K-SK101B Capacitor, Inlet filters & O-ring
4	(x1) K-SK1013B Inlet filters & O-ring	
5	(x1) K-SK1013B Inlet filters & O-ring	
6		(x1) K-SK101B Capacitor, Inlet filters & O-ring
7	(x1) K-SK1013B Inlet filters & O-ring	
8	(x1) K-SK1013B Inlet filters & O-ring	
9		(x1) K-SK101B Capacitor, Inlet filters & O-ring
10	(x1) K-SK1013B Inlet filters & O-ring	

\* Servicing schedule also applies to "E" version of this compressor

# ANNUAL SERVICING SCHEDULE

## AC200 Compressor (~1)

(with HEPA Filter for drying column)



\* Servicing schedule also applies to "E" version of this compressor

Year	Minor Service Kits	Intermediate Service Kits
1	(x1) K-SK2013B Inlet filters & O-ring	
2	(x1) K-SK2013B Inlet filters & O-ring	
3		(x1) K-SK201B Capacitor, Inlet filters & O-ring
4	(x1) K-SK2013B Inlet filters & O-ring	
5	(x1) K-SK2013B Inlet filters & O-ring	
6		(x1) K-SK201B Capacitor, Inlet filters & O-ring
7	(x1) K-SK2013B Inlet filters & O-ring	
8	(x1) K-SK2013B Inlet filters & O-ring	
9		(x1) K-SK201B Capacitor, Inlet filters & O-ring
10	(x1) K-SK2013B Inlet filters & O-ring	

# ANNUAL SERVICING SCHEDULE

## AC200Q Compressor (~1)

(with HEPA Filter for drying column)



\* Servicing schedule also applies to "E" version of this compressor

Year	Minor Service Kits	Intermediate Service Kits
1	(x1) K-SK2013B Inlet filters & O-ring	
2	(x1) K-SK2013B Inlet filters & O-ring	
3		(x1) K-SK201B Capacitor, Inlet filters & O-ring
4	(x1) K-SK2013B Inlet filters & O-ring	
5	(x1) K-SK2013B Inlet filters & O-ring	
6		(x1) K-SK201B Capacitor, Inlet filters & O-ring
7	(x1) K-SK2013B Inlet filters & O-ring	
8	(x1) K-SK2013B Inlet filters & O-ring	
9		(x1) K-SK201B Capacitor, Inlet filters & O-ring
10	(x1) K-SK2013B Inlet filters & O-ring	

# ANNUAL SERVICING SCHEDULE

## AC300 Compressor (~1)

(with HEPA Filter for drying column)



Year	Minor Service Kits	Intermediate Service Kits
1	(x1) K-SK3013B Inlet filters & O-ring	
2	(x1) K-SK3013B Inlet filters & O-ring	
3		(x1) K-SK301B Capacitor, Inlet filters & O-ring
4	(x1) K-SK3013B Inlet filters & O-ring	
5	(x1) K-SK3013B Inlet filters & O-ring	
6		(x1) K-SK301B Capacitor, Inlet filters & O-ring
7	(x1) K-SK3013B Inlet filters & O-ring	
8	(x1) K-SK3013B Inlet filters & O-ring	
9		(x1) K-SK301B Capacitor, Inlet filters & O-ring
10	(x1) K-SK3013B Inlet filters & O-ring	

\* Servicing schedule also applies to "E" version of this compressor

# ANNUAL SERVICING SCHEDULE

## AC300Q Compressor (~1)

(with HEPA Filter for drying column)



Year	Minor Service Kits	Intermediate Service Kits
1	(x1) K-SK3013B Inlet filters & O-ring	
2	(x1) K-SK3013B Inlet filters & O-ring	
3		(x1) K-SK301B Capacitor, Inlet filters & O-ring
4	(x1) K-SK3013B Inlet filters & O-ring	
5	(x1) K-SK3013B Inlet filters & O-ring	
6		(x1) K-SK301B Capacitor, Inlet filters & O-ring
7	(x1) K-SK3013B Inlet filters & O-ring	
8	(x1) K-SK3013B Inlet filters & O-ring	
9		(x1) K-SK301B Capacitor, Inlet filters & O-ring
10	(x1) K-SK3013B Inlet filters & O-ring	

\* Servicing schedule also applies to "E" version of this compressor



# ANNUAL SERVICING SCHEDULE

## AC4001P Compressor (~1)

(with HEPA Filter for drying column)



Year	Minor Service Kits	Intermediate Service Kits
1	(x2) K-SK2013B Inlet filters & O-ring	
2	(x2) K-SK2013B Inlet filters & O-ring	
3		(x2) K-SK201B Capacitor, Inlet filters & O-ring
4	(x2) K-SK2013B Inlet filters & O-ring	
5	(x2) K-SK2013B Inlet filters & O-ring	
6		(x2) K-SK201B Capacitor, Inlet filters & O-ring
7	(x2) K-SK2013B Inlet filters & O-ring	
8	(x2) K-SK2013B Inlet filters & O-ring	
9		(x2) K-SK201B Capacitor, Inlet filters & O-ring
10	(x2) K-SK2013B Inlet filters & O-ring	

\* Servicing schedule also applies to "E" version of this compressor

# ANNUAL SERVICING SCHEDULE

## AC400Q1P Compressor (~1)

(with HEPA Filter for drying column)



\* Servicing schedule also applies to "E" version of this compressor

Year	Minor Service Kits	Intermediate Service Kits
1	(x2) K-SK2013B Inlet filters & O-ring	
2	(x2) K-SK2013B Inlet filters & O-ring	
3		(x2) K-SK201B Capacitor, Inlet filters & O-ring
4	(x2) K-SK2013B Inlet filters & O-ring	
5	(x2) K-SK2013B Inlet filters & O-ring	
6		(x2) K-SK201B Capacitor, Inlet filters & O-ring
7	(x2) K-SK2013B Inlet filters & O-ring	
8	(x2) K-SK2013B Inlet filters & O-ring	
9		(x2) K-SK201B Capacitor, Inlet filters & O-ring
10	(x2) K-SK2013B Inlet filters & O-ring	

# ANNUAL SERVICING SCHEDULE

## AC400 Compressor (~3)

(with HEPA Filter for drying column)



Year	Minor Service Kits
1	(x2) K-SK2013B Inlet filters & O-ring
2	(x2) K-SK2013B Inlet filters & O-ring
3	(x2) K-SK2013B Inlet filters & O-ring
4	(x2) K-SK2013B Inlet filters & O-ring
5	(x2) K-SK2013B Inlet filters & O-ring
6	(x2) K-SK2013B Inlet filters & O-ring
7	(x2) K-SK2013B Inlet filters & O-ring
8	(x2) K-SK2013B Inlet filters & O-ring
9	(x2) K-SK2013B Inlet filters & O-ring
10	(x2) K-SK2013B Inlet filters & O-ring

\* Servicing schedule also applies to "E" version of this compressor

# ANNUAL SERVICING SCHEDULE

## AC400Q Compressor (~3)

(with HEPA Filter for drying column)



Year	Minor Service Kits
1	(x2) K-SK2013B Inlet filters & O-ring
2	(x2) K-SK2013B Inlet filters & O-ring
3	(x2) K-SK2013B Inlet filters & O-ring
4	(x2) K-SK2013B Inlet filters & O-ring
5	(x2) K-SK2013B Inlet filters & O-ring
6	(x2) K-SK2013B Inlet filters & O-ring
7	(x2) K-SK2013B Inlet filters & O-ring
8	(x2) K-SK2013B Inlet filters & O-ring
9	(x2) K-SK2013B Inlet filters & O-ring
10	(x2) K-SK2013B Inlet filters & O-ring

\* Servicing schedule also applies to "E" version of this compressor

# ANNUAL SERVICING SCHEDULE

## AC600 Compressor (~3)

(with HEPA Filter for drying column)



Year	Minor Service Kits
1	(x2) K-SK3013B Inlet filters & O-ring
2	(x2) K-SK3013B Inlet filters & O-ring
3	(x2) K-SK3013B Inlet filters & O-ring
4	(x2) K-SK3013B Inlet filters & O-ring
5	(x2) K-SK3013B Inlet filters & O-ring
6	(x2) K-SK3013B Inlet filters & O-ring
7	(x2) K-SK3013B Inlet filters & O-ring
8	(x2) K-SK3013B Inlet filters & O-ring
9	(x2) K-SK3013B Inlet filters & O-ring
10	(x2) K-SK3013B Inlet filters & O-ring

\* Servicing schedule also applies to "E" version of this compressor

# ANNUAL SERVICING SCHEDULE

## AC600Q Compressor (~3)

(with HEPA Filter for drying column)



\* Servicing schedule also applies to "E" version of this compressor

Year	Minor Service Kits
1	(x2) K-SK3013B Inlet filters & O-ring
2	(x2) K-SK3013B Inlet filters & O-ring
3	(x2) K-SK3013B Inlet filters & O-ring
4	(x2) K-SK3013B Inlet filters & O-ring
5	(x2) K-SK3013B Inlet filters & O-ring
6	(x2) K-SK3013B Inlet filters & O-ring
7	(x2) K-SK3013B Inlet filters & O-ring
8	(x2) K-SK3013B Inlet filters & O-ring
9	(x2) K-SK3013B Inlet filters & O-ring
10	(x2) K-SK3013B Inlet filters & O-ring

# ANNUAL SERVICING SCHEDULE

## AC900 Compressor (~3)

(with HEPA Filter for drying column)



Year	Minor Service Kits
1	(x3) K-SK3013B Inlet filters & O-ring
2	(x3) K-SK3013B Inlet filters & O-ring
3	(x3) K-SK3013B Inlet filters & O-ring
4	(x3) K-SK3013B Inlet filters & O-ring
5	(x3) K-SK3013B Inlet filters & O-ring
6	(x3) K-SK3013B Inlet filters & O-ring
7	(x3) K-SK3013B Inlet filters & O-ring
8	(x3) K-SK3013B Inlet filters & O-ring
9	(x3) K-SK3013B Inlet filters & O-ring
10	(x3) K-SK3013B Inlet filters & O-ring

\* Servicing schedule also applies to "E" version of this compressor

# ANNUAL SERVICING SCHEDULE

## AC1200 Compressor (~3)

(with HEPA Filter for drying column)



Year	Minor Service Kits
1	(x2) K-SK0106-11B Inlet filters & O-ring
2	(x2) K-SK0106-11B Inlet filters & O-ring
3	(x2) K-SK0106-11B Inlet filters & O-ring
4	(x2) K-SK0106-11B Inlet filters & O-ring
5	(x2) K-SK0106-11B Inlet filters & O-ring
6	(x2) K-SK0106-11B Inlet filters & O-ring
7	(x2) K-SK0106-11B Inlet filters & O-ring
8	(x2) K-SK0106-11B Inlet filters & O-ring
9	(x2) K-SK0106-11B Inlet filters & O-ring
10	(x2) K-SK0106-11B Inlet filters & O-ring

\* Servicing schedule also applies to "E" version of this compressor



# ANNUAL SERVICING SCHEDULE

## AC1800 Compressor (~3)

(with HEPA Filter for drying column)



Year	Minor Service Kits
1	(x3) K-SK0106-11B Inlet filters & O-ring
2	(x3) K-SK0106-11B Inlet filters & O-ring
3	(x3) K-SK0106-11B Inlet filters & O-ring
4	(x3) K-SK0106-11B Inlet filters & O-ring
5	(x3) K-SK0106-11B Inlet filters & O-ring
6	(x3) K-SK0106-11B Inlet filters & O-ring
7	(x3) K-SK0106-11B Inlet filters & O-ring
8	(x3) K-SK0106-11B Inlet filters & O-ring
9	(x3) K-SK0106-11B Inlet filters & O-ring
10	(x3) K-SK0106-11B Inlet filters & O-ring

\* Servicing schedule also applies to "E" version of this compressor

# ANNUAL SERVICING SCHEDULE

## AC2400 Compressor (~3)

(with HEPA H14 Filtration)



Year	Minor Service Kits
1	(x24) K-168429 (Inlet Filters) (x1) C-ZV2020 (3 Micron Prefilter) (x1) C-ZX2020 (0.01 Micron Filter)
2	(x24) K-168429 (Inlet Filters) (x1) C-ZV2020 (3 Micron Prefilter) (x1) C-ZX2020 (0.01 Micron Filter)
3	(x24) K-168429 (Inlet Filters) (x1) C-ZV2020 (3 Micron Prefilter) (x1) C-ZX2020 (0.01 Micron Filter)
4	(x24) K-168429 (Inlet Filters) (x1) C-ZV2020 (3 Micron Prefilter) (x1) C-ZX2020 (0.01 Micron Filter)
5	(x24) K-168429 (Inlet Filters) (x1) C-ZV2020 (3 Micron Prefilter) (x1) C-ZX2020 (0.01 Micron Filter)

Year	Minor Service Kits
6	(x24) K-168429 (Inlet Filters) (x1) C-ZV2020 (3 Micron Prefilter) (x1) C-ZX2020 (0.01 Micron Filter)
7	(x24) K-168429 (Inlet Filters) (x1) C-ZV2020 (3 Micron Prefilter) (x1) C-ZX2020 (0.01 Micron Filter)
8	(x24) K-168429 (Inlet Filters) (x1) C-ZV2020 (3 Micron Prefilter) (x1) C-ZX2020 (0.01 Micron Filter)
9	(x24) K-168429 (Inlet Filters) (x1) C-ZV2020 (3 Micron Prefilter) (x1) C-ZX2020 (0.01 Micron Filter)
10	(x24) K-168429 (Inlet Filters) (x1) C-ZV2020 (3 Micron Prefilter) (x1) C-ZX2020 (0.01 Micron Filter)

# ANNUAL SERVICING SCHEDULE

## Blok-Jet (9x) 6 Cylinder

(with HEPA H14 Filtration)



Year	Minor Service Kits
1	(x54) K-168429 (Inlet Filters) (x1) F-211771 (3 Micron Pre-Filter) (x1) F-211773 (0.01 Micron Post-Filter)
2	(x54) K-168429 (Inlet Filters) (x1) F-211771 (3 Micron Pre-Filter) (x1) F-211773 (0.01 Micron Post-Filter)
3	(x54) K-168429 (Inlet Filters) (x1) F-211771 (3 Micron Pre-Filter) (x1) F-211773 (0.01 Micron Post-Filter)
4	(x54) K-168429 (Inlet Filters) (x1) F-211771 (3 Micron Pre-Filter) (x1) F-211773 (0.01 Micron Post-Filter)
5	(x54) K-168429 (Inlet Filters) (x1) F-211771 (3 Micron Pre-Filter) (x1) F-211773 (0.01 Micron Post-Filter)

Year	Minor Service Kits
6	(x54) K-168429 (Inlet Filters) (x1) F-211771 (3 Micron Pre-Filter) (x1) F-211773 (0.01 Micron Post-Filter)
7	(x54) K-168429 (Inlet Filters) (x1) F-211771 (3 Micron Pre-Filter) (x1) F-211773 (0.01 Micron Post-Filter)
8	(x54) K-168429 (Inlet Filters) (x1) F-211771 (3 Micron Pre-Filter) (x1) F-211773 (0.01 Micron Post-Filter)
9	(x54) K-168429 (Inlet Filters) (x1) F-211771 (3 Micron Pre-Filter) (x1) F-211773 (0.01 Micron Post-Filter)
10	(x54) K-168429 (Inlet Filters) (x1) F-211771 (3 Micron Pre-Filter) (x1) F-211773 (0.01 Micron Post-Filter)

# ANNUAL SERVICING SCHEDULE

## AC210 Compressor (~1)

(with HEPA Filter for drying column)



\* Servicing schedule also applies to "E" version of this compressor

Year	Minor Service Kits	Intermediate Service Kits
1	(x1) K-SK2013B Inlet filters & O-ring	
2	(x1) K-SK2013B Inlet filters & O-ring	
3		(x1) K-SK201B Capacitor, Inlet filters & O-ring
4	(x1) K-SK2013B Inlet filters & O-ring	
5	(x1) K-SK2013B Inlet filters & O-ring	
6		(x1) K-SK201B Capacitor, Inlet filters & O-ring
7	(x1) K-SK2013B Inlet filters & O-ring	
8	(x1) K-SK2013B Inlet filters & O-ring	
9		(x1) K-SK201B Capacitor, Inlet filters & O-ring
10	(x1) K-SK2013B Inlet filters & O-ring	

# ANNUAL SERVICING SCHEDULE

## AC210Q Compressor (~1)

(with HEPA Filter for drying column)



\* Servicing schedule also applies to "E" version of this compressor

Year	Minor Service Kits	Intermediate Service Kits
1	(x1) K-SK2013B Inlet filters & O-ring	
2	(x1) K-SK2013B Inlet filters & O-ring	
3		(x1) K-SK201B Capacitor, Inlet filters & O-ring
4	(x1) K-SK2013B Inlet filters & O-ring	
5	(x1) K-SK2013B Inlet filters & O-ring	
6		(x1) K-SK201B Capacitor, Inlet filters & O-ring
7	(x1) K-SK2013B Inlet filters & O-ring	
8	(x1) K-SK2013B Inlet filters & O-ring	
9		(x1) K-SK201B Capacitor, Inlet filters & O-ring
10	(x1) K-SK2013B Inlet filters & O-ring	

# ANNUAL SERVICING SCHEDULE

## AC310 Compressor (~1)

(with HEPA Filter for drying column)



\* Servicing schedule also applies to "E" version of this compressor

Year	Minor Service Kits	Intermediate Service Kits
1	(x1) K-SK3013B Inlet filters & O-ring	
2	(x1) K-SK3013B Inlet filters & O-ring	
3		(x1) K-SK301B Capacitor, Inlet filters & O-ring
4	(x1) K-SK3013B Inlet filters & O-ring	
5	(x1) K-SK3013B Inlet filters & O-ring	
6		(x1) K-SK301B Capacitor, Inlet filters & O-ring
7	(x1) K-SK3013B Inlet filters & O-ring	
8	(x1) K-SK3013B Inlet filters & O-ring	
9		(x1) K-SK301B Capacitor, Inlet filters & O-ring
10	(x1) K-SK3013B Inlet filters & O-ring	

# ANNUAL SERVICING SCHEDULE

## AC310Q Compressor (~1)

(with HEPA Filter for drying column)



\* Servicing schedule also applies to "E" version of this compressor

Year	Minor Service Kits	Intermediate Service Kits
1	(x1) K-SK3013B Inlet filters & O-ring	
2	(x1) K-SK3013B Inlet filters & O-ring	
3		(x1) K-SK301B Capacitor, Inlet filters & O-ring
4	(x1) K-SK3013B Inlet filters & O-ring	
5	(x1) K-SK3013B Inlet filters & O-ring	
6		(x1) K-SK301B Capacitor, Inlet filters & O-ring
7	(x1) K-SK3013B Inlet filters & O-ring	
8	(x1) K-SK3013B Inlet filters & O-ring	
9		(x1) K-SK301B Capacitor, Inlet filters & O-ring
10	(x1) K-SK3013B Inlet filters & O-ring	

# ANNUAL SERVICING SCHEDULE

## AC4101P Compressor (~1)

(with HEPA Filter for drying column)



Year	Minor Service Kits	Intermediate Service Kits
1	(x2) K-SK2013B Inlet filters & O-ring	
2	(x2) K-SK2013B Inlet filters & O-ring	
3		(x2) K-SK201B Capacitor, Inlet filters & O-ring
4	(x2) K-SK2013B Inlet filters & O-ring	
5	(x2) K-SK2013B Inlet filters & O-ring	
6		(x2) K-SK201B Capacitor, Inlet filters & O-ring
7	(x2) K-SK2013B Inlet filters & O-ring	
8	(x2) K-SK2013B Inlet filters & O-ring	
9		(x2) K-SK201B Capacitor, Inlet filters & O-ring
10	(x2) K-SK2013B Inlet filters & O-ring	

\* Servicing schedule also applies to "E" version of this compressor



# ANNUAL SERVICING SCHEDULE

## AC410Q1P Compressor (~1)

(with HEPA Filter for drying column)



\* Servicing schedule also applies to "E" version of this compressor

Year	Minor Service Kits	Intermediate Service Kits
1	(x2) K-SK2013B Inlet filters & O-ring	
2	(x2) K-SK2013B Inlet filters & O-ring	
3		(x2) K-SK201B Capacitor, Inlet filters & O-ring
4	(x2) K-SK2013B Inlet filters & O-ring	
5	(x2) K-SK2013B Inlet filters & O-ring	
6		(x2) K-SK201B Capacitor, Inlet filters & O-ring
7	(x2) K-SK2013B Inlet filters & O-ring	
8	(x2) K-SK2013B Inlet filters & O-ring	
9		(x2) K-SK201B Capacitor, Inlet filters & O-ring
10	(x2) K-SK2013B Inlet filters & O-ring	

# ANNUAL SERVICING SCHEDULE

## AC410 Compressor (~3)

(with HEPA Filter for drying column)



Year	Minor Service Kits
1	(x2) K-SK2013B Inlet filters & O-ring
2	(x2) K-SK2013B Inlet filters & O-ring
3	(x2) K-SK2013B Inlet filters & O-ring
4	(x2) K-SK2013B Inlet filters & O-ring
5	(x2) K-SK2013B Inlet filters & O-ring
6	(x2) K-SK2013B Inlet filters & O-ring
7	(x2) K-SK2013B Inlet filters & O-ring
8	(x2) K-SK2013B Inlet filters & O-ring
9	(x2) K-SK2013B Inlet filters & O-ring
10	(x2) K-SK2013B Inlet filters & O-ring

\* Servicing schedule also applies to "E" version of this compressor

# ANNUAL SERVICING SCHEDULE

## AC410Q Compressor (~3)

(with HEPA Filter for drying column)



\* Servicing schedule also applies to "E" version of this compressor

Year	Minor Service Kits
1	(x2) K-SK2013B Inlet filters & O-ring
2	(x2) K-SK2013B Inlet filters & O-ring
3	(x2) K-SK2013B Inlet filters & O-ring
4	(x2) K-SK2013B Inlet filters & O-ring
5	(x2) K-SK2013B Inlet filters & O-ring
6	(x2) K-SK2013B Inlet filters & O-ring
7	(x2) K-SK2013B Inlet filters & O-ring
8	(x2) K-SK2013B Inlet filters & O-ring
9	(x2) K-SK2013B Inlet filters & O-ring
10	(x2) K-SK2013B Inlet filters & O-ring

# ANNUAL SERVICING SCHEDULE

## AC610 Compressor (~3)

(with HEPA Filter for drying column)



Year	Minor Service Kits
1	(x2) K-SK3013B Inlet filters & O-ring
2	(x2) K-SK3013B Inlet filters & O-ring
3	(x2) K-SK3013B Inlet filters & O-ring
4	(x2) K-SK3013B Inlet filters & O-ring
5	(x2) K-SK3013B Inlet filters & O-ring
6	(x2) K-SK3013B Inlet filters & O-ring
7	(x2) K-SK3013B Inlet filters & O-ring
8	(x2) K-SK3013B Inlet filters & O-ring
9	(x2) K-SK3013B Inlet filters & O-ring
10	(x2) K-SK3013B Inlet filters & O-ring

\* Servicing schedule also applies to "E" version of this compressor

# ANNUAL SERVICING SCHEDULE

## AC610Q Compressor (~3)

(with HEPA Filter for drying column)



Year	Minor Service Kits
1	(x2) K-SK3013B Inlet filters & O-ring
2	(x2) K-SK3013B Inlet filters & O-ring
3	(x2) K-SK3013B Inlet filters & O-ring
4	(x2) K-SK3013B Inlet filters & O-ring
5	(x2) K-SK3013B Inlet filters & O-ring
6	(x2) K-SK3013B Inlet filters & O-ring
7	(x2) K-SK3013B Inlet filters & O-ring
8	(x2) K-SK3013B Inlet filters & O-ring
9	(x2) K-SK3013B Inlet filters & O-ring
10	(x2) K-SK3013B Inlet filters & O-ring

\* Servicing schedule also applies to "E" version of this compressor

# ANNUAL SERVICING SCHEDULE


## AC910 Compressor (~3)

(with HEPA Filter for drying column)



Year	Minor Service Kits
1	(x3) K-SK3013B Inlet filters & O-ring
2	(x3) K-SK3013B Inlet filters & O-ring
3	(x3) K-SK3013B Inlet filters & O-ring
4	(x3) K-SK3013B Inlet filters & O-ring
5	(x3) K-SK3013B Inlet filters & O-ring
6	(x3) K-SK3013B Inlet filters & O-ring
7	(x3) K-SK3013B Inlet filters & O-ring
8	(x3) K-SK3013B Inlet filters & O-ring
9	(x3) K-SK3013B Inlet filters & O-ring
10	(x3) K-SK3013B Inlet filters & O-ring

\* Servicing schedule also applies to "E" version of this compressor

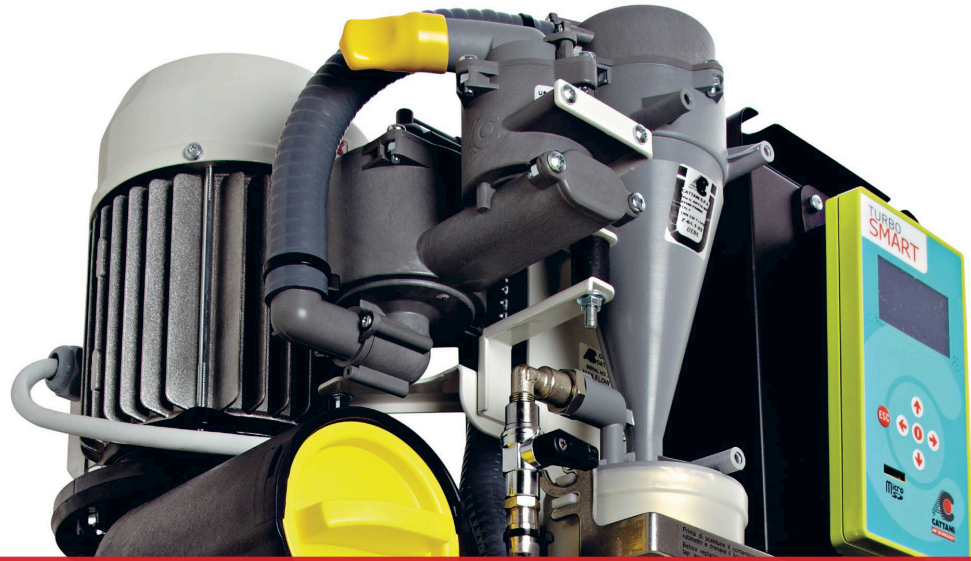

# SMART SERIES SUCTION



# SMART SERIES SUCTION ANNUAL SERVICING SCHEDULE

**Cattani SMART Suction Units are serviced annually. There are three service intervals: Annual, Intermediate and Major Servicing.**

## **Annual Servicing (2,000 Hrs/1 Year)**

– Suction inlet filter o-ring, Lubri-jet oil (for o-ring) and system check

## **Intermediate Servicing (4,000 Hrs/2 Years)**

– Annual Service parts and Service Kit for Hydrocyclone Unit (Amalgam Separator)

## **Major Servicing (10,000 Hrs/5 Years)**

– Annual Service parts and Service Kit for Suction Motor, Separator and Internal Hose Kit

Refer to Page 81 to view  
Aftercare SMART Suction Service Report.



# ANNUAL SERVICING SCHEDULE

## Micro SMART



Year	Minor Service Kits	Major Service Kits
1	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
2	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
3	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
4	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
5	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MSMSK0 Micro SMART Major Service Kit Includes: (x1) C-025401-SK (Micro SMART Separator) (x1) C-014057-SK (Micro SMART Blower) (x1) C-MSIHSK (Internal Hose)
6	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
7	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
8	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
9	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
10	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE

## Micro SMART with Hydrocyclone



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
2	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MR60SK ISO 6 Hydrocyclone	
3	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
4	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MR60SK ISO 6 Hydrocyclone	
5	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		(x1) C-MSMSK0 Micro SMART Major Service Kit Includes: (x1) C-025401-SK (Micro SMART Separator) (x1) C-014057-SK (Micro SMART Blower) (x1) C-MSIHSK (Internal Hose)
6	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MR60SK ISO 6 Hydrocyclone	
7	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
8	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MR60SK ISO 6 Hydrocyclone	
9	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
10	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MR60SK ISO 6 Hydrocyclone	

# ANNUAL SERVICING SCHEDULE

## Micro SMART Cube



Year	Minor Service Kits	Major Service Kits
1	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
2	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
3	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
4	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
5	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MSCMSK Micro SMART CUBE Major Service Kit Includes: (x1) C-025401-SK (Micro SMART Separator) (x1) C-017500-SK (Micro SMART Cube Blower) (x1) C-MSCIHO-SK (Internal Hose)
6	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
7	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
8	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
9	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
10	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE

## Micro SMART Cube with Hydrocyclone



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
2	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MR60SK ISO 6 Hydrocyclone	
3	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
4	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MR60SK ISO 6 Hydrocyclone	
5	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		(x1) C-MSCMSK Micro SMART CUBE Major Service Kit Includes: (x1) C-025401-SK (Micro SMART Separator) (x1) C-017500-SK (Micro SMART Cube Blower) (x1) C-MSCIH0-SK (Internal Hose)
6	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MR60SK ISO 6 Hydrocyclone	
7	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
8	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MR60SK ISO 6 Hydrocyclone	
9	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
10	(x1) C-211956 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-MR60SK ISO 6 Hydrocyclone	

# ANNUAL SERVICING SCHEDULE

## Turbo SMART A



Year	Minor Service Kits	Major Service Kits
1	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
2	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
3	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
4	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
5	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x1) C-0034MS-3SK (Turbo SMART Separator) (x1) C-1M0000-SK (Turbo SMART Blower) (x1) C-TSIHSK (Internal Hose)
6	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
7	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
8	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
9	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
10	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE

## Turbo SMART A with Hydrocyclone



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
2	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
3	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
4	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
5	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		(x1) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x1) C-0034MS-3SK (Turbo SMART Separator) (x1) C-1M0000-SK (Turbo SMART Blower) (x1) C-TSIHSK (Internal Hose)
6	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
7	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
8	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
9	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
10	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(1x) C-1H00SK ISO18 Hydrocyclone	

# ANNUAL SERVICING SCHEDULE

## Turbo SMART B



Year	Minor Service Kits	Major Service Kits
1	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
2	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
3	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
4	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
5	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x1) C-0034MS-3SK (Turbo SMART Separator) (x1) C-1M0000-SK (Turbo SMART Blower) (x1) C-TSIHSK (Internal Hose)
6	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
7	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
8	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
9	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
10	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE

## Turbo SMART B with Hydrocyclone



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
2	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
3	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
4	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
5	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		(x1) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x1) C-0034MS-3SK (Turbo SMART Separator) (x1) C-1M0000-SK (Turbo SMART Blower) (x1) C-TSIHSK (Internal Hose)
6	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
7	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
8	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
9	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
10	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(1x) C-1H00SK ISO18 Hydrocyclone	



# ANNUAL SERVICING SCHEDULE

## Turbo SMART A Cube



Year	Minor Service Kits	Major Service Kits
1	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
2	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
3	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
4	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
5	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-TSCMSK Turbo SMART Cube Major Service Kit Includes: (x1) C-0034MS-3SK (Turbo SMART Separator) (x1) C-014095-SK (Turbo SMART Cube Blower) (x1) C-TSCIH0-SK (Internal Hose)
6	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
7	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
8	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
9	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
10	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE


## Turbo SMART A Cube with Hydrocyclone



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
2	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
3	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
4	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
5	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		(x1) C-TSCMSK Turbo SMART Cube Major Service Kit Includes: (x1) C-0034MS-3SK (Turbo SMART Separator) (x1) C-014095-SK (Turbo SMART Cube Blower) (x1) C-TSCHI0-SK (Internal Hose)
6	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
7	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
8	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
9	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
10	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(1x) C-1H00SK ISO18 Hydrocyclone	

# ANNUAL SERVICING SCHEDULE

## Turbo SMART B Cube

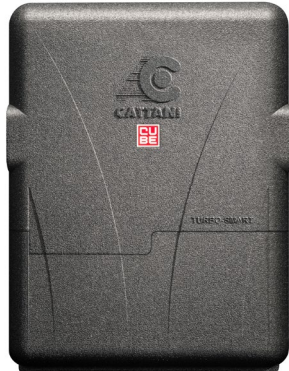

Year	Minor Service Kits	Major Service Kits
1	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
2	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
3	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
4	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
5	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-TSCMSK Turbo SMART Cube Major Service Kit Includes: (x1) C-0034MS-3SK (Turbo SMART Separator) (x1) C-014095-SK (Turbo SMART Cube Blower) (x1) C-TSCIH0-SK (Internal Hose)
6	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
7	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
8	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
9	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	
10	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE


## Turbo SMART B Cube with Hydrocyclone



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
2	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
3	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
4	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
5	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		(x1) C-TSCMSK Turbo SMART Cube Major Service Kit Includes: (x1) C-0034MS-3SK (Turbo SMART Separator) (x1) C-014095-SK (Turbo SMART Cube Blower) (x1) C-TSCIH0-SK (Internal Hose)
6	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
7	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
8	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-1H00SK ISO18 Hydrocyclone	
9	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
10	(x1) C-211974 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(1x) C-1H00SK ISO18 Hydrocyclone	

# ANNUAL SERVICING SCHEDULE

## Tandem Micro SMART

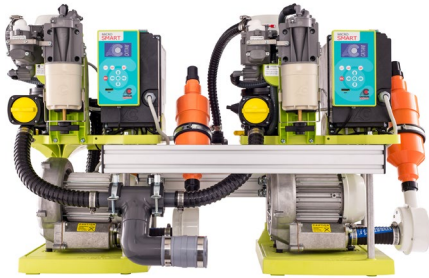

Year	Minor Service Kits	Major Service Kits
1	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
2	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
3	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
4	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
5	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	<b>(x2) C-MSMSK0 Micro SMART Major Service Kit</b> Includes: (x2) C-025401-SK (Micro SMART Separator) (x2) C-014057-SK (Micro SMART Blower) (x2) C-MSIHSK (Internal Hose)
6	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
7	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
8	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
9	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
10	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE

## Tandem Micro SMART with Hydrocyclone



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
2	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-MR60SK ISO 6 Hydrocyclone	
3	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
4	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-MR60SK ISO 6 Hydrocyclone	
5	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		(x2) C-MSMSK0 Micro SMART Major Service Kit Includes: (x2) C-025401-SK (Micro SMART Separator) (x2) C-014057-SK (Micro SMART Blower) (x2) C-MSIHSK (Internal Hose)
6	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-MR60SK ISO 6 Hydrocyclone	
7	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
8	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-MR60SK ISO 6 Hydrocyclone	
9	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
10	(x2) C-211956 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-MR60SK ISO 6 Hydrocyclone	

# ANNUAL SERVICING SCHEDULE

## Tandem Turbo SMART A



Year	Minor Service Kits	Major Service Kits
1	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
2	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
3	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
4	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
5	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x2) C-0034MS-3SK (Turbo SMART Separator) (x2) C-1M0000-SK (Turbo SMART Blower) (x2) C-TSIHSC (Internal Hose)
6	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
7	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
8	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
9	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
10	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE

## Tandem Turbo SMART A with Hydrocyclone



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
2	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-1H00SK ISO18 Hydrocyclone	
3	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
4	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-1H00SK ISO18 Hydrocyclone	
5	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		(x2) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x2) C-0034MS-35K (Turbo SMART Separator) (x2) C-1M0000-SK (Turbo SMART Blower) (x2) C-TSIHSK (Internal Hose)
6	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-1H00SK ISO18 Hydrocyclone	
7	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
8	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-1H00SK ISO18 Hydrocyclone	
9	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
10	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-1H00SK ISO18 Hydrocyclone	



# ANNUAL SERVICING SCHEDULE

## Tandem Turbo SMART B



Year	Minor Service Kits	Major Service Kits
1	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
2	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
3	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
4	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
5	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-TSMK0 Turbo SMART Major Service Kit Includes: (x2) C-0034MS-3SK (Turbo SMART Separator) (x2) C-1M0000-SK (Turbo SMART Blower) (x2) C-TSIHSK (Internal Hose)
6	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
7	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
8	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
9	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	
10	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE

## Tandem Turbo SMART B with Hydrocyclone



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
2	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-1H00SK ISO18 Hydrocyclone	
3	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
4	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-1H00SK ISO18 Hydrocyclone	
5	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		(x2) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x2) C-0034MS-3SK (Turbo SMART Separator) (x2) C-1M0000-SK (Turbo SMART Blower) (x2) C-TSIHSK (Internal Hose)
6	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-1H00SK ISO18 Hydrocyclone	
7	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
8	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-1H00SK ISO18 Hydrocyclone	
9	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)		
10	(x2) C-211974 Inlet filter o-ring (x2) M-040727 Lubrijet (single bottle)	(x2) C-1H00SK ISO18 Hydrocyclone	

# ANNUAL SERVICING SCHEDULE

## Multi SMART T3



Year	Minor Service Kits	Major Service Kits
1	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	
2	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	
3	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	
4	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	
5	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	(x3) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x3) C-0034MS-3SK (Turbo SMART Separator) (x3) C-1M0000-SK (Turbo SMART Blower) (x3) C-TSIHSK (Internal Hose)
6	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	
7	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	
8	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	
9	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	
10	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	

\* Replace (3x) C-160540 annually if HEPA Filters are fitted.

# ANNUAL SERVICING SCHEDULE

## Multi SMART T3 with Hydrocyclone



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)		
2	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	(x3) C-1H00SK ISO18 Hydrocyclone	
3	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)		
4	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	(x3) C-1H00SK ISO18 Hydrocyclone	
5	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)		(x3) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x3) C-0034MS-3SK (Turbo SMART Separator) (x3) C-1M0000-SK (Turbo SMART Blower) (x3) C-TSIH5K (Internal Hose)
6	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	(x3) C-1H00SK ISO18 Hydrocyclone	
7	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)		
8	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	(x3) C-1H00SK ISO18 Hydrocyclone	
9	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)		
10	(x3) C-211974 Inlet filter o-ring (x3) M-040727 Lubrijet (single bottle)	(x3) C-1H00SK ISO18 Hydrocyclone	

\* Replace (3x) C-160540 annually if HEPA Filters are fitted.

# ANNUAL SERVICING SCHEDULE

## Multi SMART T4




Year	Minor Service Kits	Major Service Kits
1	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	
2	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	
3	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	
4	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	
5	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	(x4) C-TSMK0 Turbo SMART Major Service Kit Includes: (x4) C-0034MS-3SK (Turbo SMART Separator) (x4) C-1M0000-SK (Turbo SMART Blower) (x4) C-TSIHSK (Internal Hose)
6	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	
7	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	
8	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	
9	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	
10	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	

\* Replace (4x) C-160540 annually if HEPA Filters are fitted.

# ANNUAL SERVICING SCHEDULE

## Multi SMART T4 with Hydrocyclone

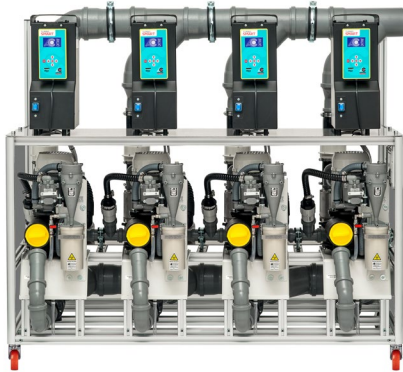


Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)		
2	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	(x4) C-1H00SK ISO18 Hydrocyclone	
3	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)		
4	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	(x4) C-1H00SK ISO18 Hydrocyclone	
5	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)		(x4) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x4) C-0034MS-3SK (Turbo SMART Separator) (x4) C-1M0000-SK (Turbo SMART Blower) (x4) C-TSIHSK (Internal Hose)
6	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	(x4) C-1H00SK ISO18 Hydrocyclone	
7	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)		
8	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	(x4) C-1H00SK ISO18 Hydrocyclone	
9	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)		
10	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)	(x4) C-1H00SK ISO18 Hydrocyclone	

\* Replace (4x) C-160540 annually if HEPA Filters are fitted.

# ANNUAL SERVICING SCHEDULE

## Multi SMART T5

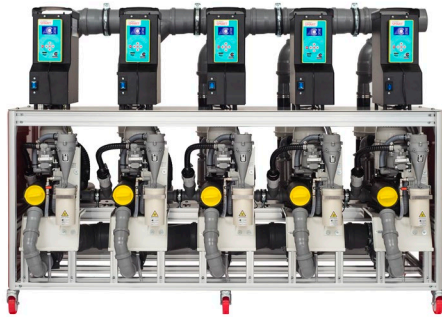


Year	Minor Service Kits	Major Service Kits
1	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	
2	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	
3	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	
4	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	
5	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	(x5) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x5) C-0034MS-3SK (Turbo SMART Separator) (x5) C-1M0000-SK (Turbo SMART Blower) (x5) C-TSIHSK (Internal Hose)
6	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	
7	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	
8	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	
9	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	
10	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	

\* Replace (5x) C-160540 annually if HEPA Filters are fitted.

# ANNUAL SERVICING SCHEDULE

## Multi SMART T5 with Hydrocyclone

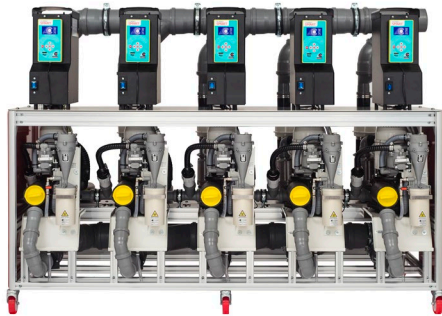

Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)		
2	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	(x5) C-1H00SK ISO18 Hydrocyclone	
3	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)		
4	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	(x5) C-1H00SK ISO18 Hydrocyclone	
5	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)		(x5) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x5) C-0034MS-3SK (Turbo SMART Separator) (x5) C-1M0000-SK (Turbo SMART Blower) (x5) C-TSIHSK (Internal Hose)
6	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	(x5) C-1H00SK ISO18 Hydrocyclone	
7	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)		
8	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	(x5) C-1H00SK ISO18 Hydrocyclone	
9	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)		
10	(x5) C-211974 Inlet filter o-ring (x5) M-040727 Lubrijet (single bottle)	(x5) C-1H00SK ISO18 Hydrocyclone	

\* Replace (5x) C-160540 annually if HEPA Filters are fitted.



# ANNUAL SERVICING SCHEDULE

## Maxi SMART with Hydrocyclone and Acoustic Cover



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
2	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-043390-SK (ISO60 Hydrocyclone)	
3	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
4	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-043390-SK (ISO60 Hydrocyclone)	
5	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		(x1) C-MAXIMSK Maxi SMART Major Service Kit (x1) C-025408-SK (Maxi SMART Separator) (x1) C-091655-SK (Maxi SMART Blower) (x1) C-MXIHSK (Internal Hose)
6	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-043390-SK (ISO60 Hydrocyclone)	
7	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
8	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-043390-SK (ISO60 Hydrocyclone)	
9	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
10	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-043390-SK (ISO60 Hydrocyclone)	

# ANNUAL SERVICING SCHEDULE

**Maxi SMART  
with hydrocyclone  
without Acoustic Cover**



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
1	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
2	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-043390-SK (ISO60 Hydrocyclone)	
3	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
4	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-043390-SK (ISO60 Hydrocyclone)	
5	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		(x1) C-MAXIMSK Maxi SMART Major Service Kit Includes: (x1) C-025408-SK (Maxi SMART Separator) (x1) C-091655-SK (Maxi SMART Blower) (x1) C-MXIHSK (Internal Hose)
6	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-043390-SK (ISO60 Hydrocyclone)	
7	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
8	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-043390-SK (ISO60 Hydrocyclone)	
9	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)		
10	(x1) C-215986 Inlet filter o-ring (x1) M-040727 Lubrijet (single bottle)	(x1) C-043390-SK (ISO60 Hydrocyclone)	

# ANNUAL SERVICING SCHEDULE

## Blok Jet Large Central Vacuum System (200 Chair)



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
0.5	(x2) C-AS1000 - Antifoam Liquid		
1	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle) (x2) C-AS1000 - Antifoam Liquid (x3) C-042094 -HEPA H14 Filter Cartridge		
1.5	(x2) C-AS1000 -Antifoam Liquid (x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)		
2	(x2) C-AS1000 - Antifoam Liquid (x3) C-042094 -HEPA H14 Filter Cartridge	(x4) C-1H00SK ISO18 Hydrocyclone	
2.5	(x2) C-AS1000 - Antifoam Liquid (x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)		
3	(x2) C-AS1000 - Antifoam Liquid (x3) C-042094 -HEPA H14 Filter Cartridge		
3.5	(x2) C-AS1000 - Antifoam Liquid (x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle)		
4	(x2) C-AS1000 - Antifoam Liquid (x3) C-042094 -HEPA H14 Filter Cartridge	(x4) C-1H00SK ISO18 Hydrocyclone	
4.5	(x2) C-AS1000 - Antifoam Liquid		
5	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle) (x2) C-AS1000 - Antifoam Liquid (x3) C-042094 -HEPA H14 Filter Cartridge		(x4) C-083067-SK (Suction Pump – Blokjet) (x4) C-TSMSK0 Turbo SMART Major Service Kit Includes: (x4) C-0034MS-3SK (Turbo SMART Separator) (x4) C-1M0000-SK (Turbo SMART Blower) (x4) C-TSIHSK (Internal Hose)

# ANNUAL SERVICING SCHEDULE

## Blok Jet Large Central Vacuum System (200 Chair)



Year	Minor Service Kits	Intermediate Service Kits	Major Service Kits
5.5	(x2) C-AS1000 - Antifoam Liquid		
6	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle) (x2) C-AS1000 - Antifoam Liquid (x3) C-042094 -HEPA H14 Filter Cartridge		
6.5	(x2) C-AS1000 -Antifoam Liquid		
7	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle) (x2) C-AS1000 - Antifoam Liquid (x3) C-042094 -HEPA H14 Filter Cartridge	(x4) C-1H00SK ISO18 Hydrocyclone	
7.5	(x2) C-AS1000 - Antifoam Liquid		
8	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle) (x2) C-AS1000 - Antifoam Liquid (x3) C-042094 -HEPA H14 Filter Cartridge		
8.5	(x2) C-AS1000 - Antifoam Liquid		
9	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle) (x2) C-AS1000 - Antifoam Liquid (x3) C-042094 -HEPA H14 Filter Cartridge	(x4) C-1H00SK ISO18 Hydrocyclone	
9.5	(x2) C-AS1000 - Antifoam Liquid		
10	(x4) C-211974 Inlet filter o-ring (x4) M-040727 Lubrijet (single bottle) (x2) C-AS1000 - Antifoam Liquid (x3) C-042094 -HEPA H14 Filter Cartridge		(x4) C-083067-SK (Suction Pump – Blok Jet) *Assess Turbo SMART for life expectancy

# ANNUAL SERVICING SCHEDULE

## Aspi Jet 6



Year	Minor Service Kits	Major Service Kits
1	[x1] C-000600-SK [x1] M-040727 Lubrijet (single bottle)	
2	[x1] C-000600-SK [x1] M-040727 Lubrijet (single bottle)	
3	[x1] C-000600-SK [x1] M-040727 Lubrijet (single bottle)	
4	[x1] C-000600-SK [x1] M-040727 Lubrijet (single bottle)	
5	[x1] C-000600-SK [x1] M-040727 Lubrijet (single bottle)	[x1] C-002300-SK (Blower Uni Jet 75 ~1)
6	[x1] C-000600-SK [x1] M-040727 Lubrijet (single bottle)	
7	[x1] C-000600-SK [x1] M-040727 Lubrijet (single bottle)	
8	[x1] C-000600-SK [x1] M-040727 Lubrijet (single bottle)	
9	[x1] C-000600-SK [x1] M-040727 Lubrijet (single bottle)	
10	[x1] C-000600-SK [x1] M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE

## Aspi Jet 6 Fitted with Bacterial



Year	Minor Service Kits	Major Service Kits
1	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
2	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
3	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
4	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
5	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	(x1) C-002300-SK (Blower Uni Jet 75 ~1)
6	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
7	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
8	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
9	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
10	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE

## Aspi Jet 7



Year	Minor Service Kits	Major Service Kits
1	(x1) C-000600-SK (x1) M-040727 Lubrijet (single bottle)	
2	(x1) C-000600-SK (x1) M-040727 Lubrijet (single bottle)	
3	(x1) C-000600-SK (x1) M-040727 Lubrijet (single bottle)	
4	(x1) C-000600-SK (x1) M-040727 Lubrijet (single bottle)	
5	(x1) C-000600-SK (x1) M-040727 Lubrijet (single bottle)	(x1) C-002300-SK (Blower Uni Jet 75 ~1)
6	(x1) C-000600-SK (x1) M-040727 Lubrijet (single bottle)	
7	(x1) C-000600-SK (x1) M-040727 Lubrijet (single bottle)	
8	(x1) C-000600-SK (x1) M-040727 Lubrijet (single bottle)	
9	(x1) C-000600-SK (x1) M-040727 Lubrijet (single bottle)	
10	(x1) C-000600-SK (x1) M-040727 Lubrijet (single bottle)	

# ANNUAL SERVICING SCHEDULE

## Aspi Jet 7 Fitted with Bacterial



Year	Minor Service Kits	Major Service Kits
1	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
2	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
3	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
4	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
5	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	(x1) C-002300-SK (Blower Uni Jet 75 ~1)
6	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
7	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
8	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
9	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	
10	(x1) C-000600-SKB (x1) M-040727 Lubrijet (single bottle)	



# ANNUAL SERVICING SCHEDULE


## Turbo Jet 1 Modular



Year	Major Service Kits
1	
2	
3	
4	
5	(x1) C-002300-SK (Blower Uni Jet 75 ~1) (x1) C-0033M0-SK (T1/T2 Separator ~1)
6	
7	
8	
9	
10	

# ANNUAL SERVICING SCHEDULE

## Turbo Jet 2 Modular

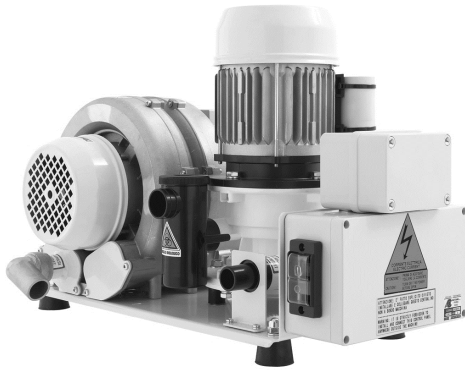

Year	Major Service Kits
1	
2	
3	
4	
5	(x1) C-0023V0-SK (Blower Uni Jet 75 2V ~1) (x1) C-0033M0-SK (T1/T2 Separator ~1)
6	
7	
8	
9	
10	

# ANNUAL SERVICING SCHEDULE

**Uni Jet 75 - 0.4 kW**  
**Single-Phase**



Year	Major Service Kits
1	
2	
3	
4	
5	(x1) C-002300-SK (Blower Uni Jet 75 ~1)
6	
7	
8	
9	
10	

# ANNUAL SERVICING SCHEDULE

**Uni Jet 75 2V - 0.7 kW**  
**Single-Phase**



Year	Major Service Kits
1	
2	
3	
4	
5	(x1) C-0023V0-SK (Blower Uni Jet 75 2V -1)
6	
7	
8	
9	
10	

# ANNUAL SERVICING SCHEDULE

**Tecno Jet - 0.75 kW**  
**Single-Phase**



Year	Major Service Kits
1	
2	
3	
4	
5	(x1) C-006900-SK (Blower Tecno Jet ~1)
6	
7	
8	
9	
10	

# ANNUAL SERVICING SCHEDULE

**Flux Jet - 1.1 kW**  
**Single-Phase**



Year	Major Service Kits
1	
2	
3	
4	
5	(x1) C-014000-SK (Blower Flux Jet ~1)
6	
7	
8	
9	
10	

# ANNUAL SERVICING SCHEDULE

**Medio Jet - 3 kW**  
**Three-Phase**



Year	Major Service Kits
1	
2	
3	
4	
5	(x1) C-A00150-SK (Blower Medio Jet ~3)
6	
7	
8	
9	
10	

# AFTERCARE SERVICE REPORT COMPRESSORS

Contact our Technical Service Department  
for more information.

## AFTERCARE SERVICE REPORT (COMPRESSORS)

Job Ref: \_\_\_\_\_ Model #: \_\_\_\_\_  
 Service Company: \_\_\_\_\_ Purchase date: \_\_\_\_\_ Hours run: \_\_\_\_\_  
 Technician: \_\_\_\_\_ Serial #: \_\_\_\_\_  
 Customer Name: \_\_\_\_\_ Warranty Exp.: \_\_\_\_\_ (7 Yr. Parts & Labour)  
 Business Name: \_\_\_\_\_  
 Contact Details - Phone: \_\_\_\_\_ Email: \_\_\_\_\_

### AFTERCARE SERVICE SCHEDULE

Service Due	1 Phase Compressor	3 Phase Compressor	Kit Qty.	Service Due	1 Phase Compressor	3 Phase Compressor	Kit Qty.
Year 1	Minor Service Kit	Minor Service Kit		Year 2	Intermediate Service Kit	Minor Service Kit	
Year 2	Minor Service Kit	Minor Service Kit		Year 3	Minor Service Kit	Minor Service Kit	
Year 3	Intermediate Service Kit	Minor Service Kit		Year 4	Minor Service Kit	Minor Service Kit	
Year 4	Minor Service Kit	Minor Service Kit		Year 5	Intermediate Service Kit	Minor Service Kit	
Year 5	Minor Service Kit	Minor Service Kit		Year 6	Minor Service Kit	Minor Service Kit	

PLEASE NOTE: All compressor service kits include a HEPA H14 antibacterial filter for the drying column.

### MINOR SERVICE TASKS

Task	Done
Replace air inlet filters on compressor head(s).	
Replace filter in the top of each drying column. Replacement filter(s): <input type="checkbox"/> HEPA	
Replace O-ring seal on the top cover of the drying column(s).	

### INTERMEDIATE SERVICE TASKS

Task	Done
Replace air inlet filters on compressor head(s).	
Replace filter in the top of each drying column. Replacement filter(s): <input type="checkbox"/> HEPA	
Replace O-ring seal on the top cover of the drying column(s).	
Replace motor start capacitor.	

### TEST & INSPECT TASKS

Task	Status	Notes
Check tension of cylinder head bolts (5Nm).	<input type="checkbox"/> OK	
Check for fluid in tank.	<input type="checkbox"/> OK	
Time duration to fill the tank with air: (6 Bar ON / 8 Bar OFF) Switch off between 45-60 seconds!	<input type="checkbox"/> OK	Time: [sec]
Check direction of motor rotation (3ph only).	<input type="checkbox"/> OK	
Check valve operation in each cylinder during operation.	<input type="checkbox"/> OK	
Check operation of the radiator cooling fan.	<input type="checkbox"/> OK	
Check drying system operation.	<input type="checkbox"/> OK	
Inspect all plumbing hoses and connections.	<input type="checkbox"/> OK	
Check for abnormal noises or air leaks during operation.	<input type="checkbox"/> OK	
Inspect plant room ventilation and verify specification for the site. Formula: Watts x 0.15 = m <sup>3</sup> /hr.	<input type="checkbox"/> OK	
Measure plant room temperature. Ambient temperature must not exceed 30°C.	<input type="checkbox"/> OK	

Signed: \_\_\_\_\_ Date: \_\_\_\_\_  
 Australia +61 3 9484 1120 New Zealand 0800 48 22 88  
 cattani@cattani.com.au cattani@cattani.co.nz WWW.CATTANI.COM.AU





# AFTERCARE SERVICE REPORT SUCTION SYSTEMS

Contact our Technical Service Department  
for more information.

## AFTERCARE SERVICE REPORT (SMART SUCTION)

Job Ref: \_\_\_\_\_ Model #: \_\_\_\_\_  
 Service Company: \_\_\_\_\_ Purchase date: \_\_\_\_\_ Hours run: \_\_\_\_\_  
 Technician: \_\_\_\_\_ Serial #: \_\_\_\_\_  
 Customer Name: \_\_\_\_\_ Warranty Exp: \_\_\_\_\_ (3 Yr Parts & Labour)  
 Business Name: \_\_\_\_\_  
 Contact Details - Phone: \_\_\_\_\_ Email: \_\_\_\_\_

### AFTERCARE SERVICE SCHEDULE

Service Due	No Amalgam Separator	With Amalgam Separator	Kit Qty.	Service Due	No Amalgam Separator	With Amalgam Separator	Kit Qty.
<input type="checkbox"/> Year 1	Minor Parts	Minor Parts		<input type="checkbox"/> Year 5	Minor Parts	Minor Parts & Intermediate Kit	
<input type="checkbox"/> Year 2	Minor Parts	Minor Parts & Intermediate Kit		<input type="checkbox"/> Year 7	Minor Parts	Minor Parts	
<input type="checkbox"/> Year 3	Minor Parts	Minor Parts		<input type="checkbox"/> Year 8	Minor Parts	Minor Parts & Intermediate Kit	
<input type="checkbox"/> Year 4	Minor Parts	Minor Parts & Intermediate Kit		<input type="checkbox"/> Year 9	Minor Parts	Minor Service Kit	
<input type="checkbox"/> Year 5	Minor Parts & Major Kit	Minor Parts & Major Kit		<input type="checkbox"/> Year 10	Minor Parts	Minor Parts & Intermediate Kit	

NOTE: All HEPA H14 antibacterial filters installed on exhaust pipework MUST be changed every 12 months.

### MINOR SERVICE TASKS

Task	Done
Clean & service suction inlet housing and filter. Replace inlet filter o-ring and lubricate with Lubri-Jet silicone oil.	
If fitted with amalgam separator: Remove amalgam cup, inspect nozzle at the base of the hydrocyclone cone for blockages, lubricate seal and cup locking guides.	
Inspect all tubing (suction inlet, fluid waste outlet, exhaust and internal) checking for loose connections or leaks. Repair as necessary.	

### INTERMEDIATE SERVICE TASKS

Task	Done
Remove and fully dismantle amalgam separator assembly.	
Clean and service all internal parts. (Note condition of brass valve IC-224165) and cage IC-200227JL.	
Replace all o-ring seals and lubricate with Lubri-Jet silicone oil.	
Replace all grey ISO tubing.	
Reassemble amalgam separator assembly and refit to SMART suction unit.	

### MAJOR SERVICE TASKS

Task	Done
Remove centrifuge and suction motor from SMART suction unit.	
Fully dismantle centrifuge, replace bearings, oil seal, flap valve, o-rings, end cover & cooling fan and re-assemble.	
Fully dismantle suction motor, replace bearings, oil seal, foam & cages, gasket, end cover, cooling & rubber mounts and re-assemble.	
Fit centrifuge and suction motor in frame. Replace suction hose between suction motor and centrifuge. Refit exhaust tubing between suction motor and exhaust non-return valve.	

Australia +61 3 9486 1120 New Zealand 0800 68 22 88  
 cattani@cattani.com.au cattani@cattani.co.nz  
 Page 1 WWW.CATTANI.COM.AU



**DISCLAIMER**

Failure to observe a danger sign or warning may harm operators, patients or technical staff. Warning signs cannot always fully express all hazards and dangers associated with equipment repairs.

Prior to commencing work, ensure all appropriate Personal Protective Equipment (PPE) such as: gloves, mask, full sleeve gown and eye protection are worn. Refer to the product "User Manual" for manufacturer's recommendations.

**Cattani Technical Support:**

+61 3 9484 1120

[cattani@cattani.com.au](mailto:cattani@cattani.com.au)

MARCH22



**SALES AND TECHNICAL SUPPORT:**

+61 3 9484 1120  
CATTANI@CATTANI.COM.AU

**COMPANY ADDRESS:**

280 DUNDAS STREET  
THORBURY VIC 3071

[WWW.CATTANI.COM.AU](http://WWW.CATTANI.COM.AU)