

SMART THINKING, THE WAY WE INNOVATE.



# CATTANI AUSTRALIA

2026 CATALOGUE & PRICE LIST

VALID TO 31 DECEMBER 2026







At Cattani Australasia, we believe sustainability is more than a promise – it's how we do business. Through our **"Connected for Good"** initiative, we're shaping a cleaner, greener future for dentistry.

We've achieved **100% carbon neutrality**, backing projects that protect biodiversity and support Traditional Land Management. Our **Hydrocyclone Amalgam Retention system** sets the industry standard, effortlessly separating amalgam within our SMART systems – no extra power, no extra noise.

Our **SMART suction technology** is engineered to deliver top performance with less energy, helping clinics reduce their footprint without compromising care. With its innovative design, the **Turbo SMART Cube** cuts raw material use by 13 kg per unit – proof that intelligent engineering and sustainability go hand in hand.

Select Cattani – where reliable performance meets a genuine commitment to the planet.





## TABLE OF CONTENTS

### COMPRESSORS

Introduction	04
Products	06

### COMPRESSORS – LARGE PLANT

Introduction	18
Products	20

### COMPRESSORS – CAD/CAM

Introduction	24
Products	26

### COMPRESSOR ACCESSORIES AND SERVICE KITS

Compressor Accessories	35
Compressor Service Kits	37
Compressor Spare Parts	37

### SMART SUCTION

Introduction	38
SMART Cube	40
Inverter Technology	41
Amalgam Retention	42
Products	43

### LARGE PLANT SOLUTIONS

Introduction	62
Products	64

### COMPACT PLANT SOLUTIONS

Introduction	70
Products	72

### CLASSIC SUCTION

Introduction	74
Products	75

### SUCTION ACCESSORIES AND SERVICE KITS

Suction Accessories	81
Suction Service Kits	88
Suction Spare Parts	91

### CONSUMABLES – MAGNOLIA

Introduction	93
Products	96

### SELECTION, INSTALLATION, MAINTENANCE AND SERVICE

Introduction	108
Equipment Installation	110

## PRODUCT SELECTION GUIDE



COMPRESSORS

**1-2  
CHAIRS**

**PAGE 6-7  
PAGE 72**

**2-3  
CHAIRS**

**PAGE 8-9  
PAGE 72**

**3-4  
CHAIRS**

**PAGE 10-11  
PAGE 72**

**5-6  
CHAIRS**

**PAGE 12-15**

**7-8  
CHAIRS**

**PAGE 16-17**

**12+  
CHAIRS**

**PAGE 20-23**



SUCTION

**PAGE 43-46  
PAGE 72**

**PAGE 47-50  
PAGE 72**

**PAGE 51-57  
PAGE 72**

**PAGE 58**

**PAGE 60-61**

**PAGE 64-69**

\* Product selection is provided as a guide only. To determine the requirements of your dental clinic, contact our customer service and technical team.

Recommended Retail Price: The price set out or referred to herein is a recommended price only and there is no obligation to comply with the recommendation.

Prices are valid until 31st December 2026. Prices may be changed without notice due to unforeseen increases of imported products or currency fluctuations.

Cattani Australasia has a commitment to innovation and as such we reserve the right to change product specifications without notice. Please refer to our website for the most current product specifications. Product images and packaging are for illustrative purposes only, variations can occur due to the printing process of colours therefore the product you receive may not exactly correspond to the image. Some items listed are "indent only" products and are subject to our "Forward Order and Indent Item Policy." Please contact your Cattani Business Development Manager or our Customer Service Team for further information.

© 2025 Cattani Australia Pty Ltd. All rights reserved

## **DRIVEN BY IMAGINATION, RESEARCH IN DESIGN AND QUALITY IN PRODUCTION.**

Our premium air technology reflects everything Cattani stands for – innovation, reliability, and pride in craftsmanship. Every system we deliver is designed and engineered on a foundation of maximum reliability, a standard that defines us and connects us to our origins in Parma, Italy.


Parma, in the Emilia-Romagna region, is world-renowned for excellence – home to Parmigiano Reggiano, Prosciutto di Parma, and iconic brands like Ferrari, Lamborghini, Maserati, and Ducati. Within this culture of precision and artistry, the late Prof. Augusto Cattani founded Cattani in 1967, pioneering advanced dental suction and air compression technology.

By 1981, Cattani S.p.A. had grown into a publicly limited company and expanded into chemical products with the Magnolia consumables range, strengthening its air technology expertise. Today, that same passion and knowledge live on through the next generation, led by Ennio Cattani, and continue to reach partners in more than 60 countries – including Cattani Australasia in Melbourne, Australia.



*“All this painstaking care for our work... are justified by the results achieved. Quality needs to be sought with determination and must be scrupulously adhered to thereafter. We cannot afford to stop, as progress is unstoppable.”*

Prof. Augusto Cattani, Founder of Cattani

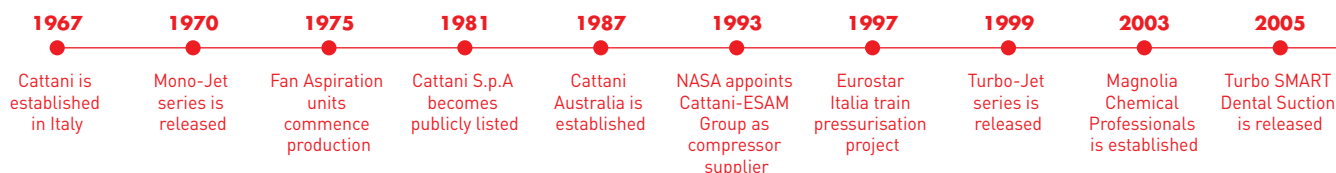



# QUALITY – AIR TECHNOLOGY SPECIALISTS

Owning Cattani equipment is a statement. It shows dental professionals are uncompromising in their pursuit of quality, performance, precision, and reliability.

## INNOVATION

For Cattani, innovation and quality are inseparable. Cattani Italy has consistently set the benchmark for intelligent suction systems and variable motor speeds to inverter control, automatic operation, and noise reduction. Even our designs reduce raw material use by 50%, proving that intelligent engineering can be robust and sustainable.

At Cattani Australasia, we embrace this legacy while driving innovation locally. With our dedicated technical team, we lead the modular installation of parallel units and respond quickly to market needs with in-house product development.

## EXPERTISE

With over 50 years of experience, rooted in Parma's world-famous engineering heritage – the home of Ferrari – Cattani has earned a global reputation for craftsmanship and precision. Our expertise is trusted worldwide, with the Cattani-ESAM group supplying compression technology for the NASA Space Shuttle and Eurostar Italia trains.

Cattani quality doesn't just happen – it results from relentless refinement and a singular focus on air technology, delivering unmatched mastery.

## SERVICE

Exceeding expectations is part of who we are. Every piece of equipment undergoes two quality inspections: first in Italy and then again by our Australian QA team. Our local technical experts support this assurance in product quality. They developed our **AFTER CARE Scheduled Service**, **BLUEPRINTS Training Program**, and **INSITES Training Program**, ensuring dental clinics receive long-term support and confidence.

## PERFORMANCE

At the heart of our design is the customer. Certainty in performance and reliability provides complete peace of mind for our dental professionals. Our compressors come with a **market-leading warranty**, and our dental suction solutions offer advanced solutions such as **30–50% energy savings**, **50% noise reduction**, and **ISO-certified precision**. Cattani delivers performance where it matters most – ensuring every clinic can rely on absolute accuracy in dentistry, all supported by our local, in-house technical team.



2009

Micro SMART  
Dental Suction  
is released

2013

Maxi SMART  
Dental Suction  
is released

2015

SMART CUBE  
range  
introduced

2019

BLUEPRINTS  
Training Program  
launched

2020

ADIA  
Education Award  
for training

2021

Turbo SMART HP  
338mBar dental  
suction launched

2023

AFTER CARE  
Training online  
launched

2026

Next-generation  
Turbo SMART TS  
released

2026

INSITES  
Training Program  
launched

## PURE RELIABILITY

Selecting Cattani means selecting uncompromising quality, performance, precision, and reliability. You can always make this decision with confidence. That's why a market-leading warranty backs every Cattani compressor.

### OVER 50 YEARS OF AIR TECHNOLOGY EXCELLENCE

Cattani has led innovation in premium air technology for the global dental industry for over five decades. Our expertise is trusted worldwide, across more than sixty countries.

### MORE THAN EQUIPMENT – AN INVESTMENT

We back our compressors with a market leading warranty because we believe in their performance. We take the risk – so you don't have to.

### DENTAL-GRADE HYGIENIC, DRY, AND OIL-FREE AIR

Cattani compressors deliver the highest standard of pure, safe air:

- Removes **99.975% of particles down to 0.3 micron**
- Dual-coated tank lined with **food-grade resin**
- **Biocote® silver ion tank protection** for antimicrobial safety
- Certified to **ISO 8573-1 Class 2:4:2**

### THE CATTANI COMMITMENT

From your first conversation to long-term support, our local in-house team, along with our Cattani-trained Accredited Dealer Network, is with you every step – advice before purchase, technical expertise, and after-sales care for the life of your equipment.

### FUTURE-PROOF FLEXIBILITY

Whether it's a single chair, a hospital, or a university, Cattani manufactures compressors in sizes and configurations to fit every practice. We lead the way in modular design to meet the unique demands of each dental clinic, regardless of size.

### QUIET BY DESIGN

Where space is limited and noise matters, Cattani offers effective solutions – from acoustic hoods to fully enclosed, temperature-controlled cabinets.



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES




LOW NOISE  
Q RANGE

### RELIABILITY

**MAINTENANCE  
FREE PISTONS**

### PROTECTION

**DUAL-COATED TANK  
WITH ANTIMICROBIAL  
PROTECTION**

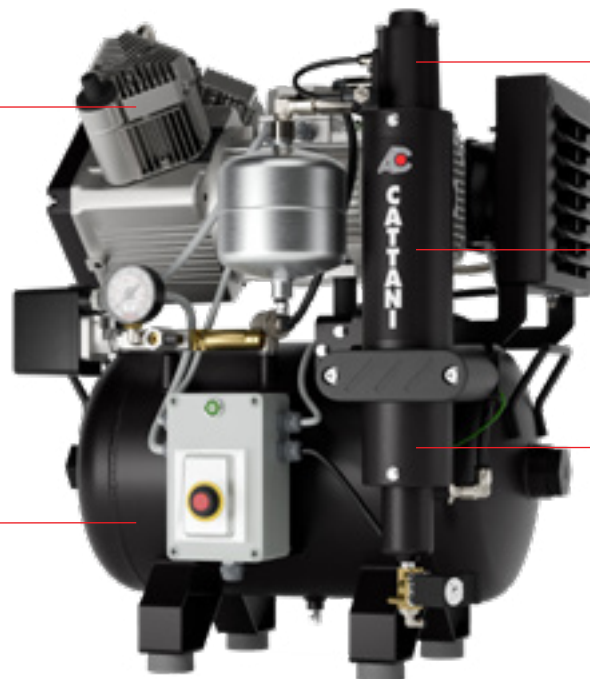

### SAFETY

**ANTIBACTERIAL  
HIGH-EFFICIENCY  
PARTICULATE FILTER  
(HEPA H14)**

### LONGEVITY

**HIGH-PERFORMANCE  
RELIABLE DRYING  
SYSTEM FOR  
LONGEVITY**

### PERFORMANCE

**6-STAGE DENTAL  
AIR PURIFICATION  
SYSTEM**



# PERFECTING DENTAL-GRADE AIR

Over 50 years of passion and precision – delivering nothing less than pure, oil-free, hygienic, and reliable air for dentistry.

## 1 IT STARTS WITH THE COMPRESSOR HEAD FILTER

Reliability begins here. The primary filter protects the head unit and pistons, removing dust and particles to 20 microns. And that's just the first step.

## 2 PIONEERING CYCLONE TECHNOLOGY

As innovators in cyclone technology, Cattani's advanced design removes condensation and water, starting the two-stage process that delivers consistently dry air.

## 3 EXTRA FILTRATION FOR SUPERIOR QUALITY

A secondary filter disc captures even finer particles as air exits the cyclone, ensuring cleaner output at every stage.

## 4 DESICCANT DRYING FOR OPTIMAL DEW POINT


Our energy-efficient desiccant drying column absorbs residual moisture, achieving **humidity class 4 compliance** with ISO 8753-1 and ISO 22052.

## 5 SINTERED BRONZE FILTER – ZERO MAINTENANCE

The sintered bronze filter protects the HEPA H14 stage and never requires replacement or cleaning.

## 6 HEPA H14 FILTRATION – 99.975% EFFICIENCY

The final stage is an antibacterial HEPA H14 filter that removes **99.975% of particles down to 0.3 microns**. Air is then preserved in a dual-coated tank lined with **food-grade resin** and **Biocote® silver ion protection**, ready for safe, dental-grade use.

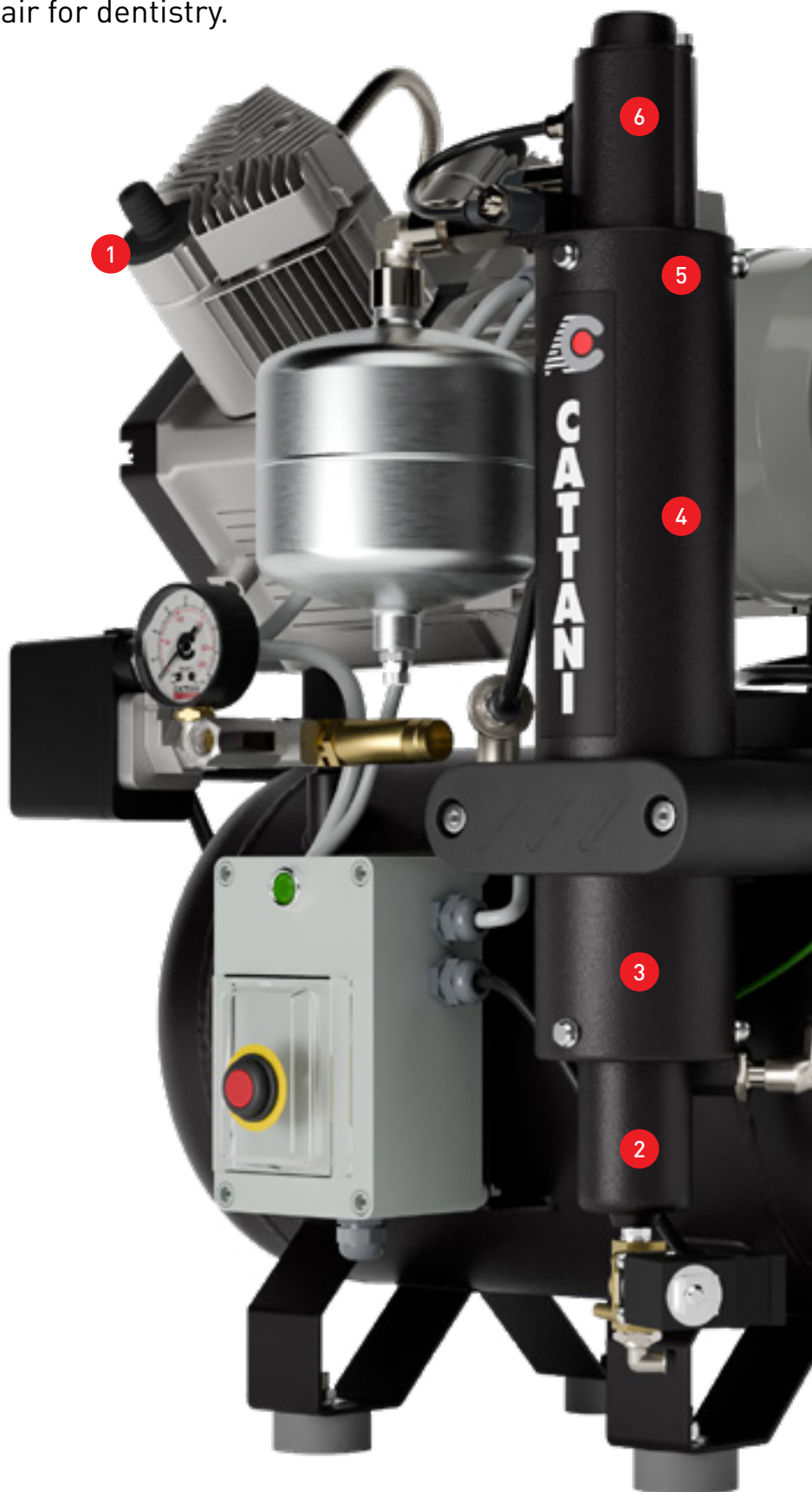

## AC100 Single Cylinder Oil-free Dental Compressor | 1 chair

The most compact of our high-performance dental compressor range, the AC100 is designed to serve 1 chair. Cattani compressors are oil-free and rigorously tested to consistently deliver clean, dry, and hygienic air.

### Details That Matter

- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Noise reduction option available (see AC100Q)
- Unrivalled technical service back-up



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

INDENT ITEM	AC100	AC100E
	Single Cylinder Oil-free Dental Compressor	Single Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013125	013125E
Price Ex GST	\$7,875.00	\$8,525.00
Price Inc GST	\$8,662.50	\$9,377.50

#### SPECIFICATIONS

Power	0.55kW 3.8A 1PH 230V	0.55kW 3.8A 1PH 230V
Sound Output dB(A)	70	70
Max Air Flow at 5 Bar NL/min	67.5	67.5
Tank Size litres	30	30

#### DIMENSIONS

Height mm	720	720
Width mm	620	620
Depth mm	460	540
Weight kg	47	47
Shipping Weight kg	65	66

### RELATED PRODUCTS



Acoustic cabinet



Compressor parallel  
connection kit



Service kit




## AC100Q Quiet Single Cylinder Oil-free Dental Compressor | 1 chair

The most compact of our high-performance dental compressor range, the AC100Q is designed to serve 1 chair. Our Q series of compressors significantly reduces noise levels, making them an excellent option when equipment is close to the surgery.

### Details That Matter

- Acoustic hood for significant noise reduction
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Unrivalled technical service back-up

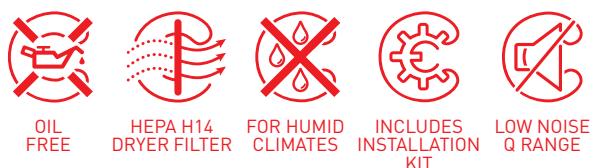


INDENT ITEM	AC100Q	AC100QE
	Quiet Single Cylinder Oil-free Dental Compressor	Quiet Single Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013144	013144E
Price Ex GST	\$10,875.00	\$11,525.00
Price Inc GST	\$11,962.50	\$12,677.50

SPECIFICATIONS		
Power	0.55kW 3.8A 1PH 230V	0.55kW 3.8A 1PH 230V
Sound Output dB(A)	63	63
Max Air Flow at 5 Bar NL/min	67.5	67.5
Tank Size litres	30	30

DIMENSIONS		
Height mm	860	860
Width mm	660	660
Depth mm	600	680
Weight kg	57	57
Shipping Weight kg	72	73

### RELATED PRODUCTS

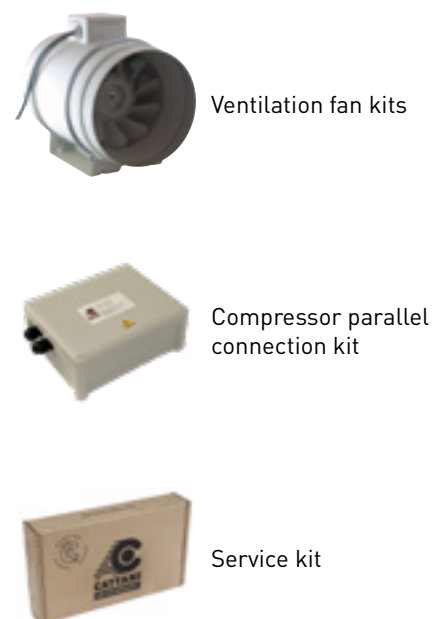

## AC200 2 Cylinder Oil-free Dental Compressor | 2-3 chairs

One of the most popular models of our dental compressor range, the AC200 is designed to serve 2 to 3 chairs. Cattani compressors are oil-free and rigorously tested to deliver precision in hygienic air.

### Details That Matter

- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Noise reduction option available (see AC200Q)
- Unrivalled technical service back-up



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

	AC200	AC200E
	2 Cylinder Oil-free Dental Compressor	2 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013208	013208E
Price Ex GST	\$8,975.00	\$9,625.00
Price Inc GST	\$9,872.50	\$10,587.50

#### SPECIFICATIONS

Power	1.25kW 7.1A 1PH 230V	1.25kW 7.1A 1PH 230V
Sound Output dB(A)	71	71
Max Air Flow at 5 Bar NL/min	160	160
Tank Size litres	30	30

#### DIMENSIONS

Height mm	720	720
Width mm	620	620
Depth mm	460	540
Weight kg	54	54
Shipping Weight kg	70	71

### RELATED PRODUCTS



Acoustic cabinet



Compressor parallel  
connection kit



Service kit



## AC200Q Quiet 2 Cylinder Oil-free Dental Compressor | 2-3 chairs

The AC200Q is designed to serve 2 to 3 chairs. Our Q series of compressors significantly reduces noise levels, making them an excellent option when equipment is close to the surgery.

### Details That Matter

- Acoustic hood for significant noise reduction
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Unrivalled technical service back-up



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT



LOW NOISE  
Q RANGE

	AC200Q	AC200QE
	Quiet 2 Cylinder Oil-free Dental Compressor	Quiet 2 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013244	013244E
Price Ex GST	\$11,975.00	\$12,625.00
Price Inc GST	\$13,172.50	\$13,887.50

#### SPECIFICATIONS

Power	1.25kW 7.1A 1PH 230V	1.25kW 7.1A 1PH 230V
Sound Output dB(A)	63	63
Max Air Flow at 5 Bar NL/min	160	160
Tank Size litres	30	30

#### DIMENSIONS

Height mm	860	860
Width mm	660	660
Depth mm	600	680
Weight kg	64	64
Shipping Weight kg	81	82

#### RELATED PRODUCTS



Ventilation fan kits



Compressor parallel  
connection kit



Service kit

## AC300 3 Cylinder Oil-free Dental Compressor | 3-4 chairs

One of our most popular models of the dental compressor range, the AC300 is designed to serve 3 to 4 chairs. Cattani compressors deliver high performance dry air and are completely oil-free.

### Details That Matter

- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Noise reduction option available (see AC300Q)
- Unrivalled technical service back-up



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

	AC300	AC300E
	3 Cylinder Oil-free Dental Compressor	3 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013308	013308E
Price Ex GST	\$10,950.00	\$11,600.00
Price Inc GST	\$12,045.00	\$12,760.00

#### SPECIFICATIONS

Power	1.5kW 9.2A 1PH 230V	1.5kW 9.2A 1PH 230V
Sound Output dB(A)	73.6	73.6
Max Air Flow at 5 Bar NL/min	238	238
Tank Size litres	45	45

#### DIMENSIONS

Height mm	750	750
Width mm	620	620
Depth mm	520	600
Weight kg	65	65
Shipping Weight kg	83	84

### RELATED PRODUCTS



Acoustic cabinet



Compressor parallel connection kit



Service kit



## AC300Q Quiet 3 Cylinder Oil-free Dental Compressor | 3-4 chairs

One of our most popular models of the quiet dental compressor range, the AC300Q is designed to serve 3 to 4 chairs. Our Q series of compressors significantly reduces noise levels, making them an excellent option when equipment is close to the surgery.

### Details That Matter

- Acoustic hood for significant noise reduction
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Unrivalled technical service back-up



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT



LOW NOISE  
Q RANGE

	AC300Q	AC300QE
	Quiet 3 Cylinder Oil-free Dental Compressor	Quiet 3 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013344	013344E
Price Ex GST	\$13,950.00	\$14,600.00
Price Inc GST	\$15,345.00	\$16,060.00

#### SPECIFICATIONS

Power	1.5kW 10.2A 1PH 230V	1.5kW 10.2A 1PH 230V
Sound Output dB(A)	68	68
Max Air Flow at 5 Bar NL/min	238	238
Tank Size litres	45	45

#### DIMENSIONS

Height mm	979	979
Width mm	720	720
Depth mm	700	780
Weight kg	80	80
Shipping Weight kg	104	105

#### RELATED PRODUCTS



Ventilation fan kits



Compressor parallel  
connection kit



Service kit

## AC400 Tandem 2 Cylinder Oil-free Dental Compressor | 5-6 chairs

With a 100-litre tank, the AC400 is designed to serve 5 to 6 chairs, the ideal setup for a growing practice. This model is designed to deliver dry, oil-free, and hygienic air operating on three-phase power (single-phase also available).

### Details That Matter

- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Noise reduction option available (see AC400Q)
- Single-phase option available (See AC400-1P)
- Unrivalled technical service back-up



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

	AC400	AC400E
	Tandem 2 Cylinder Oil-free Dental Compressor	Tandem 2 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013432	013432E
Price Ex GST	\$16,750.00	\$18,050.00
Price Inc GST	\$18,425.00	\$19,855.00

#### SPECIFICATIONS

Power	3.0kW 7.2A 3PH 400V	3.0kW 7.2A 3PH 400V
Sound Output dB(A)	73	73
Max Air Flow at 5 Bar NL/min	320	320
Tank Size litres	100	100

#### DIMENSIONS

Height mm	745	745
Width mm	1200	1200
Depth mm	550	610
Weight kg	112	112
Shipping Weight kg	134	136

### RELATED PRODUCTS



Ventilation fan kits



Three-phase  
compressor parallel  
connection kit



Service kit



## AC400-1P Tandem 2 Cylinder Oil-free Dental Compressor | 5-6 chairs

With a 100-litre tank, the AC400-1P is designed to serve 5 to 6 chairs, the ideal setup for a growing practice. This model is designed to deliver dry, oil-free, and hygienic air operating on single-phase power (three-phase also available).

### Details That Matter

- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Noise reduction option available (see AC400Q-1P)
- Three-phase option available (See AC400)



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

	AC400-1P	AC400-1PE
	Tandem 2 Cylinder Oil-free Dental Compressor	Tandem 2 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013422	013422E
Price Ex GST	\$16,750.00	\$18,050.00
Price Inc GST	\$18,425.00	\$19,855.00

#### SPECIFICATIONS

Power	2.4kW 15.4A 1PH 230V	2.4kW 15.4A 1PH 230V
Sound Output dB(A)	73	73
Max Air Flow at 5 Bar NL/min	320	320
Tank Size litres	100	100

#### DIMENSIONS

Height mm	745	745
Width mm	1200	1200
Depth mm	550	610
Weight kg	112	112
Shipping Weight kg	134	136

#### RELATED PRODUCTS



Ventilation fan kits



Compressor parallel  
connection kit




Service kit

## AC400Q Quiet Tandem 2 Cylinder Oil-free Dental Compressor | 5-6 chairs

With a 100-litre tank, the AC400Q is designed to serve 5 to 6 chairs, the ideal setup for a growing practice. This model is designed with noise reduction, operating on three-phase power (single-phase also available).

### Details That Matter

- Acoustic hood for significant noise reduction
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Single-phase option available (See AC400Q-1P)
- Unrivalled technical service back-up

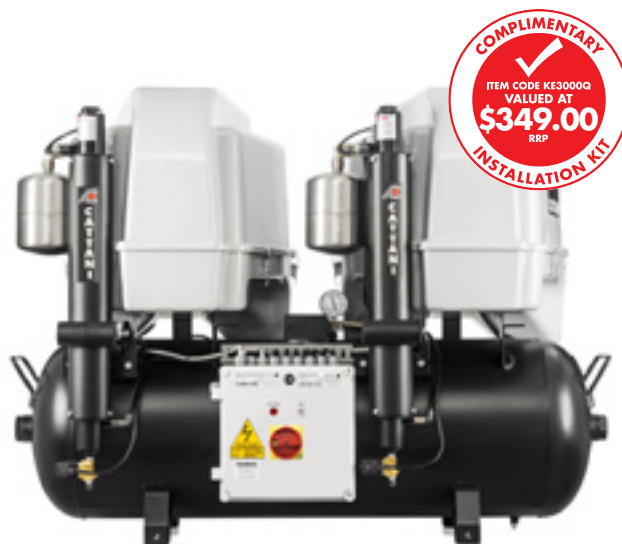

OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT



LOW NOISE  
Q RANGE

	AC400Q	AC400QE
	Quiet Tandem 2 Cylinder Oil-free Dental Compressor	Quiet Tandem 2 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013450	013450E
Price Ex GST	\$22,750.00	\$24,050.00
Price Inc GST	\$25,025.00	\$26,455.00
<b>SPECIFICATIONS</b>		
Power	3.0kW 7.4A 3PH 400V	3.0kW 7.4A 3PH 400V
Sound Output dB(A)	63.8	63.8
Max Air Flow at 5 Bar NL/min	320	320
Tank Size litres	100	100
<b>DIMENSIONS</b>		
Height mm	890	890
Width mm	1200	1200
Depth mm	650	710
Weight kg	135	135
Shipping Weight kg	167	169

### RELATED PRODUCTS



Ventilation fan kits



Three-phase  
compressor parallel  
connection kit




Service kit

## AC400Q-1P Quiet Tandem 2 Cylinder Oil-free Dental Compressor | 5-6 chairs

With a 100-litre tank, the AC400Q-1P is designed to serve 5 to 6 chairs, the ideal setup for a growing practice. This model is designed with noise reduction, operating on single-phase power (three-phase also available).

### Details That Matter

- Acoustic hood for significant noise reduction
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Three-phase option available (See AC400Q)

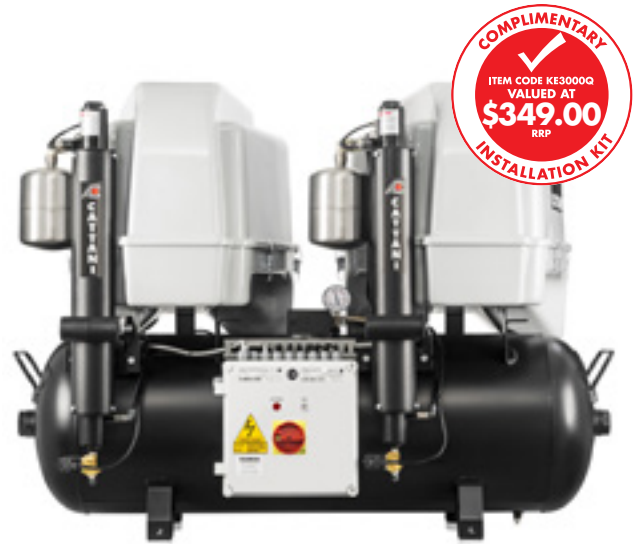

OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT



LOW NOISE  
Q RANGE

	AC400Q-1P	AC400Q-1PE
	Quiet Tandem 2 Cylinder Oil-free Dental Compressor	Quiet Tandem 2 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013444	013444E
Price Ex GST	\$22,750.00	\$24,050.00
Price Inc GST	\$25,025.00	\$26,455.00
<b>SPECIFICATIONS</b>		
Power	2.4kW 15.4A 1PH 230V	2.4kW 15.4A 1PH 230V
Sound Output dB(A)	63.8	63.8
Max Air Flow at 5 Bar NL/min	320	320
Tank Size litres	100	100
<b>DIMENSIONS</b>		
Height mm	890	890
Width mm	1200	1200
Depth mm	650	710
Weight kg	135	135
Shipping Weight kg	167	169

### RELATED PRODUCTS



Ventilation fan kits



Compressor parallel  
connection kit



Service kit



## AC600 Tandem 3 Cylinder Oil-free Dental Compressor | 7-8 chairs

Designed to deliver dry, oil-free, and hygienic air at large facilities, the AC600 serves 7 to 8 chairs. With a 150-litre tank, this oil-free compressor has been rigorously tested to deliver world-class precision in dental air technology.

### Details That Matter

- 150-litre tank capacity
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Noise reduction option available (see AC600Q)
- Unrivalled technical service back-up



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

	AC600	AC600E
	Tandem 3 Cylinder Oil-free Dental Compressor	Tandem 3 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013532	013532E
Price Ex GST	\$20,950.00	\$22,250.00
Price Inc GST	\$23,045.00	\$24,475.00
<b>SPECIFICATIONS</b>		
Power	3.0kW 7.4A 3PH 400V	3.0kW 7.4A 3PH 400V
Sound Output dB(A)	74	74
Max Air Flow at 5 Bar NL/min	476	476
Tank Size litres	150	150
<b>DIMENSIONS</b>		
Height mm	890	890
Width mm	1320	1320
Depth mm	590	630
Weight kg	124	124
Shipping Weight kg	154	156

### RELATED PRODUCTS



Ventilation fan kits



Three-phase  
compressor parallel  
connection kit




Service kit

## AC600Q Quiet Tandem 3 Cylinder Oil-free Dental Compressor | 7-8 chairs

Designed to deliver dry, oil-free, and hygienic air at large facilities, the AC600Q serves 7 to 8 chairs. This model significantly reduces noise levels, making it an excellent option when equipment is close to the surgery.

### Details That Matter

- 150-litre tank capacity
- Acoustic hood for significant noise reduction
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Unrivalled technical service back-up

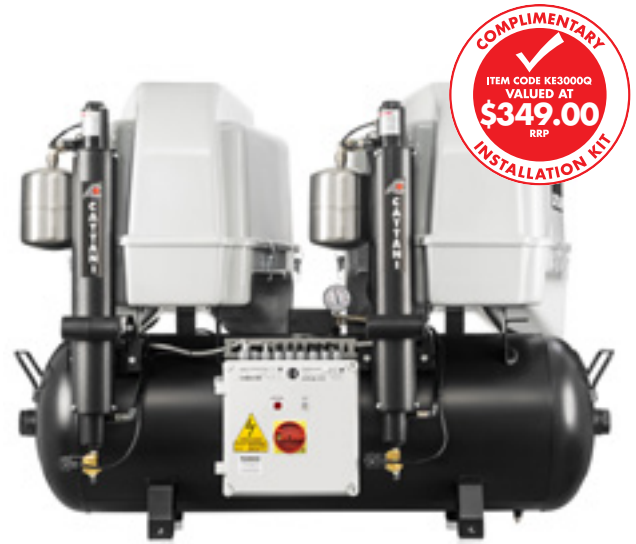

OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT



LOW NOISE  
Q RANGE

INDENT ITEM	AC600Q	AC600QE
	Quiet Tandem 3 Cylinder Oil-free Dental Compressor	Quiet Tandem 3 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013557	013557E
Price Ex GST	\$26,950.00	\$28,250.00
Price Inc GST	\$29,645.00	\$31,075.00

#### SPECIFICATIONS

Power	3.0kW 7.4A 3PH 400V	3.0kW 7.4A 3PH 400V
Sound Output dB(A)	70.4	70.4
Max Air Flow at 5 Bar NL/min	476	476
Tank Size litres	150	150

#### DIMENSIONS

Height mm	1040	1040
Width mm	1320	1320
Depth mm	770	770
Weight kg	169	169
Shipping Weight kg	205	207

#### RELATED PRODUCTS



Ventilation fan kits



Three-phase  
compressor parallel  
connection kit



Service kit

### BIG IN PERFORMANCE, COMPACT IN SIZE

Specialising in delivering dental grade air to large-scale facilities including hospitals and universities, we don't just understand large installations, we are a leader in this space. As compressed air systems are imperative for critical end-uses in large facilities, we provide the best compressor systems to go with our modular installation suction systems.

All compressor systems come in dual coated tanks lined with a food grade resin and Biocote® silver ion for antimicrobial protection.

#### **AC900 SYSTEM**

- Up to 12 chairs
- Triple 3 Cylinder Oil-Free Dental Compressor
- 300-litre storage tank

#### **AC1200 SYSTEM**

- Up to 16 chairs
- Tandem 6 Cylinder Oil-Free Dental Compressor
- 300-litre storage tank

#### **AC1800 SYSTEM**

- Up to 24 chairs
- Triple 6 Cylinder Oil-Free Dental Compressor
- 300-litre storage tank

#### **AC2400 SYSTEM**

- Up to 32 chairs
- Quad 6 Cylinder Oil-Free Dental Compressor
- 200-litre storage tank

### AT CATTANI, WE ARE THE FORCE BEHIND YOUR OPERATION.





## AC900 Triple 3 Cylinder Oil-free Dental Compressor | Up to 12 chairs

Designed to deliver dry, oil-free, and hygienic air at large facilities, the AC900 serves up to 12 chairs. With a 300-litre tank, this oil-free compressor has been rigorously tested and ready to deliver world-class performance in dental air technology.

### Details That Matter

- 300-litre tank capacity
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Unrivalled technical service back-up



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

INDENT ITEM	AC900	AC900E
	Triple 3 Cylinder Oil-free Dental Compressor	Triple 3 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013599	013599E
Price Ex GST	\$35,150.00	\$37,100.00
Price Inc GST	\$38,665.00	\$40,810.00

### RELATED PRODUCTS



Ventilation fan kits



Three-phase  
compressor parallel  
connection kit



Service kit

SPECIFICATIONS		
Power	4.5kW 11.1A 3PH 400V	4.5kW 11.1A 3PH 400V
Sound Output dB(A)	75	75
Max Air Flow at 5 Bar NL/min	714	714
Tank Size litres	300	300
DIMENSIONS		
Height mm	1000	1000
Width mm	1800	1800
Depth mm	810	895
Weight kg	260	260
Shipping Weight kg	304	311

## AC1200 Tandem 6 Cylinder Oil-free Dental Compressor | Up to 16 chairs

Designed to deliver dry, oil-free, and hygienic air at large facilities, the AC1200 serves up to 16 chairs. With a 300-litre tank, this high-performance unit has been rigorously tested to deliver continuous and reliable precision in air technology.

### Details That Matter

- 300-litre tank capacity
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Unrivalled technical service back-up



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

INDENT ITEM	AC1200	AC1200E
	Tandem 6 Cylinder Oil-free Dental Compressor	Tandem 6 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013595	013595E
Price Ex GST	\$44,500.00	\$47,100.00
Price Inc GST	\$48,950.00	\$51,810.00

#### SPECIFICATIONS

Power	6.5kW 15A 3PH 400V	6.5kW 15A 3PH 400V
Sound Output dB(A)	81	81
Max Air Flow at 5 Bar NL/min	952	952
Tank Size litres	300	300

#### DIMENSIONS

Height mm	1100	1100
Width mm	1800	1800
Depth mm	880	960
Weight kg	251	254
Shipping Weight kg	302	305

#### RELATED PRODUCTS



Ventilation fan kits



Three-phase  
compressor parallel  
connection kit



Service kit

## AC1800 Triple 6 Cylinder Oil-free Dental Compressor | Up to 24 chairs

Manufactured with patented low-friction components to prolong the lifespan of the unit, the AC1800 serves up to 24 chairs. With a 300-litre tank, this high-performance unit is ready to deliver world-class precision at large facilities.

### Details That Matter

- 300-litre tank capacity
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Unrivalled technical service back-up



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

INDENT ITEM	AC1800	AC1800E
	Triple 6 Cylinder Oil-free Dental Compressor	Triple 6 Cylinder Oil-free Dental Compressor with Pre-filter
Item Code	013594	013594E
Price Ex GST	\$68,150.00	\$72,050.00
Price Inc GST	\$74,965.00	\$79,255.00

SPECIFICATIONS		
Power	9.75kW 22.6A 3PH 400V	9.75kW 22.6A 3PH 400V
Sound Output dB(A)	82.5	82.5
Max Air Flow at 5 Bar NL/min	1428	1428
Tank Size litres	300	300

DIMENSIONS		
Height mm	1100	1100
Width mm	1800	1800
Depth mm	1100	1100
Weight kg	363	368
Shipping Weight kg	418	423

### RELATED PRODUCTS



Ventilation fan kits



Three-phase  
compressor parallel  
connection kit



Service kit



## AC2400 Quad 6 Cylinder Oil-free Dental Compressor | Up to 32 chairs

The largest of our AC Cattani compressor range, the AC2400 serves up to 32 chairs. Manufactured with patented low-friction components for continuous and reliable operation, this high-performance unit is ready to deliver world-class precision at large facilities.

### Details That Matter

- 200-litre tank capacity
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Unrivalled technical service back-up



OIL  
FREE



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT



#### INDENT ITEM

#### AC2400

Quad 6 Cylinder  
Oil-free  
Dental Compressor

Item Code 010656

Price Ex GST Price on application

Price Inc GST Price on application

#### SPECIFICATIONS

Power 13kW 31A

3PH 400V

Sound Output dB(A) 83.5

Max Air Flow at 5 Bar NL/min 1680

Tank Size litres 200

#### DIMENSIONS

Height mm 2000

Width mm 1550

Depth mm 760

Weight kg 300

Shipping Weight kg Upon Request

#### RELATED PRODUCTS



Ventilation fan kits



Three-phase  
compressor parallel  
connection kit



Service kit

# THE PURPOSE-BUILT ADVANTAGE

Making the correct choice for your plant equipment early on is crucial for your practice and/or dental laboratory in the long run. Cattani users know we specialise in unequaled precision and quality of purpose-built solutions. This is especially true when it comes to CAD/CAM milling.

Cattani CAD/CAM compressors are the purpose-built solution developed specifically to meet the demands set by CAD/CAM mill manufacturers.



OIL  
FREE



FOR HUMID  
CLIMATES



LOW NOISE  
Q RANGE

### THE OPTIMAL PURPOSE-BUILT MILL PERFORMANCE

Generalised solutions can be appealing in the short-term. In the case of dental grade air delivery, it is best to leave nothing to chance and have the right equipment fit for purpose. Cattani CAD/CAM compressors are built to maintain a constant and sustainable operating pressure of between 8 and 10 Bar, which is 2 Bar higher in pressure than the standard compressors.

### DENTAL GRADE HYGIENE

Cattani compressors produce quality dental grade air by removing 99.975% of particles down to 0.3 micron. Air is then preserved in a dual coated compressor tank lined with a food grade resin and Biocote® silver ion for antimicrobial protection. Cattani compressed air quality is supported by an ISO 8573-1 standard compliance with Class 1:4:2.

### GUARANTEED CONTINUOUS AIRFLOW

Cattani compressors guarantee a constant supply of air from the dual coated tank, even during system regeneration and cooling times. The intelligent technology is designed this way to ensure maximum performance while preserving the life of the unit.

### NOISE REDUCTION OPTIONS AVAILABLE

Where space is valuable and limited, noise can be an issue. For that, we offer noise reduction options for the (Q Series) CAD/CAM compressors.



## AC310 3 Cylinder CAD/CAM Compressor

One of the most popular model in our CAD/CAM compressor range, the AC310 comes with a 45-litre tank capacity. Cattani compressors are completely oil-free, offered specifically to meet the demands set by CAD/CAM manufacturers.

### Details That Matter

- Purpose-built to maintain an 8-bar pressure over long CAM cycles
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Noise reduction option available (see AC310Q)
- Pre-filter for humid climates available upon request



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

AC310	
3 Cylinder CAD/CAM Compressor	
Item Code	013310
Price Ex GST	\$12,275.00
Price Inc GST	\$13,502.50

#### SPECIFICATIONS

Power	1.5kW 10.2A 1PH 230V
Sound Output dB(A)	73.6
<b>Continuous Airflow at 8 Bar NL/min</b>	<b>108</b>
Max Air Flow at 8 Bar NL/min	165
Tank Size litres	45

#### DIMENSIONS

Height mm	750
Width mm	620
Depth mm	520
Weight kg	68
Shipping Weight kg	83

### RELATED PRODUCTS



Ventilation fan kits



Service kit



## AC310Q Quiet 3 Cylinder CAD/CAM Compressor

The AC310Q comes with a 45-litre tank capacity. AC310Q comes with a 45-litre tank capacity, and is ideal when noise is an issue, making them an excellent option when milling equipment is close to the surgery.

### Details That Matter

- Acoustic hood for significant noise reduction
- Purpose-built to maintain an 8-bar pressure over long CAM cycles
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Pre-filter for humid climates available upon request



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT



LOW NOISE  
Q RANGE

### INDENT ITEM AC310Q

Quiet 3 Cylinder  
CAD/CAM Compressor

Item Code	013355
Price Ex GST	\$15,275.00
Price Inc GST	\$16,802.50

### SPECIFICATIONS

Power	1.5kW 10.2A 1PH 230V
Sound Output dB(A)	68
<b>Continuous Airflow at 8 Bar NL/min</b>	<b>108</b>
Max Air Flow at 8 Bar NL/min	165
Tank Size litres	45

### DIMENSIONS

Height mm	979
Width mm	720
Depth mm	700
Weight kg	84
Shipping Weight kg	104

### RELATED PRODUCTS



Ventilation fan kits



Service kit

## AC410 Tandem 2 Cylinder CAD/CAM Compressor

Specifically built for CAD/CAM milling, the AC410 compressor comes with a 100-litre tank capacity. This model is designed to deliver dry, oil-free, and hygienic air operating on three-phase power (single-phase also available).

### Details That Matter

- Purpose-built to maintain an 8-bar pressure over long CAM cycles
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Noise reduction option available (see AC410Q)
- Single-phase option available (see AC410-1P)
- Pre-filter for humid climates available upon request



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

INDENT ITEM	AC410
	Tandem 2 Cylinder CAD/CAM Compressor
Item Code	013413
Price Ex GST	\$18,075.00
Price Inc GST	\$19,882.50
<b>SPECIFICATIONS</b>	
Power	3kW 7.2A 3PH 400V
Sound Output dB(A)	73
<b>Continuous Airflow at 8 Bar NL/min</b>	<b>140</b>
Max Air Flow at 8 Bar NL/min	215
Tank Size litres	100
<b>DIMENSIONS</b>	
Height mm	745
Width mm	1155
Depth mm	550
Weight kg	112
Shipping Weight kg	134

### RELATED PRODUCTS



Ventilation fan kits



Service kit

## AC410-1P Tandem 2 Cylinder CAD/CAM Compressor

Specifically built for CAD/CAM milling, the AC410-1P compressor comes with a 100-litre tank capacity. This model is designed to deliver dry, oil-free, and hygienic air operating on single-phase power (three-phase also available).

### Details That Matter

- Purpose-built to maintain an 8-bar pressure over long CAM cycles
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank.
- Noise reduction option available (see AC410Q-1P)
- Three-phase option available (see AC410)
- Pre-filter for humid climates available upon request



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

INDENT ITEM	AC410-1P
	Tandem 2 Cylinder CAD/CAM Compressor
Item Code	013410
Price Ex GST	\$18,075.00
Price Inc GST	\$19,882.50
<b>SPECIFICATIONS</b>	
Power	2.4kW 15.4A 1PH 230V
Sound Output dB(A)	73
<b>Continuous Airflow at 8 Bar NL/min</b>	<b>140</b>
Max Air Flow at 8 Bar NL/min	215
Tank Size litres	100
<b>DIMENSIONS</b>	
Height mm	745
Width mm	1155
Depth mm	550
Weight kg	112
Shipping Weight kg	134

### RELATED PRODUCTS



Ventilation fan kits



Service kit

## AC410Q Quiet Tandem 2 Cylinder CAD/CAM Compressor

Specifically built for CAD/CAM milling, the AC410Q compressor comes with a 100-litre tank capacity. This model is designed for premium air delivery with noise reduction, operating on three-phase power (single-phase also available).

### Details That Matter

- Purpose-built to maintain an 8-bar pressure over long CAM cycles
- Acoustic hood for significant noise reduction
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Single-phase option available (See AC410Q-1P)
- Pre-filter for humid climates available upon request



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT



LOW NOISE  
Q RANGE

INDENT ITEM	AC410Q
	Quiet Tandem 2 Cylinder CAD/CAM Compressor
Item Code	013458
Price Ex GST	\$24,075.00
Price Inc GST	\$26,482.50

SPECIFICATIONS	
Power	3kW 7.4A 3PH 400V
Sound Output dB(A)	63.8
<b>Continuous Airflow at 8 Bar NL/min</b>	<b>140</b>
Max Air Flow at 8 Bar NL/min	215
Tank Size litres	100
DIMENSIONS	
Height mm	890
Width mm	1160
Depth mm	650
Weight kg	153
Shipping Weight kg	167

### RELATED PRODUCTS



Ventilation fan kits



Service kit



## AC410Q-1P Quiet Tandem 2 Cylinder CAD/CAM Compressor

Specifically built for CAD/CAM milling, the AC410Q-1P compressor comes with a 100-litre tank capacity. This model is designed for premium air delivery with noise reduction, operating on single-phase power (three-phase also available).

### Details That Matter

- Purpose-built to maintain an 8-bar pressure over long CAM cycles
- Acoustic hood for significant noise reduction
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Three-phase option available (see AC410Q)
- Pre-filter for humid climates available upon request



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT



LOW NOISE  
Q RANGE

INDENT ITEM	AC410Q-1P
	Quiet Tandem 2 Cylinder CAD/CAM Compressor
Item Code	013455
Price Ex GST	\$24,075.00
Price Inc GST	\$26,482.50
<b>SPECIFICATIONS</b>	
Power	2.4kW 15.4A 1PH 230V
Sound Output dB(A)	63.8
<b>Continuous Airflow at 8 Bar NL/min</b>	<b>140</b>
Max Air Flow at 8 Bar NL/min	215
Tank Size litres	100
<b>DIMENSIONS</b>	
Height mm	890
Width mm	1160
Depth mm	650
Weight kg	153
Shipping Weight kg	167

### RELATED PRODUCTS



Ventilation fan kits



Service kit

## AC610 Tandem 3 Cylinder CAD/CAM Compressor

The AC610 compressor is designed to deliver premium quality for higher performance requirements. With a 150-litre tank capacity, this oil-free compressor has been rigorously tested to deliver world-class precision for CAD/CAM milling.

### Details That Matter

- Purpose-built to maintain an 8-bar pressure over long CAM cycles
- 150-litre tank capacity
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Noise reduction option available (see AC610Q)
- Pre-filter for humid climates available upon request



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

INDENT ITEM	AC610
	Tandem 3 Cylinder CAD/CAM Compressor
Item Code	013513
Price Ex GST	\$22,275.00
Price Inc GST	\$24,502.50
<b>SPECIFICATIONS</b>	
Power	3kW 7.4A 3PH 400V
Sound Output dB(A)	74
<b>Continuous Airflow at 8 Bar NL/min</b>	<b>217</b>
Max Air Flow at 8 Bar NL/min	330
Tank Size litres	150
<b>DIMENSIONS</b>	
Height mm	890
Width mm	1320
Depth mm	590
Weight kg	124
Shipping Weight kg	154

### RELATED PRODUCTS



Ventilation fan kits



Service kit

## AC610Q Quiet Tandem 3 Cylinder CAD/CAM Compressor

The AC610Q compressor comes with a 150-litre tank capacity for higher performance requirements. Our Q series of compressors significantly reduces noise levels, making them an excellent option when CAD/CAM equipment is close to the surgery.

### Details That Matter

- Purpose-built to maintain an 8-bar pressure over long CAM cycles
- Acoustic hood for significant noise reduction
- 150-litre tank capacity
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank
- Pre-filter for humid climates available upon request



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT



LOW NOISE  
Q RANGE

INDENT ITEM	AC610Q
	Quiet Tandem 3 Cylinder CAD/CAM Compressor
Item Code	013558
Price Ex GST	\$28,275.00
Price Inc GST	\$31,102.50
<b>SPECIFICATIONS</b>	
Power	3kW 7.4A 3PH 400V
Sound Output dB(A)	70.4
<b>Continuous Airflow at 8 Bar NL/min</b>	<b>217</b>
Max Air Flow at 8 Bar NL/min	330
Tank Size litres	150
<b>DIMENSIONS</b>	
Height mm	1040
Width mm	1320
Depth mm	770
Weight kg	174
Shipping Weight kg	205

### RELATED PRODUCTS



Ventilation fan kits



Service kit

## AC910 Triple 3 Cylinder CAD/CAM Compressor

Our largest of the CAD/CAM compressor range, the AC910 comes with a 300-litre tank capacity. All our compressors are oil-free and have been rigorously tested to deliver world class performance in dental air technology.

### Details That Matter

- Purpose-built to maintain an 8-bar pressure over long CAM cycles
- 300-litre tank capacity
- Certified food grade resin coated tank
- Added antimicrobial protection with Biocote® silver-ion lining in tank.
- Pre-filter for humid climates available upon request



OIL  
FREE



HEPA H14  
DRYER FILTER



FOR HUMID  
CLIMATES



INCLUDES  
INSTALLATION  
KIT

INDENT ITEM	AC910
	Triple 3 Cylinder CAD/CAM Compressor
Item Code	013578
Price Ex GST	\$36,475.00
Price Inc GST	\$40,122.50
<b>SPECIFICATIONS</b>	
Power	4.5kW 11.1A 3PH 400V
Sound Output dB(A)	75
<b>Continuous Airflow at 8 Bar NL/min</b>	<b>326</b>
Max Air Flow at 8 Bar NL/min	495
Tank Size litres	300
<b>DIMENSIONS</b>	
Height mm	1000
Width mm	1800
Depth mm	810
Weight kg	258
Shipping Weight kg	304

### RELATED PRODUCTS



Ventilation fan kits



Service kit



# COMPRESSOR ACCESSORIES

## Acoustic Cabinet for AC100, AC200 and AC300

The Cattani acoustic cabinet is a complete weatherproof, temperature controlled and acoustic solution. Designed for both outdoors or indoors installation. The cabinet is made using electro-galvanised metal plate and epoxy-polyester powders that can withstand rain and direct sunlight in hot and cold climates. The cabinet includes an inbuilt fan to effectively ventilate the compressor.

Reduces noise of the compressor to:

AC100 - 50 dB(A)

AC200 - 51 dB(A)

AC300 - 51 dB(A)



Order Number	Description	Price Inc GST
K-010800	Acoustic Cabinet 870mm W x 887mm H x 700mm D (84kg)	\$5,720.00

## Single Phase Compressor Parallel Connection Kit

Designed to allow like for like compressors to run in parallel allowing for greater dental chair capacity.

D-RK2000 - Suitable to run 2 x AC100, 2 x AC200 or 2 x AC300

D-RK3000 - Suitable to run 2 x AC300Q



Order Number	Price Inc GST
D-RK2000	\$2,613.60
D-RK3000	\$2,613.60

## Three Phase Compressor Parallel Connection Kit

Designed to allow up to 4 of the same three phase type of compressors to run in parallel allowing for greater dental chair capacity.

Order Number	Description	Price Inc GST
K-070904	Suitable for 3PH compressor models	\$4,449.50

**\*Please contact your Business Development Manager for recommendations on the appropriate air regulator for multiple compressor installations.**



## COMPRESSOR ACCESSORIES

### HEPA H14 Filter for Compressor Drying Column

HEPA H14 filter and o-ring pack for drying column.

Order Number	Pack Qty	Price Inc GST
K168429-1	1	\$152.08



### Air Inlet Filter for Compressor Head Unit

Order Number	Pack Qty	Price Inc GST
K-168563	1	\$94.44



### Dryer Pre Filter (for Humid Climates) Kit for Compressor

Fitted between the condenser and the desiccant dryer, and removes a percentage of condensation from the air before reaching the dryer. Can be easily retrofitted to existing compressors. (1 filter required per desiccant dryer, therefore 2 filters required for AC400 and AC600 etc).

Order Number	Pack Qty	Price Inc GST
K-E10000	1	\$801.90



### Ventilation Fan Kits

Recommendations – 150 m<sup>3</sup>/hr for each kW of equipment installed.

Order Number	Description	Price Inc GST
L-TD2500-KIT	250m <sup>3</sup> /hr kit includes fan, ducting and vent cover	\$1,111.00
L-TD5000-150	500m <sup>3</sup> /hr kit includes fan, ducting and vent cover	\$1,663.20
L-TD8000	800m <sup>3</sup> /hr kit includes fan only	\$1,900.80



## COMPRESSOR SERVICE KITS & SPARE PARTS

### Minor Service Kits (HEPA H14 Filter for Drying Column)

Includes inlet filters for head unit, HEPA H14 filter for the drying column and a replacement o-ring.

Order Number	Description	Pack Qty	Price Inc GST
K-SK1013B	1 Cylinder Head Unit with HEPA H14 Filter – Minor Service Kit	1	\$270.60
K-SK2013B	2 Cylinder Head Unit with HEPA H14 Filter – Minor Service Kit	1	\$366.30
K-SK3013B	3 Cylinder Head Unit with HEPA H14 Filter – Minor Service Kit	1	\$457.60
K-SK010611B	6 Cylinder Head Unit with HEPA H14 Filter – Minor Service Kit	1	\$735.90



### Intermediate Service Kits (HEPA H14 Filter for Drying Column) Single Phase Systems Only

Includes inlet filters for head unit, HEPA H14 filter for the drying column, a replacement o-ring and suitable motor start capacitor.

Order Number	Description	Pack Qty	Price Inc GST
K-SK101B	1 Cylinder Head Unit with HEPA H14 Filter - Intermediate Service Kit	1	\$414.70
K-SK201B	2 Cylinder Head Unit with HEPA H14 Filter - Intermediate Service Kit	1	\$629.20
K-SK301B	3 Cylinder Head Unit with HEPA H14 Filter - Intermediate Service Kit	1	\$665.50



### Compressor Airline Installation Kit

Order Number	Description	Pack Qty	Price Inc GST
K-E20000	Compressor Airline Connection Kit with Reg and Gauge (AC100-AC300)	1	\$219.00
K-E30000	Compressor Airline Connection Kit with Reg and Gauge (AC400-AC600)	1	\$349.00
K-E3000Q	Compressor Airline Connection Kit with Reg and Gauge (AC400Q-AC600Q)	1	\$349.00
K-E90000	Compressor Airline Connection Kit with Reg and Gauge (AC900)	1	\$349.00

Note: These kits are included with compressor purchase.

### Capacitors

Order Number	Description	Pack Qty	Price Inc GST
C-189146	20 µF Capacitor C87.0 "UL" Metal With 150mm Cable AC100 and AC100Q	1	\$139.70
C-189147	40 µF Capacitor C87.0 "UL" Metal With 350mm Cable AC200 and AC200Q	1	\$221.65
C-189148	55 µF Capacitor C87.0 "UL" Metal With 150mm Cable AC300 and AC300Q	1	\$277.48

### OPTIMISE AND SAVE

Rather than being a follower of the past, we choose to be a pioneer of the future. At Cattani, we decided to be one of the first in the dental industry to use variable frequency inverter technology.

Traditional suction motors have a fixed speed that makes them notoriously inefficient. For example, when only a small number of chairs are in use, the motor's energy consumption remains constantly high. But Cattani SMART suction motors can adjust performance accordingly, as it always knows what's going on in the surgeries, thanks to inverter technology.



#### SMART AND ECONOMICAL

Pioneered by Cattani, the inverter control intelligence is designed to optimise performance suction, while reducing energy costs. From the number of chairs in use, down to size of suction tip, the inverter control senses the real-time requirements of your practice, to adjust the unit speed and output accordingly. No more, no less.

#### SMART SELF-PROTECTION MODE

In the event of a functional anomaly that could damage the machine, such as overload of fluids, excessive temperatures or even power surge, the SMART suction unit automatically switches to self-protection mode to overcome the issue, returning to normal operation once the situation is stabilised.

#### NOISE REDUCTION BY HALF

The SMART Cube suction range is a perfect example of what can be achieved by Cattani innovation. The Cube's acoustic housing has the ability to cut perceived noise by half, not to mention the lower running costs through its clever design and inverter technology.

#### PLUG AND PLAY POSSIBILITIES

As a market leader in modular installations, our "plug and play" parallel installation allows SMART suction systems to communicate with each other to deliver precise performance for your practice. From the single chair up to a hospital and university set-up, we have configurations to suit all needs, while ensuring built-in redundancy to ensure continuous operation.

#### AMALGAM RETENTION

As a world leader in amalgam separation, Cattani's Hydrocyclone technology is at the forefront of amalgam retention technology. The Hydrocyclone has a retention rate of more than 98%, with no extra power consumption or noise, and can be easily retrofitted by a technician.

## AT CATTANI, WE DON'T COMPROMISE. WE OPTIMISE.





# SMART CUBE: HALF AS LOUD

## ENVIRONMENTAL SUSTAINABILITY

The introduction of our SMART suction systems in 2015 has shown a significant reduction in raw materials used compared to our Classic suction range. The vast improvement from this evolution has been evident in our Turbo SMART Cube system, where 13 kg (59%) of raw materials have been spared in each unit.

- Acoustic housing cuts perceived noise by 50%
- Inverter control slashes energy consumption
- Active self-protection mode during emergencies
- Sustainable production

## PEACE AND QUIET, PEACE OF MIND

All plant room equipment generates fluctuating noise which is inevitable at any dental practice. This led Cattani engineers to develop an innovative solution that not only reduces noise, but also with lower operating costs. Cattani's SMART Cube suction range is the prime example of what can be achieved with noise reduction and other efficiencies.

## THE SCIENCE WITHIN THE HOUSING

Cattani's engineers have been able to cut perceived noise by 50% with the Cube's acoustic housing. The Cube's compact design is just as powerful as its SMART counterparts, but uses a smaller motor, translating into lower running costs. With inverter technology as part of the package, energy savings are a given with SMART Cube.

## SMART SELF-PROTECTION MODE

In the event of a functional anomaly that could damage the machine, such as overload of fluids, excessive temperatures or even power surge, the SMART suction unit automatically switches to self-protection mode to overcome the issue, returning to normal operation once the situation is stabilised.



## SMART CUBE PROMISES NOT ONLY PEACE AND QUIET, BUT ALSO PEACE OF MIND.

# INVERTER TECHNOLOGY

## THE TRUE MEANING OF PEAK PRECISION PERFORMANCE

Traditionally, conventional motor systems can lose over half their input energy before delivering end-use service. Therefore, whilst performance of the individual motor may be high, the efficiency of the whole system could improve substantially. This is where Cattani's inverter technology helps overcome this energy efficiency problem.

Cattani pioneered the use of inverter technology in dental suction equipment, which use a variable frequency drive (VFD) or variable speed drive (VSD) to control the speed of the motor for the purpose of energy saving and precision in performance control.

## BENEFITS OF INVERTER CONTROL:

- Power saving efficiency
- Independent variable speed control and speed transition control of the centrifugal separator and suction motor
- System management from multiple inputs (voltage monitoring, current monitoring, temperature monitoring for machine self-protection and fluid flow management).

## ULTIMATE CONTROL

The inverter technology controls the rotational speed of the motor by controlling its frequency and voltage supply. As the performance demand changes, the inverter control can simply increase or decrease the motor speed to meet the suction requirement. Each motor, (centrifuge and suction) are intelligently controlled by an independent inverter module. The result is constant, consistent, and reliable suction for a great patient and user experience.

## WHY SMART MATTERS

SMART suction inverter technology uses sensors to measure this demand for suction and adjusts the motor's speed to deliver precision performance. From the number of chairs in use, right down to size of suction tip and volume of fluids being aspirated, the inverter control identifies real-time requirements of your practice, monitoring its performance, as well as detecting and reporting abnormalities.

## PLUG AND PLAY, FOR CONTINUOUS OPERATION

Thanks to inverter technology, Cattani leads the way in modular installation of parallel units. Our "plug and play" parallel installation allows SMART suction systems to communicate with each other to deliver peak precision performance for your facility. From the single chair up to a hospital and university set-up, we have configurations to suit all needs, while ensuring built-in redundancy to ensure continuous operation.

## AT CATTANI, WE HAVE MASTERED PRECISION.



# AMALGAM RETENTION: THE CATTANI HYDROCYCLONE

Cattani is passionate about protecting the environment for future generations and has been at the forefront of addressing amalgam retention and separation.

The Hydrocyclone is a clever and effective concept that uses the fluid pressure created by the suction system to separate and retain amalgam, all without the need for moving parts. Fluids enter the Hydrocyclone cone and are forced downward in a spiral motion.

When the fluids hit the base of the Hydrocyclone they are forced to make a rapid inversion, while the heavier amalgam particles fall into a collection canister. If fluid velocity is not sufficient to achieve maximum retention, the system is intelligent enough to force the fluids back into the machine to recirculate until retention is achieved.

The retention rate of 98% or more exceeds with ISO 11143.

## **NO ADDITIONAL POWER CONSUMPTION, NO NOISE, NO EXTRA SPACE**

The beauty of the Hydrocyclone unit is that it uses no additional power at all, creates no noise, and operates within the same footprint of the SMART suction system. This is a revolutionary step towards a more sustainable world.

## **EASY INSTALLATION WITH RETROFIT OPTION**

Cattani SMART suction units can easily be retrofitted with Hydrocyclone units at any stage. As the Hydrocyclone fits within the same footprint of the existing SMART suction system no additional space requirements are needed.

The process is carried out on-site by a trained technician and takes less than an hour, to ensure minimal disruption to operation.

## **COLLECTION AND REPLACEMENT**

Efficient collection and replacement of canisters is also very simple. Cattani has partnered with specialist recycling company ECOCYCLE.


Contact ECOCYCLE directly to arrange your canister collection and recycling:



**AU: 1300 32 62 92**  
**NZ: 0800 732 922**

- Exceeds ISO 11143 Amalgam retention standard
- Fits within footprint of suction system
- No additional power required
- No additional noise

## **AT CATTANI, WE ARE ABOUT SUSTAINABILITY.**

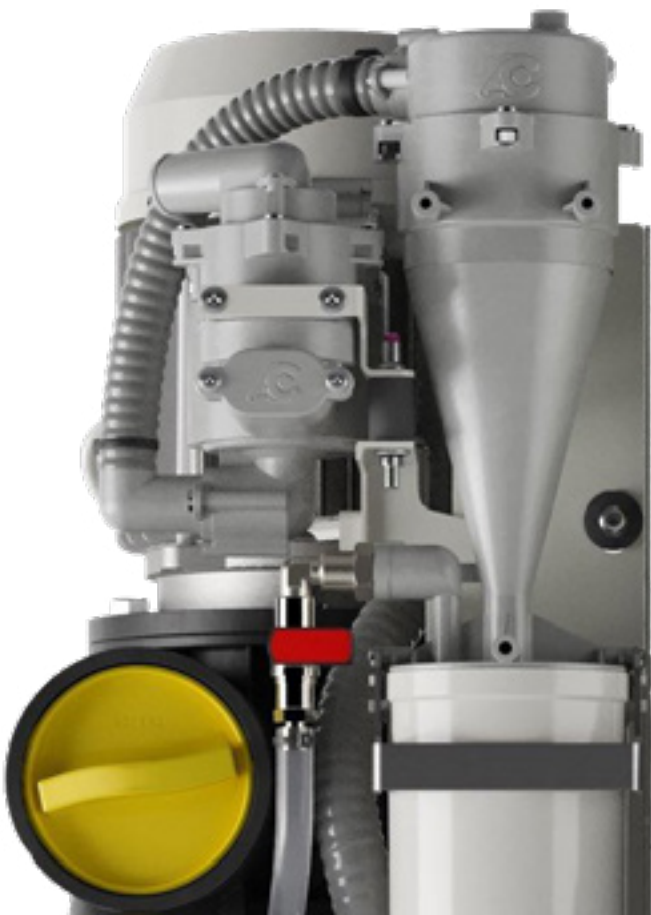



## Micro SMART with Keypad | 1-2 chairs

The most compact of our high-performance SMART suction range, the Micro SMART with Keypad is designed to serve 1 to 2 chairs. Developed with inverter control to provide consistent, high-performance suction while reducing energy costs.

### Details That Matter

- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities
- Noise reduction option available (see SMART Cube)
- Amalgam retention option available



INVERTER  
CONTROL



INCLUDES  
INSTALLATION  
KIT

Micro SMART with Keypad	
Item Code	035050TP
Price Ex GST	\$7,975.00
Price Inc GST	\$8,772.50
<b>SPECIFICATIONS</b>	
Power	1.12kW 8A 1PH 230V
Sound Output dB(A)	64
Max Air Flow at l/min	900
Max Vacuum for Continuous Operation mBar	210
<b>DIMENSIONS</b>	
Height mm	600
Width mm	380
Depth mm	350
Weight kg	31
Shipping Weight kg	32

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Micro SMART with ISO 6 Hydrocyclone | 1-2 chairs

The most compact of our high-performance SMART suction range, the Micro SMART is designed to serve 1 to 2 chairs. Integrated with ISO 6 Hydrocyclone amalgam retention.

### Details That Matter

- ISO 6 Hydrocyclone amalgam retention
- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities
- Noise reduction option available (see SMART Cube)



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT

#### Micro SMART with ISO 6 Hydrocyclone

Item Code	035060
Price Ex GST	\$8,600.00
Price Inc GST	\$9,460.00

#### SPECIFICATIONS

Power	1.12kW 8A 1PH 230V
Sound Output dB(A)	64
Max Air Flow at l/min	900
Max Vacuum for Continuous Operation mBar	210

#### DIMENSIONS

Height mm	600
Width mm	380
Depth mm	350
Weight kg	33
Shipping Weight kg	34

#### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Micro SMART Cube | 1-2 chairs

The most compact of our high-performance SMART suction Cube range, the Micro SMART Cube is designed to serve 1 to 2 chairs. The Cube's acoustic housing cuts perceived noise by 50%.

### Details That Matter

- Acoustic housing reducing perceived noise by 50%
- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities
- Amalgam retention option available



INVERTER  
CONTROL



INCLUDES  
INSTALLATION  
KIT



LOW  
NOISE

Micro SMART Cube	
Item Code	035031TP
Price Ex GST	\$8,825.00
Price Inc GST	\$9,707.50
<b>SPECIFICATIONS</b>	
Power	0.95KW 5.8A 1PH 230V
Sound Output dB(A)	59
Max Air Flow at l/min	1000
Max Vacuum for Continuous Operation mBar	210
<b>DIMENSIONS</b>	
Height mm	470
Width mm	420
Depth mm	350
Weight kg	31
Shipping Weight kg	33

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Micro SMART Cube with ISO 6 Hydrocyclone | 1-2 chairs

The most compact of our high-performance SMART suction Cube range, the Micro SMART is designed to serve 1 to 2 chairs. Integrated with amalgam retention, the Cube's acoustic housing cuts perceived noise by 50%.

### Details That Matter

- Acoustic housing reducing perceived noise by 50%
- ISO 6 Hydrocyclone amalgam retention
- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT



LOW  
NOISE

#### Micro SMART Cube with ISO 6 Hydrocyclone

Item Code	035030
Price Ex GST	\$9,450.00
Price Inc GST	\$10,395.00

#### SPECIFICATIONS

Power	0.95KW 5.8A 1PH 230V
Sound Output dB(A)	59
Max Air Flow at l/min	1000
Max Vacuum for Continuous Operation mBar	210

#### DIMENSIONS

Height mm	470
Width mm	420
Depth mm	350
Weight kg	31
Shipping Weight kg	34

#### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Turbo SMART A | 2-3 chairs

Designed with Cattani pioneered inverter technology, the Turbo SMART A delivers consistent, high-performance suction to 2 to 3 chairs. Capable of upgrading to Turbo SMART B to serve 4 chairs, without the need for additional space.

### Details That Matter

- Capability to upgrade to a 4-chair capacity
- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities
- Amalgam retention option available
- Noise reduction option available (see SMART Cube)



INVERTER  
CONTROL



INCLUDES  
INSTALLATION  
KIT

Turbo SMART A	
Item Code	035105
Price Ex GST	\$10,850.00
Price Inc GST	\$11,935.00
<b>SPECIFICATIONS</b>	
Power	1.5kW 6.5A 1PH 230V
Sound Output dB(A)	69
Max Air Flow at l/min	1400
Max Vacuum for Continuous Operation mBar	210
<b>DIMENSIONS</b>	
Height mm	645
Width mm	400
Depth mm	330
Weight kg	49
Shipping Weight kg	52

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets



## Turbo SMART A with ISO 18 Hydrocyclone | 2-3 chairs

Designed with inverter technology, the Turbo SMART A delivers consistent, high-performance suction to 2 to 3 chairs. Integrated with amalgam retention, with the capability of upgrading to Turbo SMART B to serve 4 chairs.

### Details That Matter

- Capability to upgrade to a 4-chair capacity
- ISO 18 Hydrocyclone amalgam retention
- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities.
- Noise reduction option available (see SMART Cube)



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT

### Turbo SMART A with ISO 18 Hydrocyclone

Item Code	035115
Price Ex GST	\$11,475.00
Price Inc GST	\$12,622.50

#### SPECIFICATIONS

Power	1.5kW 6.5A 1PH 230V
Sound Output dB(A)	69
Max Air Flow at l/min	1400
Max Vacuum for Continuous Operation mBar	210

#### DIMENSIONS

Height mm	645
Width mm	400
Depth mm	330
Weight kg	50
Shipping Weight kg	54

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Turbo SMART A Cube | 2-3 chairs

Designed with inverter technology, the Turbo SMART A delivers consistent, high-performance suction to 2 to 3 chairs. Comes with noise reduction acoustic housing, with the capability of upgrading to Turbo SMART B to serve 4 chairs.

### Details That Matter

- Acoustic housing reducing perceived noise by 50%
- Capability to upgrade to a 4-chair capacity
- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities
- Amalgam retention option available



INVERTER  
CONTROL



INCLUDES  
INSTALLATION  
KIT



LOW  
NOISE

Turbo SMART A Cube	
Item Code	035121
Price Ex GST	\$11,800.00
Price Inc GST	\$12,980.00
<b>SPECIFICATIONS</b>	
Power	1.1kW 6.8A 1PH 230V
Sound Output dB(A)	60
Max Air Flow at l/min	1400
Max Vacuum for Continuous Operation mBar	210
<b>DIMENSIONS</b>	
Height mm	650
Width mm	500
Depth mm	370
Weight kg	43
Shipping Weight kg	53

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Turbo SMART A Cube with ISO 18 Hydrocyclone | 2-3 chairs

Designed with inverter technology, the Turbo SMART A delivers consistent, high-performance suction to 2 to 3 chairs. Integrated with Hydrocyclone amalgam retention, with the capability of upgrading to Turbo SMART B to serve 4 chairs.

### Details That Matter

- Acoustic housing reducing perceived noise by 50%
- ISO 18 Hydrocyclone amalgam retention
- Capability to upgrade to a 4-chair capacity
- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT



LOW  
NOISE

### Turbo SMART A Cube with ISO 18 Hydrocyclone

Item Code	035120
Price Ex GST	\$12,425.00
Price Inc GST	\$13,667.50

#### SPECIFICATIONS

Power	1.1kW 6.8A 1PH 230V
Sound Output dB(A)	60
Max Air Flow at l/min	1400
Max Vacuum for Continuous Operation mBar	210

#### DIMENSIONS

Height mm	650
Width mm	500
Depth mm	370
Weight kg	44
Shipping Weight kg	55

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Turbo SMART B | Up to 4 chairs

Designed with Cattani pioneered inverter technology, the Turbo SMART B adjusts performance according to suction demands, improving energy efficiency. This model delivers consistent, high-performance suction to 4 chairs.

### Details That Matter

- Inverter technology for performance control and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities
- Amalgam retention option available
- Noise reduction option available (see SMART Cube)



INVERTER  
CONTROL



INCLUDES  
INSTALLATION  
KIT

Turbo SMART B	
Item Code	035105B
Price Ex GST	\$12,225.00
Price Inc GST	\$13,447.50
<b>SPECIFICATIONS</b>	
Power	1.8kW 7.5A 1PH 230V
Sound Output dB(A)	73.7
Max Air Flow at l/min	1750
Max Vacuum for Continuous Operation mBar	210
<b>DIMENSIONS</b>	
Height mm	645
Width mm	400
Depth mm	330
Weight kg	50
Shipping Weight kg	52

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Turbo SMART B with ISO 18 Hydrocyclone | Up to 4 chairs

Designed with Cattani pioneered inverter technology, the Turbo SMART B delivers consistent, high-performance suction to 4 chairs. Integrated with ISO 18 Hydrocyclone amalgam retention.

### Details That Matter

- Inverter control for performance and energy efficiency
- ISO 18 Hydrocyclone amalgam retention
- SMART self-protection in case of functional anomalies
- Modular installation capabilities
- Noise reduction option available (see SMART Cube)



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT

### Turbo SMART B with ISO 18 Hydrocyclone

Item Code	035115B
Price Ex GST	\$12,850.00
Price Inc GST	\$14,135.00

#### SPECIFICATIONS

Power	1.8kW 7.5A 1PH 230V
Sound Output dB(A)	73.7
Max Air Flow at l/min	1750
Max Vacuum for Continuous Operation mBar	210

#### DIMENSIONS

Height mm	645
Width mm	400
Depth mm	330
Weight kg	50
Shipping Weight kg	54

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets



## Turbo SMART B Cube | Up to 4 chairs

Designed with Cattani-pioneered inverter technology, the Turbo SMART B adjusts performance according to suction demands, improving energy efficiency. Comes with noise reduction acoustic housing, delivering high-performance suction to 4 chairs.

### Details That Matter

- Acoustic housing reducing perceived noise by 50%
- Inverter technology for performance control and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities
- Amalgam retention option available



INVERTER  
CONTROL



INCLUDES  
INSTALLATION  
KIT



LOW  
NOISE

Turbo SMART B Cube	
Item Code	035121B
Price Ex GST	\$13,175.00
Price Inc GST	\$14,492.50
<b>SPECIFICATIONS</b>	
Power	1.5kW 8.7A 1PH 230V
Sound Output dB(A)	60
Max Air Flow at l/min	1700
Max Vacuum for Continuous Operation mBar	210
<b>DIMENSIONS</b>	
Height mm	650
Width mm	500
Depth mm	370
Weight kg	44
Shipping Weight kg	53

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Turbo SMART B Cube with ISO 18 Hydrocyclone | Up to 4 chairs

Designed with Cattani-pioneered inverter technology, the Turbo SMART B delivers consistent, high-performance suction to 4 chairs. Comes with noise reduction acoustic housing and integrated with ISO 18 Hydrocyclone amalgam retention.

### Details That Matter

- ISO 18 Hydrocyclone amalgam retention
- Acoustic housing reducing perceived noise by 50%
- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT



LOW  
NOISE

### Turbo SMART B Cube with ISO 18 Hydrocyclone

Item Code	035120B
Price Ex GST	\$13,800.00
Price Inc GST	\$15,180.00

#### SPECIFICATIONS

Power	1.5kW 8.7A 1PH 230V
Sound Output dB(A)	60
Max Air Flow at l/min	1700
Max Vacuum for Continuous Operation mBar	210

#### DIMENSIONS

Height mm	650
Width mm	500
Depth mm	370
Weight kg	44
Shipping Weight kg	55

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Turbo SMART B 2V | Up to 4 chairs

Designed with Cattani-pioneered inverter technology, the Turbo SMART B delivers consistent, high-performance suction to 4 chairs. Comes with noise reduction acoustic housing and integrated with ISO 18 Hydrocyclone amalgam retention.

### Details That Matter

- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities



INVERTER  
CONTROL



INCLUDES  
INSTALLATION  
KIT

INDENT ITEM	Turbo SMART B 2V
Item Code	035130
Price Ex GST	\$14,500.00
Price Inc GST	\$15,950.00
<b>SPECIFICATIONS</b>	
Power	2.2kW 9.8A 1PH 230V
Sound Output dB(A)	70
Max Air Flow at l/min	1600
Max Vacuum for Continuous Operation mBar	250
<b>DIMENSIONS</b>	
Height mm	650
Width mm	540
Depth mm	400
Weight kg	41
Shipping Weight kg	63



### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Turbo SMART B 2V with ISO 18 Hydrocyclone | Up to 4 chairs

Designed with Cattani-pioneered inverter technology, the Turbo SMART B delivers consistent, high-performance suction to 4 chairs. Comes with noise reduction acoustic housing and integrated with ISO 18 Hydrocyclone amalgam retention.

### Details That Matter

- ISO 18 Hydrocyclone amalgam retention
- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Modular installation capabilities



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT

INDENT ITEM	Turbo SMART B 2V
Item Code	035135
Price Ex GST	\$15,125.00
Price Inc GST	\$16,637.50
<b>SPECIFICATIONS</b>	
Power	2.2kW 9A 1PH 230V
Sound Output dB(A)	70
Max Air Flow at l/min	1600
Max Vacuum for Continuous Operation mBar	250
<b>DIMENSIONS</b>	
Height mm	650
Width mm	540
Depth mm	400
Weight kg	43
Shipping Weight kg	63

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Turbo SMART HP | High-Performance | 338 mbar Vacuum Pressure | 3 Chairs\*

### Raising The Bar

From dental hygiene to oral surgery, the all-new TURBO SMART HP is the next generation of first-class dental suction to deliver constant peak performance, fully customisable to suit your dental clinic and individual requirements.

### Details That Matter

- High performance 338 mBar vacuum pressure for highly personalised dental suction performance
- Capability to customise suction power to serve 4 surgeries\*
- Purpose-built for intraoral dental aerosol control
- 70db(A) – improved noise reduction via ventilation technology
- Improved dual-contour blade design for more suction power while using less energy
- Additional vacuum pressure ideal for where dental surgeries have challenging pipework



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT



INDENT ITEM	Turbo SMART HP with ISO 18 Hydrocyclone
Item Code	035145
Price Ex GST	\$18,950.00
Price Inc GST	\$20,845.00
<b>SPECIFICATIONS</b>	
Power	2.7kW 12A 1PH 230V
Sound Output dB(A)	70
Max Air Flow at l/min	1916
Max Vacuum for Continuous Operation mBar	338
<b>DIMENSIONS</b>	
Height mm	765
Width mm	460
Depth mm	350
Weight kg	55

\* 3 chairs as standard, 4 chairs dependent on maximum head setting.  
For more information, please contact us on +61 3 9484 1120.

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets



# TURBO SMART *TS*



## **The Next Generation, Reimagined.**

Unprecedented Power, Precision and Safety  
Combined for Five Dental Chairs.

**Launching at ADX Sydney 2026**



HEPA H14

300 l/min

210 mBar

Hg



5

## Tandem Turbo SMART B | Up to 8 chairs

Designed with Cattani pioneered inverter technology, the Turbo SMART B units are installed in parallel to deliver twice the suction output. Adjusts performance according to suction demands to serve 8 chairs.

### Details That Matter

- Installed in parallel with twice the suction
- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Compatible with Vento frame
- Amalgam retention option available



Model with ISO 18 Hydrocyclone pictured.



INVERTER  
CONTROL



INCLUDES  
INSTALLATION  
KIT

Tandem Turbo SMART B	
Item Code	035105BX2
Price Ex GST	\$27,850.00
Price Inc GST	\$30,635.00
<b>SPECIFICATIONS</b>	
Power	1.8kW 7.5A 1PH 230V 2 x 10A power outlets required
Sound Output dB(A)	76.7
Max Air Flow at l/min	3500
Max Vacuum for Continuous Operation mBar	210
<b>DIMENSIONS</b>	
Height mm	740
Width mm	1250
Depth mm	760
Weight kg	85
Shipping Weight kg	112

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Tandem Turbo SMART B with ISO 18 Hydrocyclone | Up to 8 chairs

Integrated with ISO 18 Hydrocyclone amalgam retention, the Turbo SMART B units are installed in parallel to deliver twice the suction output. Adjusts performance according to suction demands to serve 8 chairs.

### Details That Matter

- ISO 18 Hydrocyclone amalgam retention
- Installed in parallel with twice the suction
- Capability to upgrade to an 8-chair capacity
- Inverter control for performance and energy efficiency
- SMART self-protection in case of functional anomalies
- Compatible with Vento frame



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT

### Tandem Turbo SMART B with 2 x ISO 18 Hydrocyclone

Item Code	035115BX2
Price Ex GST	\$29,100.00
Price Inc GST	\$32,010.00

#### SPECIFICATIONS

Power	1.8kW 7.5A 1PH 230V 2 x 10A power outlets required
Sound Output dB(A)	76.7
Max Air Flow at l/min	3500
Max Vacuum for Continuous Operation mBar	210

#### DIMENSIONS

Height mm	740
Width mm	1250
Depth mm	760
Weight kg	85
Shipping Weight kg	118

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

# LARGE PLANT ROOM SOLUTIONS



## **MULTI SMART: DESIGNED FOR THE AUSTRALASIAN REGION**

### **12-20 CHAIRS**

Leading innovation without compromising quality, our Australian technical team collaborated with Cattani Italy to co-engineer the new generation of the Multi SMART range, a more compact 12 – 20 chair solution specifically designed for the Australasian region.

The turn-key solution comes in the following configurations:

- T3 Model – up to 12 chairs.
- T4 Model – up to 16 chairs.
- T5 Model – up to 20 chairs.

### **EASY MAINTENANCE, NO DISRUPTIONS**

Installation of Multi SMART system is a simple process with a modular design that allows easy access for maintenance and HEPA filter upgrade onsite.

Each suction module of the system can be isolated to allow for easy servicing while still delivering uninterrupted suction, keeping the dental site running without disruption.

### **CONSISTENT AND RELIABLE SUCTION**

Multi SMART systems are assembled in parallel, and thanks to inverter technology, provide steady and reliable suction with built-in redundancy.

## **MAXI SMART: THE WORLD'S FIRST INTEGRATED SUCTION UNIT FOR LARGE CLINICS**

### **20-60 CHAIRS**

The Maxi SMART system is the world's first integrated suction unit for large clinics, with the capability to provide a constant level of suction at each individual dental chair.

The system is designed to be installed as a modular system, in pairs, with the potential to serve 60 or more chairs. The system units are built with sequential activation intelligence, according to suction demand. Maxi SMART systems are supplied in kit form, with all interconnecting components necessary for parallel installation.

## **AT CATTANI, WE FOCUS ON INNOVATION.**





## Multi SMART T3 | Up to 12 chairs

Designed for the Australasian region, our Italian and Australian engineering and technical teams worked closely to develop a new solution that is more compact with no compromise on performance. The Multi SMART T3 is designed to serve up to 12 chairs.

### Details That Matter

- Compact design where space is a premium
- Simple installation and easy access for servicing
- Allows continuous operation even during maintenance
- Designed with built-in redundancy
- Modular design allowing for easy HEPA upgrade onsite
- Amalgam retention option available



Hydrocyclone model shown.



INVERTER  
CONTROL



INCLUDES  
INSTALLATION  
KIT

MADE TO ORDER		Multi SMART T3
Description	Multi SMART T3	Multi SMART T3 with H14 Filtration
Item Code	035105BX3	035105BHX3
Price Ex GST	\$57,750.00	\$62,250.00
Price Inc GST	\$63,525.00	\$68,475.00
SPECIFICATIONS		
Power	3 x 1.8kW 7.5A 1PH 230V 1x 10A (control panel) Total of 4 x 10A power outlets required	3 x 1.8kW 7.5A 1PH 230V 1x 10A (control panel) Total of 4 x 10A power outlets required
Sound Output dB(A)	78.5	78.5
Max Air Flow at l/min	5100	5100
Max Vacuum for Continuous Operation mBar	210	210
DIMENSIONS		
Height mm	1280	1280
Width mm*	1000	1000
Depth mm	760	760
Weight kg	150	157
Shipping Weight kg	189	196

\*Dimensions are for frame only. Please allow extra space for installation purposes. Contact our technical team for further information.

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Multi SMART T3 with ISO 18 Hydrocyclone | Up to 12 chairs

Designed for the Australasian region, our Italian and Australian engineering and technical teams worked closely to develop more compact solution. Integrated with ISO 18 Hydrocyclone amalgam retention, the Multi SMART T3 is designed to serve up to 12 chairs.

### Details That Matter

- ISO 18 Hydrocyclone amalgam retention
- Compact design where space is a premium
- Simple installation and easy access for servicing
- Allows continuous operation even during maintenance
- Designed with built-in redundancy
- Modular design allowing for easy HEPA upgrade onsite



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT

#### MADE TO ORDER

#### Multi SMART T3

Description	Multi SMART T3 with ISO 18 Hydrocyclone	Multi SMART T3 with ISO 18 Hydrocyclone with H14 Filtration
Item Code	035115BX3	035115BHX3
Price Ex GST	\$61,750.00	\$66,250.00
Price Inc GST	\$67,925.00	\$72,875.00

#### SPECIFICATIONS

Power	3 x 1.8kW 7.5A 1PH 230V 1x 10A (control panel) Total of 4 x 10A power outlets required	3 x 1.8kW 7.5A 1PH 230V 1x 10A (control panel) Total of 4 x 10A power outlets required
Sound Output dB(A)	78.5	78.5
Max Air Flow at l/min	5100	5100
Max Vacuum for Continuous Operation mBar	210	210

#### DIMENSIONS

Height mm	1280	1280
Width mm*	1000	1000
Depth mm	760	760
Weight kg	184	186
Shipping Weight kg	204	211

\*Dimensions are for frame only. Please allow extra space for installation purposes. Contact our technical team for further information.

#### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Multi SMART T4 | Up to 16 chairs

Designed for the Australasian region, our Italian and Australian engineering and technical teams worked closely to develop a new solution that is more compact with no compromise on performance. The Multi SMART T4 is designed to serve up to 16 chairs.

### Details That Matter

- Compact design where space is a premium
- Simple installation and easy access for servicing
- Allows continuous operation even during maintenance
- Designed with built-in redundancy
- Modular design allowing for easy HEPA upgrade onsite
- Amalgam retention option available



Hydrocyclone model shown.



INVERTER  
CONTROL



INCLUDES  
INSTALLATION  
KIT

MADE TO ORDER		Multi SMART T4
Description	Multi SMART T4	Multi SMART T4 with H14 Filtration
Item Code	035105BX4	035105BHX4
Price Ex GST	\$75,150.00	\$81,150.00
Price Inc GST	\$82,665.00	\$89,265.00
SPECIFICATIONS		
Power	4 x 1.8kW 7.5A 1PH 230V 1 x 10A (control panel) Total of 5 x 10A power outlets required	4 x 1.8kW 7.5A 1PH 230V 1 x 10A (control panel) Total of 5 x 10A power outlets required
Sound Output dB(A)	79.7	79.7
Max Air Flow at l/min	6800	6800
Max Vacuum for Continuous Operation mBar	210	210
DIMENSIONS		
Height mm	1280	1280
Width mm*	1345	1345
Depth mm	760	760
Weight kg	212	224
Shipping Weight kg	234	254

\*Dimensions are for frame only. Please allow extra space for installation purposes. Contact our technical team for further information.

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Multi SMART T4 with ISO 18 Hydrocyclone | Up to 16 chairs

Designed for the Australasian region, our Italian and Australian engineering and technical teams worked closely to develop more compact solution. Integrated with ISO 18 Hydrocyclone amalgam retention, the Multi SMART T4 is designed to serve up to 16 chairs.

### Details That Matter

- ISO 18 Hydrocyclone amalgam retention
- Compact design where space is a premium
- Simple installation and easy access for servicing
- Allows continuous operation even during maintenance
- Designed with built-in redundancy
- Modular design allowing for easy HEPA upgrade onsite



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT

#### MADE TO ORDER

#### Multi SMART T4

Description	Multi SMART T4 with ISO 18 Hydrocyclone	Multi SMART T4 with ISO 18 Hydrocyclone with H14 Filtration
Item Code	035115BX4	035115BHX4
Price Ex GST	\$80,525.00	\$86,525.00
Price Inc GST	\$88,577.50	\$95,177.50

#### SPECIFICATIONS

Power	4 x 1.8kW 7.5A 1PH 230V 1 x 10A (control panel) Total of 5 x 10A power outlets required	4 x 1.8kW 7.5A 1PH 230V 1 x 10A (control panel) Total of 5 x 10A power outlets required
Sound Output dB(A)	79.7	79.7
Max Air Flow at l/min	6800	6800
Max Vacuum for Continuous Operation mBar	210	210

#### DIMENSIONS

Height mm	1280	1280
Width mm*	1345	1345
Depth mm	760	760
Weight kg	224	249
Shipping Weight kg	254	274

\*Dimensions are for frame only. Please allow extra space for installation purposes. Contact our technical team for further information.

#### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets



## Multi SMART T5 | Up to 20 chairs

Designed for the Australasian region, our Italian and Australian engineering and technical teams worked closely to develop a new solution that is more compact with no compromise on performance. The Multi SMART T5 is designed to serve up to 20 chairs.

### Details That Matter

- Compact design where space is a premium
- Simple installation and easy access for servicing
- Allows continuous operation even during maintenance
- Designed with built-in redundancy
- Modular design allowing for easy HEPA upgrade onsite
- Amalgam retention option available



Hydrocyclone model shown.



INVERTER  
CONTROL



INCLUDES  
INSTALLATION  
KIT

MADE TO ORDER		Multi SMART T5
Description	Multi SMART T5	Multi SMART T5 with H14 Filtration
Item Code	035105BX5	035105BHX5
Price Ex GST	\$89,500.00	\$97,000.00
Price Inc GST	\$98,450.00	\$106,700.00
SPECIFICATIONS		
Power	5 x 1.8kW 7.5A 1PH 230V 1 x 10A (control panel) Total of 6 x 10A power outlets required	5 x 1.8kW 7.5A 1PH 230V 1 x 10A (control panel) Total of 6 x 10A power outlets required
Sound Output dB(A)	80.7	80.7
Max Air Flow at l/min	8500	8500
Max Vacuum for Continuous Operation mBar	210	210
DIMENSIONS		
Height mm	1280	1280
Width mm*	1690	1690
Depth mm	760	760
Weight kg	265	272
Shipping Weight kg	290	315

\*Dimensions are for frame only. Please allow extra space for installation purposes. Contact our technical team for further information.

### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

## Multi SMART T5 with ISO 18 Hydrocyclone | Up to 20 chairs

Designed for the Australasian region, our Italian and Australian engineering and technical teams worked closely to develop more compact solution. Integrated with ISO 18 Hydrocyclone amalgam retention, the Multi SMART T5 is designed to serve up to 20 chairs.

### Details That Matter

- ISO 18 Hydrocyclone amalgam retention
- Compact design where space is a premium
- Simple installation and easy access for servicing
- Allows continuous operation even during maintenance
- Designed with built-in redundancy
- Modular design allowing for easy HEPA upgrade onsite



INVERTER  
CONTROL



AMALGAM  
RETENTION



INCLUDES  
INSTALLATION  
KIT

#### MADE TO ORDER

#### Multi SMART T5

Description	Multi SMART T5 with ISO 18 Hydrocyclone	Multi SMART T5 with ISO 18 Hydrocyclone with H14 Filtration
Item Code	035115BX5	035115BHX5
Price Ex GST	\$96,500.00	\$104,000.00
Price Inc GST	\$106,150.00	\$114,400.00

#### SPECIFICATIONS

Power	5 x 1.8kW 7.5A 1PH 230V 1 x 10A (control panel) Total of 6 x 10A power outlets required	5 x 1.8kW 7.5A 1PH 230V 1 x 10A (control panel) Total of 6 x 10A power outlets required
Sound Output dB(A)	80.7	80.7
Max Air Flow at l/min	8500	8500
Max Vacuum for Continuous Operation mBar	210	210

#### DIMENSIONS

Height mm	1280	1280
Width mm*	1690	1690
Depth mm	760	760
Weight kg	265	272
Shipping Weight kg	290	315

\*Dimensions are for frame only. Please allow extra space for installation purposes. Contact our technical team for further information.

#### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets

# COMPACT PLANT ROOM SOLUTIONS




INVERTER  
CONTROL



AMALGAM  
RETENTION

## SPACE IS A PREMIUM

Space is valuable and, in many practices, strictly limited. There can also be issues with noise upsetting staff, patients and even neighbours. Cattani understands that, and provides compact solutions in a variety of configurations and noise-reducing options that can even do away with the need for a plant room entirely.

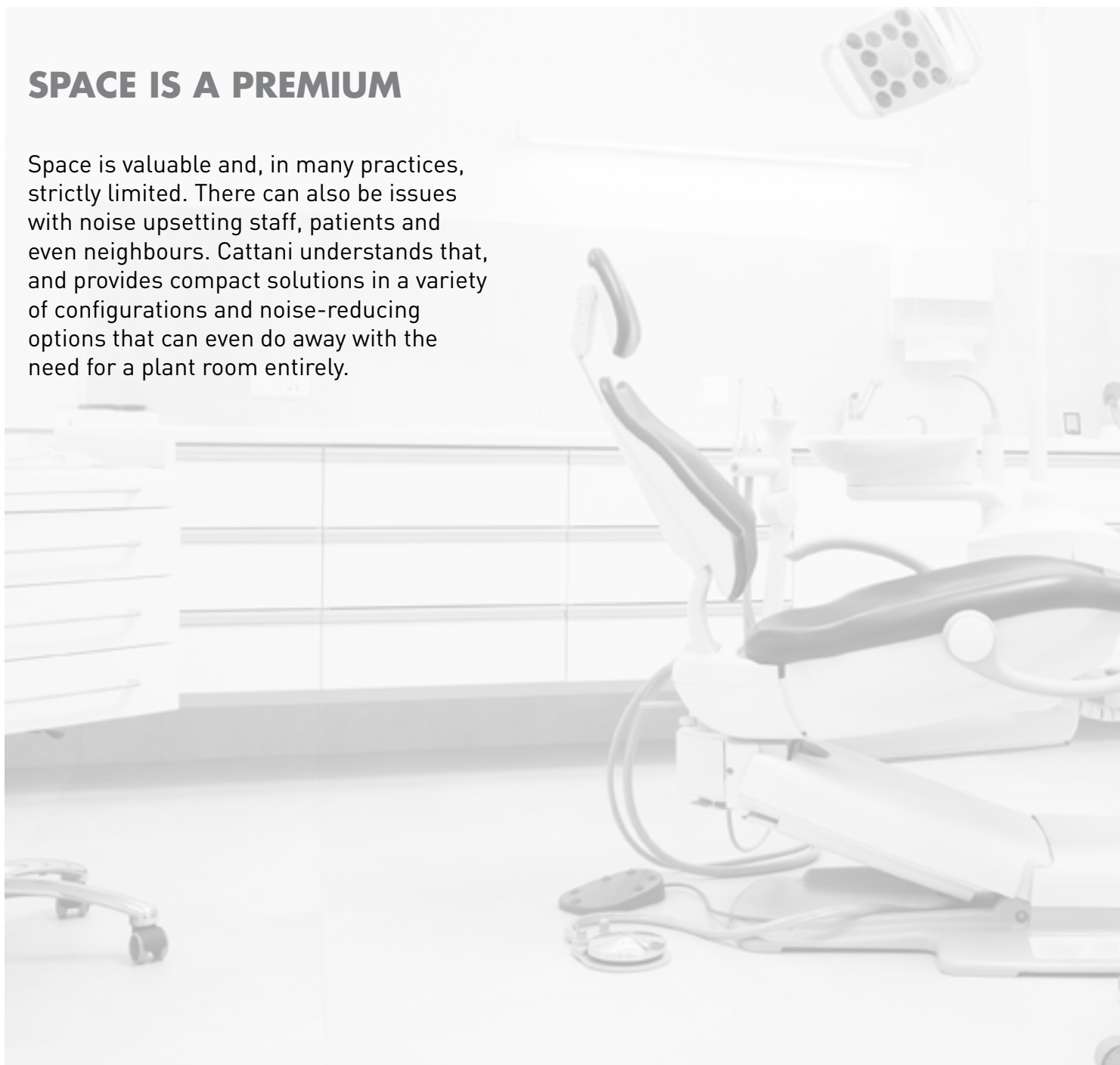



### Blok-Jet Silent 1 & 2: the self-contained plant solutions | 1-2 and 3-4 chairs

The Blok-Jet Silent solution is to stack the units inside their own cabinet. Serves up to four chairs and is ultra-quiet, it is an excellent option when noise mitigation is a priority. The cabinet also has a very small footprint and stands on caster wheels, for ease of positioning and service.

#### Details That Matter

- Serves up to four chairs
- Compact – the size of a fridge
- Ultra-quiet
- Mobile – it stands on lockable wheels
- Cabinet doors also lockable
- External display panel shows compressor and suction status
- Arrives assembled to save installation time



Blok-Jet Silent 2 pictured.

INDENT ITEM	Blok-Jet Silent 1	Blok-Jet Silent 2
	Blok-Jet Silent 1 AC200 Compressor & Micro SMART with Hydrocyclone	Blok-Jet Silent 2 AC300 Compressor & Turbo SMART B with Hydrocyclone
Item Code	073500	073525
Price Ex GST	\$27,450.00	\$38,705.00
Price Inc GST	\$30,195.00	\$42,575.50
<b>SPECIFICATIONS</b>		
Compressor	AC200	AC300
Suction	Micro SMART	Turbo SMART B
Sound Output dB(A)	57.5	55
<b>DIMENSIONS</b>		
Height mm	1583	1935
Width mm	727	760
Depth mm	577	650
Weight kg	220	280
Shipping Weight kg		Upon Request

#### RELATED PRODUCTS



PULI-JET Gentle 2.0  
Non-foaming  
Disinfectant



Pulse Cleaner



Antifoaming  
Disinfectant Tablets



## Vento solutions: the ultimate space saver for up to eight chairs

Designed and developed by Cattani Australia, the Vento approach is to stack the suction unit and the compressor on an engineered frame, halving the floor space. This is the perfect plant configuration for practices where space is tight, and a dedicated plant room is a luxury.

Vento can be configured to service up to eight chairs. Units with noise reducing options are also available.

## Vento solutions can incorporate:

- Micro SMART
- Turbo SMART
- Noise-reducing SMART Cube suction units
- Our entire compressor range, including CAD/CAM
- Acoustic compressor hoods (Q Series)
- Amalgam retention



FRAME 1  
Example with AC200 and  
Turbo SMART with Hydrocyclone



FRAME 2  
Example with AC600 and Tandem  
Turbo SMART with Hydrocyclone

## How to configure your Vento solution

1. Select your Compressor (pages 6-17)
2. Select your Suction (pages 43-61)
3. Select your Vento frame based on your compressor size (except when choosing Turbo SMART Cube suction system)\*
4. Combine the pricing for your compressor, suction, and Vento frame to determine the overall price of the Vento solution.

	FRAME 1 (Turbo SMART)	FRAME 1B (Turbo SMART Cube)	FRAME 2
Order number	JVT0000	JVTTSC0	JCSTTS
Compatibility	AC100, AC200 & AC300 Series	AC100, AC200 & AC300 Series	Compatible with AC400 & AC600 Series
Price Inc GST	\$2,393.60	\$2,393.60	\$2,922.70
<b>DIMENSIONS</b>			
Height mm	820	820	950
Width mm	695	695	1280
Depth mm	490	610	600
Shipping Weight kg	Upon Request		

\*Frames are only sold as part of a complete Vento configuration. For the individual sale of the frame component, contact your local Business Development Manager.

# CLASSIC SUCTION

## DRY SUCTION: PROVEN CATTANI TECHNOLOGY

### BUILT TO LAST

Cattani dry suction motors deliver high volume airflow with high pressure, even with a large suction tip. Our separators, control panels, and vacuum relief valves are all you need if dry suction is your preferred option.

Cattani dry suction motors are made from quality materials and built to last. The machines are incredibly easy to operate, and the mechanics and circuitry are easy to maintain:

- 1-7 chairs
- Single or three-phase, depending on the unit
- Noise reduction options available
- Units designed to be installed close to the dental unit

## ASPI-JET: MOBILE OR BACKUP SUCTION SYSTEM

### PLUG AND PLAY

If you need a simple, safe, and mobile chairside aspirator for a single chair, Aspi-Jet 6 is for you – just plug it in and turn it on. Aspi-Jet 6 is designed for continuous use and reliable high flow, medium vacuum, and effective air/fluid separation.

### MOBILE RELIABILITY

Aspi-Jet 6 is on wheels for easy transfer from surgery to surgery. It is a handy back-up system and a good option where it is difficult to get plumbing to the dental unit.

Powered by a robust Uni-Jet 75 motor, the Aspi-Jet has a maximum negative pressure of 130 mBar with maximum air flow rate 650 l/min.

The Aspi-Jet range includes plumbed versions in the Aspi-Jet 7, 8 and 9 models that include auto-drain options, all with foam-lined cabinet for quietness.



## MOBILE SUCTION



**IMPORTANT!** The air exhausted by the Aspi-Jet should be filtered, with a certified antibacterial filter available on request, and expelled to the outside.



Aspi-Jet 6		
Description	Unit without Bacterial Filter	Unit with Bacterial Filter
Item Code	010010	010010B
Price Ex GST	\$5,918.00	\$6,777.00
Price Inc GST	\$6,509.80	\$7,454.70
<b>SPECIFICATIONS</b>		
Sound Output dB(A)	60	60
Max Air Flow at l/min	650	650
Max Vacuum for Continuous Operation mBar	130	130
Tank Size (litres)	4	4
<b>DIMENSIONS</b>		
Height mm	750	750
Width mm	325	325
Depth mm	420	420

Manual drain 4 litre canister capacity.



INDENT ITEMS	Aspi-Jet 7	Aspi-Jet 8	Aspi-Jet 9
Item Code	010021	010030	010041
Price Ex GST	\$7,231.00	\$8,942.00	\$10,255.00
Price Inc GST	\$7,954.10	\$9,836.20	\$11,280.50
<b>SPECIFICATIONS</b>			
Sound Output dB(A)	60	60	60
Max Air Flow at l/min	650	650	650
Max Vacuum for Continuous Operation mBar	130	130	130
<b>DIMENSIONS</b>			
Height mm	750	750	750
Width mm	325	325	325
Depth mm	420	420	520
	Automatic drain pump	Automatic drain pump & cup filler	Automatic drain pump, cup filler & spittoon
	Requires plumbing to waste	Requires plumbing to waste Requires water supply	Requires plumbing to waste Requires water supply

## Uni-Jet 75 Single Phase | 1 chair



Uni-Jet 75	
Description	Single Phase
Item Code	020322
Price Ex GST	\$1,253.00
Price Inc GST	\$1,378.30

### SPECIFICATIONS

Power	0.42kW 3.1A 1PH
Sound Output dB(A)	62
Max Air Flow at l/min	1250
Max Vacuum for Continuous Operation mBar	130

### DIMENSIONS

Height mm	275
Width mm	232
Depth mm	246

### Accessories

C-020632 Control Panel – H400mm x W300mm x D170mm	\$4,785.00
C-160562 Silencer – L400mm x Dia. 80mm (30mm ports)	\$250.80

## Uni-Jet 75 2V Single Phase | 1 chair



Uni-Jet 75 2V	
Description	Single Phase
Item Code	020370
Price Ex GST	\$2,322.00
Price Inc GST	\$2,554.20

### SPECIFICATIONS

Power	0.7kW 5.2A 1PH
Sound Output dB(A)	64
Max Air Flow at l/min	1330
Max Vacuum for Continuous Operation mBar	270

### DIMENSIONS

Height mm	275
Width mm	246
Depth mm	491

### Accessories

C-020632 Control Panel – H400mm x W300mm x D170mm	\$4,785.00
C-160562 Silencer – L400mm x Dia. 80mm (30mm ports)	\$250.80

## Tecno-Jet Single Phase | 2-3 chairs



Tecno-Jet	
Description	Single Phase
Item Code	046759
Price Ex GST	\$2,252.00
Price Inc GST	\$2,477.20

### SPECIFICATIONS

Power	0.75kW 5.5A 1PH
Sound Output dB(A)	64
Max Air Flow at l/min	2000
Max Vacuum for Continuous Operation mBar	140

### DIMENSIONS

Height mm	320
Width mm	310
Depth mm	310

### Accessories

C-020631 Control Panel – H400mm x W300mm x D170mm	\$4,785.00
C-160564 Silencer – L400mm x Dia. 160mm (50mm ports)	\$415.80
C-A31080 Vacuum Relief Valve	\$335.23

## Tecno-Jet 2V Single Phase | 2-3 chairs



INDENT ITEM	Tecno-Jet 2V
Description	Single Phase
Item Code	020921
Price Ex GST	\$4,752.00
Price Inc GST	\$5,227.20

### SPECIFICATIONS

Power	1.1kW 7.5A 1PH
Sound Output dB(A)	65
Max Air Flow at l/min	2170
Max Vacuum for Continuous Operation mBar	200

### DIMENSIONS

Height mm	320
Width mm	310
Depth mm	550

### Accessories

C-020631 Control Panel – H400mm x W300mm x D170mm	\$4,785.00
C-160564 Silencer – L400mm x Dia. 160mm (50mm ports)	\$415.80
C-A31080 Vacuum Relief Valve	\$335.23



## Flux-Jet 1.1kW Single Phase | 4 chairs



Flux-Jet 1.1kW	
Description	Single Phase
Item Code	048137
Price Ex GST	\$2,754.00
Price Inc GST	\$3,029.40
<b>SPECIFICATIONS</b>	
Power	1.1kW 7.6A 1PH
Sound Output dB(A)	68.5
Max Air Flow at l/min	3300
Max Vacuum for Continuous Operation mBar	150
<b>DIMENSIONS</b>	
Height mm	350
Width mm	340
Depth mm	410
<b>Accessories</b>	
C-020630 Control Panel – H400mm x W300mm x D170mm	\$4,785.00
C-160564 Silencer – L400mm x Dia. 160mm (50mm ports)	\$415.80
C-400870 Vacuum Relief Valve	\$323.13

## Flux-Jet 2V 3 Phase | 4 chairs



Flux-Jet 2V	
Description	3 Phase
Item Code	020715
Price Ex GST	\$7,560.00
Price Inc GST	\$8,316.00
<b>SPECIFICATIONS</b>	
Power	2.2kW 5.2A 3PH
Sound Output dB(A)	72
Max Air Flow at l/min	3300
Max Vacuum for Continuous Operation mBar	230
<b>DIMENSIONS</b>	
Height mm	350
Width mm	340
Depth mm	590
<b>Accessories</b>	
C-020630 Control Panel – H400mm x W300mm x D170mm	\$4,785.00
C-160564 Silencer – L400mm x Dia. 160mm (50mm ports)	\$415.80
C-160640 Vacuum Relief Valve	\$748.55

## Medio-Jet Blower 3kW 3 Phase | 7 chairs



Medio-Jet Blower 3kW	
Description	3 Phase
Item Code	061503
Price Ex GST	\$4,320.00
Price Inc GST	\$4,752.00
SPECIFICATIONS	
Power	3kW 6.8A 3PH
Sound Output dB(A)	76
Max Air Flow at l/min	5000
Max Vacuum for Continuous Operation mBar	230
DIMENSIONS	
Height mm	420
Width mm	400
Depth mm	335
Accessories	
C-021050 Control Panel – H400mm x W300mm x D170mm	\$4,785.00
C-S2S000 Silencer – L265mm x Dia. 90mm (50mm ports)	\$323.68
C-A1549V Vacuum Relief Valve	\$1,081.30

## Uni-Jet 75 in Cover with Control



Uni-Jet 75	
Description	In Cover with Control
Item Code	020276
Price Ex GST	\$3,337.00
Price Inc GST	\$3,670.70
SPECIFICATIONS	
Power	0.42kW 3.1A 1PH
Sound Output dB(A)	58
Max Vacuum for Continuous Operation mBar	130
DIMENSIONS	
Height mm	330
Width mm	320
Depth mm	280
Accessories	
C-160562 Silencer – L400mm x Dia. 80mm (30mm ports)	\$250.80



## Continuous Suction Separator

Continuous Suction Separator		
Description	Multi Surgery Unit with Bracket	Multi Surgery Unit without Bracket
Item Code	C0016A0	C0016AB
Price Ex GST	\$2,300.00	\$2,155.00
Price Inc GST	\$2,530.00	\$2,370.50

### SPECIFICATIONS

Power	24V	24V
-------	-----	-----

### DIMENSIONS

Height mm	350	350
Width mm	250	250
Depth mm	100	100



## Continuous Suction Separator in Cover

Continuous Suction Separator in Cover		
INDENT ITEM	Single Surgery Unit	Multi Surgery Unit
Description		
Item Code	020998	020996
Price Ex GST	\$2,819.00	\$3,337.00
Price Inc GST	\$3,100.90	\$3,670.70

### SPECIFICATIONS

Power	24V	24V
-------	-----	-----

### DIMENSIONS

Height mm	350	350
Width mm	250	250
Depth mm	100	100

## SUCTION ACCESSORIES

### Control Panel – Classic Suction

The classic suction control panels include an off delay timer that avoids rapid stop/start of the motor.

Order Number	Dimensions	Model Suited	Price Inc GST
C-020632	Height 400mm Width 300mm Depth 170mm	Uni-Jet 1 Phase	\$4,785.00
C-020631	Height 400mm Width 300mm Depth 170mm	Tecno-Jet 1 Phase	\$4,785.00
C-020630	Height 400mm Width 300mm Depth 170mm	Flux-Jet 1 Phase	\$4,785.00
C-021050	Height 400mm Width 300mm Depth 170mm	All 3 Phase Units	\$4,785.00



### Vacuum Relief Valve – Tecno-Jet

Limits maximum vacuum to not overload the suction motor.

Order Number	Model Suited	Price Inc GST
C-A31080	Tecno-Jet	\$335.23



### Vacuum Relief Valve – Flux-Jet

Limits maximum vacuum to not overload the suction motor.

Order Number	Model Suited	Price Inc GST
C-400870	Flux-Jet	\$323.13



### Vacuum And Heat Relief Valve – Flux-Jet 2V

Limits maximum vacuum to not overload the suction motor.

Order Number	Model Suited	Price Inc GST
C-160640	Flux-Jet 2V	\$748.55



### Vacuum Relief Valve – Medio-Jet 2V

Limits maximum vacuum to not overload the suction motor.

Order Number	Model Suited	Price Inc GST
C-A1549V	Medio-Jet 2V	\$1,081.30



## SUCTION ACCESSORIES

### Silencer for Uni-Jet

Designed to reduce noise by 2dB (A).

Order Number	Dimensions	Pack Qty	Price Inc GST
C-160562	Length 400mm Diameter 80mm (30mm ports)	1	\$250.80



### Silencer for Tecno-Jet and Flux-Jet

Designed to reduce noise by 2dB (A).

Order Number	Dimensions	Pack Qty	Price Inc GST
C-160564	Length 440mm Diameter 160mm (50mm ports)	1	\$415.80



### Silencer for Medio-Jet

Designed to reduce noise by 2dB (A).

Order Number	Dimensions	Pack Qty	Price Inc GST
C-S2S000	Length 265mm Diameter 90mm (50mm ports)	1	\$323.68



### Heat Resistant Hose – Suitable for Suction or Exhaust Air

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-H40HR0	40mm	Black	\$368.23 PER M
C-H50HR0	50mm	Black	\$403.98 PER M
C-H60HR0	60mm	Black	\$418.00 PER M
C-H80HR0	80mm	Black	\$516.73 PER M



### Heat Resistant Hose – Suitable for Exhaust Air Only

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-H30EHR	30mm	Black	\$207.90 PER M
C-H40EHR	40mm	Black	\$239.25 PER M
C-H50EHR	50mm	Black	\$259.33 PER M



Available in up to 4 metre lengths



## SUCTION ACCESSORIES

### Genuine Cattani Air Flow Meter

Service technician tool to measure high volume evacuation terminals within the clinic ensuring appropriate levels of suction at each chair.

Order Number	Description	Pack Qty	Price Inc GST
C-A87000	Air flow meter, case & accessories	1	\$1,835.90



### Conversion Switching System

The conversion switching system is a 24V power supply that converts the SMART suction systems 2 wire switching system into a 3 wire system i.e. This is used to provide 24V power to isolation and spittoon drain valves. 30 VA (1.25 Amp). Please ensure that valves that are being powered by this panel do not exceed this rating. High rated power units are available upon request – contact our Cattani Technical Team for more information.

Order Number	Dimensions	Pack Qty	Price Inc GST
D-RZ0000	Height 200mm Width 240mm Depth 95mm	1	\$2,103.20



### Alarm Panel – Suitable with Turbo SMART Systems

The alarm panel is designed to be installed in highly visible area (i.e. reception). The lamp/s illuminate when there is an alarm present on the SMART suction system to alert staff. Includes power supply for lamps and mounting plate. Highly recommended for SMART units installed in parallel.

Order Number	Lamps	Pack Qty	Price Inc GST
D-RA1000	1	1	\$1,782.00
D-RA2000	2	1	\$2,262.70
D-RA3000	3	1	\$1,782.00
D-RA4000	4	1	\$1,514.70



## SUCTION ACCESSORIES

### EP Suction Isolation Valve

This 24V isolation valve is designed for use with all types of suction systems to isolate each chair's suction.

Order Number	Dimensions	Pack Qty	Price Inc GST
C-0039C0	Height 68mm Width 85mm Depth 70mm	1	\$665.50



### Automatic Air Injector Valve – SMART Suction

Automatic air injector valve designed to be installed at the end of the suction line. Allows air to enter the suction line when it is working at a closed head, keeping fluids flowing.

Order Number	Pack Qty	Price Inc GST
C-AAIV00	1	\$2,328.70



### Manual Air Injector Valve – Classic Suction

A manual air injector valve that is designed to be installed at the end of the suction line. Allows air to enter the suction line when it is working at a closed head, keeping fluids flowing.

Order Number	Pack Qty	Price Inc GST
C-1AIV00	1	\$243.65



### Spittoon Drain Valve

Automatic valve that connects the spittoon waste to the suction system. Requires 24V AD/DC supply. Valve is activated when fluid is sensed by the electronic level sensing probes. Valve also activates suction system if not already running.

- Post Box - installation inside the post box.
- Universal - installation in the floor box of dental unit.

Order Number	Dimensions	Pack Qty	Price Inc GST
C-021960	Post Box – Height 180mm Width 47mm Depth 93mm	1	\$1,081.30
C-021974	Universal – Height 180mm Width 110.7mm Depth 100mm	1	\$1,586.20



## Bacterial Filter – Aspi-Jet & Turbo Jet

Filters exhaust suction air prior to releasing to the atmosphere. Recommended when exhaust air has the potential to be released into areas accessible by people. Suitable for Aspi-Jet and Turbo-Jet 1 and 2.

- Removes 99.9% of particles
- Removes bacterial particles down to 0.01 micron
- Odour removal via activated carbon
- 30mm port connection (refer hose connection page for exhaust hoses)
- Filter should be replaced approximately every 12 months



Order Number	Description	Size	Pack Qty	Price Inc GST
CE11024	Filter Housing and Filter	Height 225mm (excludes port protusion) Diameter 140mm	1	\$1,086.80

## HEPA H14 Filter – 1-4 Surgeries

Filters exhaust suction air prior to releasing to the atmosphere. Recommended when exhaust air has the potential to be released into areas accessible by people. Suitable for Turbo SMART, Micro SMART and selected dry suction units, please contact Cattani customer service team for suitable recommendations.

- 99.975% efficiency at 0.3 micron
- Filter should be replaced approximately every 12 months
- Filters 90m<sup>3</sup>/h
- 63mm port connection (refer hose connection page for suitable exhaust hoses)



Order Number	Description	Size	Pack Qty	Price Inc GST
C-042090	Filter Housing and Filter	Height 230mm Diameter 200mm (excludes port protusion)	1	\$2,340.80
C-042090-KIT	Filter Installation Kit (Turbo SMART & Turbo SMART Cube)	Quick connecton kit for 042090 – includes hose connector, elbow, coupling and reduction bush	1	\$243.65
C-042090-KMS	Filter Installation Kit (Micro SMART & Micro SMART Cube)	Quick connecton kit for 042090 on Micro SMART only – includes hose connector, elbow, coupling and reduction bush.	1	\$290.95
C-160540	Replacement Filter	-	1	\$665.50

### HEPA H14 Filter – 5-8 Surgeries

Filters exhaust suction air prior to releasing to the atmosphere. Recommended when exhaust air has the potential to be released to areas accessible by people. Suitable for Turbo SMART and selected dry suction units, please contact Cattani customer service team for suitable recommendations.

- Filter should be replaced approximately every 12 months
- Filters 188m<sup>3</sup>/h
- 63mm port connection (refer hose connection page for exhaust hoses)

Order Number	Description	Size	Pack Qty	Price Inc GST
C-042092	Filter Housing and Filter (Tandem Turbo SMART & Tandem Turbo SMART Cube)	Height 460mm Diameter 200mm (excludes port protusion)	1	\$4,519.90
C-160541	Replacement Filter	-	1	\$997.70



### Password Upgrade – Turbo SMART A to B

Increase the capacity of your serviceable dental chairs through the use of the Cattani password upgrade. Simply entered on the keypad of the suction system and you are ready to go. That's SMART, that's Cattani.

Suitable to upgrade:

- Turbo SMART A to Turbo SMART B
- Turbo SMART Cube A to Turbo SMART Cube B

To streamline order process please provide:

1. Serial Number found on I.D. Label on chassis
2. Generated Code found on Control Panel



Order Number	Description	Price Inc GST
035500	Turbo SMART Password Upgrade (A to B)	\$2,832.50

PLEASE NOTE: Refunds/returns are NOT available on password upgrades once ordered from Cattani Italy.

## SUCTION ACCESSORIES

### Micro SMART Series – ISO 6 Hydrocyclone Amalgam Retention Retrofit Kit

Upgrade your Micro SMART to include amalgam retention.

- Flow Rates – Min 0.5 l/min | Max 5.5 l/min

To streamline order process please provide:

1. Serial Number found on I.D. Label on chassis

Order Number	Description	Pack Qty	Price Inc GST
C-MR6000	Micro SMART Amalgam Retrofit Kit	1	\$1,859.00
C-043382	Micro SMART CUBE Amalgam Retrofit Kit	1	\$1,859.00



### Turbo SMART Series – ISO 18 Hydrocyclone Amalgam Retention Retrofit Kit

Upgrade your Turbo SMART to include amalgam retention.

- Flow Rates – Min 0.5 l/min | Max 18 l/min

To streamline order process please provide:

1. Serial Number found on I.D. Label on chassis

Order Number	Description	Pack Qty	Price Inc GST
C-1H3C00	Turbo SMART Amalgam Retrofit Kit (Detachable keypad version)	1	\$2,281.40
C-1H3CUB	Turbo SMART CUBE Amalgam Retrofit Kit	1	\$2,868.80



### Replacement Amalgam Containers – Smart Suction

Cattani makes it easy to safely recycle mercury and amalgam through our partner, Ecocycle – Your Specialist Recycler.

Ecocycle helps keep mercury out of landfill and waterways through their state-of-the-art recycling facilities across the nation.

Please contact Ecocycle for pricing on replacement containers, pick up and recycling services – 1300 32 62 92 Australia, 0800 73 29 22 New Zealand.

Order Number	Description	Pack Qty
C-043132	ISO 6 Micro SMART and Micro SMART CUBE Amalgam Container	1
C-043135	ISO 18 Turbo SMART and Turbo SMART CUBE Amalgam Container	1
C-043138	ISO 60 Maxi SMART Amalgam Container	1



Contact Ecocycle on  
1300 32 62 92 Australia,  
0800 73 29 22 New Zealand.



## SUCTION SERVICE KITS

### Minor Service Parts

Order Number	Description	Pack Qty	Price Inc GST
C-215956 M-040727	Micro SMART Minor Service Kit – Includes inlet filter o-ring and 4 x 30ml bottle of LUBRI-JET	1	\$8.25 \$43.95
C-211974 M-040727	Turbo SMART Minor Service Kit – Includes inlet filter o-ring and 4 x 30ml bottle of LUBRI-JET	1	\$12.60 \$43.95
C-215986 M-040727	Maxi SMART Minor Service Kit – Includes inlet filter o-ring and 4 x 30ml bottle of LUBRI-JET	1	\$8.25 \$43.95

### Intermediate Service Kits

Order Number	Description	Pack Qty	Price Inc GST
C-MR60SK	Micro SMART and Micro SMART CUBE Intermediate Service Kit – ISO 6 Hydrocyclone	1	\$599.50
C-1H00SK	Turbo SMART and Turbo SMART CUBE Intermediate Service Kit – ISO 18 Hydrocyclone	1	\$176.00
C-043390SK	Maxi SMART Intermediate Service Kit – ISO 60 Hydrocyclone	1	\$257.40

### Major Service Kits

Order Number	Description	Price Inc GST
C-MSMSK0	Micro SMART Major Service Kit – Includes: C-025401-SK Separator Kit C-014057-SK Blower Kit and C-MSIHSK Internal Hose Kit	\$1,163.80
C-MSCMSK	Micro SMART CUBE Major Service Kit – Includes: C-025401-SK Separator Kit, C-017500-SK Blower Kit and C-MSCIH0-SK Internal Hose Kit	\$1,051.60
C-TSMSK0	Turbo SMART Major Service Kit – Includes: C-0034MS-3SK Separator Kit, C-1M0000-SK Blower Kit and C-TSIHSK Internal Hose Kit	\$1,086.80
C-TSCMSK	Turbo SMART Cube Major Service Kit – Includes: C-0034MS-3SK Separator Kit, C-014095-SK Blower Kit and C-TSCIH0-SK Internal Hose Kit	\$1,081.30
C-TS2VMSK0	Turbo SMART 2V – Major Service Kit – Includes: C-0034MS-3SK Separator Kit, C-019164-SK Blower Kit and C-TS2VIH-SK Internal Hose Kit	\$1,460.80
C-MAXIMSK	Maxi SMART Major Service Kit – Includes: C-025408-SK Separator Kit, C-091655-SK Blower Kit and C-MXIHSK Internal Hose Kit	\$1,871.10



Intermediate Service Kit



Separator Major Service Kit



Blower Major Service Kit

## Separator Major Service Kits

Order Number	Description	Pack Qty	Price Inc GST
C-025401SK	Micro SMART and Micro SMART CUBE Separator Major Service Kit	1	\$718.30
C-0034MS3SK	Turbo SMART, Turbo SMART CUBE, Turbo SMART 2V Separator Major Service Kit	1	\$807.40
C-025408SK	Maxi SMART Separator Major Service Kit	1	\$707.30

## Blower Major Service Kits

Order Number	Description	Pack Qty	Price Inc GST
C-014057SK	Micro SMART Blower (U.J. 40) Major Service Kit	1	\$356.40
C-017500SK	Micro SMART CUBE Blower (U.J. 25) Major Service Kit	1	\$347.60
C-1M0000SK	Turbo SMART Blower (U.J. 75) Major Service Kit	1	\$419.10
C-014095SK	Turbo SMART CUBE Blower (U.J.40) Major Service Kit	1	\$270.60
C-019164SK	Turbo SMART 2V Blower (U.J. 75 2V) Major Service Kit	1	\$403.70
C-091655SK	Maxi SMART Blower (M.J.2V) Major Service Kit	1	\$730.40

## Internal Hose Service Kits

Order Number	Description	Pack Qty	Price Inc GST
C-MSIHSK	Micro SMART Internal Hose Kit	1	\$45.10
C-MSCIH0-SK	Micro SMART CUBE Internal Hose Kit	1	\$187.00
C-TSIHSK	Turbo SMART Internal Hose Kit	1	\$124.30
C-TSCIH0-SK	Turbo SMART Cube Internal Hose Kit	1	\$410.30
C-TS2VIH-SK	Turbo SMART 2V Internal Hose Kit	1	\$247.50
C-MXIHSK	Maxi SMART Internal Hose Kit	1	\$380.60

## Major Service Kit (10,000 Hours)

Order Number	Description	Pack Qty	Price Inc GST
C-014000SK	Flux-Jet Blower – Major Service Kit (10,000 Hours)	1	\$578.60
C-A00150SK	Medio-Jet (~3) Blower – Major Service Kit (10,000 Hours)	1	\$386.10
C-006900SK	Tecno-Jet (~1) Blower – Major Service Kit (10,000 Hours)	1	\$522.50
C-0033M0SK	Turbo-Jet 1, Turbo-Jet 1 and 2 – Modular Separator (~1) Major Service Kit (10,000 Hours)	1	\$567.60
C-002300SK	Uni-Jet 75 (~1) Blower – Major Service Kit (10,000 Hours)	1	\$492.80
C-0023V0SK	Uni-Jet 75 2V (~1) Blower – Major Service Kit (10,000 Hours)	1	\$435.60
C-080090SK	Uni-Jet 500 (~3) Blower – Major Service Kit (10,000 Hours)	1	\$1,954.70
C-083067SK	Uni-Jet 1000 (~3) Blower – Major Service Kit (10,000 Hours)	1	\$2,400.20

## SUCTION SERVICE KITS

### Aspi-Jet Mobile Suction – Minor Service Kits (2,000 Hours or 1 Year)

Order Number	Description	Pack Qty	Price Inc GST
C-000600SK	Aspi-Jet – Minor Service Kit	1	\$353.10
C-000600SKB	Aspi-Jet – Minor Service Kit with replacement HEPA H14 Filter	1	\$1,395.90

### SMART Suction Complete Filter (Element and Housing)

Order Number	Description	Pack Qty	Price Inc GST
C-201547	Micro SMART and Micro SMART CUBE	1	\$267.30
C-201544	Turbo SMART, Turbo SMART CUBE and Turbo SMART 2V	1	\$267.30
C-201543	Maxi SMART	1	\$314.88



### Miscellaneous

Order Number	Size	Pack Qty	Price Inc GST
C-182575	Switch for black control panel – Turbo SMART	1	\$112.75
C-186485	Cooling Fan for Turbo SMART and Micro SMART	1	\$460.35

## Capacitors

Order Number	Description	Pack Qty	Price Inc GST
C-189106	10 µF Capacitor Metal Turbo-Jet 1 Range, Turbo-Jet 2 Centrifuge, Aspi-Jet Range and Uni-Jet 75	1	\$121.83
C-189146	20 µF Capacitor C87.0 "UL" Metal With 150mm Cable Turbo-Jet 2 Suction Motor, Tecno-Jet, and Uni-Jet 75 2V	1	\$139.70
C-189147	40 µF Capacitor C87.0 "UL" Metal With 350mm Cable Tecno-Jet 2V and Flux-Jet	1	\$221.65

## Aspi-Jet

Order Number	Size	Pack Qty	Price Inc GST
C-604200	Aspi-Jet Door Handle	1	\$24.53
C-606800	Rubber Elbow	1	\$73.21
C-603941	Canister Top – Aspi-Jet 6	1	\$256.85
C-603600	Castors – Aspi-Jet 6, 7, 8 and 9	1	\$23.16

## Filter – Inline Complete (Turbo-Jet 1 and 2 Modular)

Inline filter with 25mm port connections (includes inlet filter)

Order Number	Description	Pack Qty	Price Inc GST
C-004000	Inlet Filter	1	\$250.80



## Filter Element for Inline Filter

Order Number	Description	Pack Qty	Price Inc GST
C-240317	Filter element	1	\$85.42



## Filter Element for Aspi-Jet Range and Manual Tip Support

Order Number	Description	Pack Qty	Price Inc GST
C-113000	Filter element	1	\$50.38



## Air Filter – Aspi-Jet

Order Number	Description	Pack Qty	Price Inc GST
C-120280	Air filter	1	\$17.22



## SUCTION SPARE PARTS

### Filter for Pneumatic Tip Support (0055)

Order Number	Description	Pack Qty	Price Inc GST
C-551400	Filter element for all pneumatic tip support units	1	\$50.38



### Elbow Hose Connectors

32mm I.D. female coupling to tapered spigot for flexible suction hose.  
25mm (C-3923B0) 30mm (C-3923A0)

Order Number	Description	Pack Qty	Price Inc GST
C-3923B0	25MM	1	\$20.08
C-3923A0	30MM	1	\$20.08



### Straight Hose Connectors

32mm I.D. female coupling to tapered spigot for flexible suction hose.  
20mm (C-3922C0) 25mm (C-3922B0) 30mm (C-3922A0)

Order Number	Description	Pack Qty	Price Inc GST
C-3922C0	20MM	1	\$20.08
C-3922B0	25MM	1	\$20.08
C-3922A0	30MM	1	\$20.08



### Hose Connector for Motorised Suction Isolation Valve

Fits into the motorised suction isolation valve with tapered spigot for flexible suction hose.

Order Number	Description	Pack Qty	Price Inc GST
C-110840	40MM	1	\$28.49



### Coupling Connector – 32mm

32mm O.D. male joining coupling with O-rings

Order Number	Description	Pack Qty	Price Inc GST
C-0040D0	Coupling connector	1	\$50.38



### Mounted Connectors – Flange fitting with Captive Nuts

Order Number	Description	Price Inc GST
C-112800	2 Hose Tip Support (0011b) & Aspi-Jets	\$35.81



### Adaptor for EP Suction Isolating Valve

Reduction bush – glues into 40mm DWV pipe (pipe may need to be heated to fit the bush). Reduces internal diameter to 32mm to accept EP suction isolation valve or coupling connector – 32mm (C-0040D0).

Order Number	Description	Pack Qty	Price Inc GST
F-AEP400	Adaptor	1	\$24.97





**MAGNOLIA**  
CHEMICAL PROFESSIONALS

# **INFECTION PREVENTION & CONTROL DENTAL SUCTION SYSTEMS**

MAGNOLIA DISINFECTION  
AND HYGIENE RANGE



# CONSUMABLES

## WHY MAGNOLIA?

As dental care professionals, it is important to apply the appropriate product to your suction equipment to ensure not only the health of your machine, but also as part of your infection control for the safety of staff and patients.



The Magnolia disinfectant range has been specifically developed by Cattani for use with Cattani suction systems. The use of products and procedures outside of manufacturer recommendations can allow bacterial layers to build up. This can impede suction, compromising performance and ultimately damaging machines and systems.

Failure to clean, disinfect or maintain systems methodically as per the manufacturer's recommendations may affect the product warranty.

## MAGNOLIA PRODUCTS



### **PULI-JET GENTLE 2.0 – NON-FOAMING DENTAL DISINFECTANT**

At Cattani, we offer the PULI-JET Gentle 2.0 dental disinfectant from our Magnolia range, registered as an antimicrobial solution specifically made to disinfect dental medical devices. This is a non-foaming commercial grade disinfectant designed to clean and disinfect the dental suction lines.

### **SURFACTANT AND ANTI-SCALING AGENT**

PULI-JET Gentle 2.0 also contains surfactant capabilities to help cover more areas of the lines for prolonged disinfection. The other powerful feature is the anti-scaling agent which helps minimise mineral deposits in the pipework and the suction system.

Easy-to-remember disinfecting procedure.

- One single product
- One dilution method
- Once a day
- No alternating schedules
- No shock treatment required

### **PULSE CLEANER – MAXIMISES DISINFECTANT ASPIRATION THROUGH THE SUCTION SYSTEM**

The Pulse Cleaner automatically aspirates PULI-JET Gentle 2.0 disinfectant solution through the suction system by creating air and fluid turbulence, evenly and widely distributing the disinfectant for the most effective cleaning and protection.

### **LUBRI-JET SILICONE LUBRICANT DROPS – EASY CARE FOR O-RINGS AND SEALS**

LUBRI-JET is a silicone-based lubricant supplied in a handy dropper bottle, ideal for lubricating rubber O-rings and seals, which increases their lifespan.

Each single pack of LUBRI-JET contains four 30ml bottles



### **ANTIFOAMING DISINFECTANT TABLETS – CONTINUOUS DISINFECTION ALL DAY**

Antifoaming Disinfectant Tablets manage the foam that can occur through surgical treatments that have a large amount of dense liquids such as blood, mucus, and thick saliva. The tablet also has a slow-release capability, so that it continuously disinfects the suction lines throughout the entire working day.

Each pack contains 50 tablets, with one required per surgery per day.

### **SURGICAL SUCTION TIPS**

Cattani Suction Tips are designed to work in harmony with clinical staff and preferred operating techniques. The range covers every clinical situation, from large volume to the very precise, with clinical staff safety and patient comfort always in mind.

Available in high and low-volume sizes to evacuate fluids and airborne particles.

Autoclavable to 135°C. Each pack contains 3 tips.

## **AT CATTANI, WE DON'T JUST CLEAN. WE DISINFECT.**

# PULI-JET GENTLE 2.0

## Non-foaming disinfectant for dental suction systems

**PULI-JET Gentle 2.0** is an antimicrobial solution from our *Magnolia* range, specifically made to disinfect dental suction devices, known for its easy-to-remember single dilution disinfecting procedure.

- Phenol free
- Aldehyde free

No alternating schedules, products, or shock treatments are needed.

Each bottle makes 33 doses. Available in 1-litre bottles (30ml of PULI-JET Gentle 2.0 to 2.5 litres of warm water).

**ONE DISINFECTANT, ONE DILUTION, ONCE A DAY**



Order Number	Size	Price Inc GST
M060980	Single	\$67.95



**SUCTION UNIT MAINTENANCE**  
Point your smartphone camera at this QR code to view our short instructional videos

## CONSUMABLES

### Antifoaming Disinfectant Tablets

Reduces foam that occur through surgical treatments that have a large amount of dense liquids such as blood and mucus, as well as continuously disinfecting the suction lines through the working day.

Each pack contains 50 tablets, with one required per surgery per day.



Order Number	Size	Price Inc GST
M040825	Single	\$152.50

### Pulse Cleaner

Automatically aspirates PULI-JET Gentle 2.0 disinfectant solution through the suction system by creating air and fluid turbulence, evenly and widely distributing the disinfectant for the most effective cleaning and protection.

Order Number	Size	Price Inc GST
C-040720	Single	\$129.95



### Reducer Adaptor For Pulse Cleaner

Order Number	Size	Price Inc GST
C-040398	Single	\$35.00



### Plunger For Pulse Cleaner

Order Number	Size	Price Inc GST
C-201846	Single	\$33.28



### LUBRI-JET Silicone Lubricant Drops 30ml

A silicone-based lubricant supplied in a handy dropper bottle, ideal for lubricating rubber O-rings and seals, which increases their lifespan.

Each pack contains 4 x 30ml bottles.

Order Number	Size	Price Inc GST
M-040727	Single Box (4 x 30ml)	\$43.95



### Antifoam Liquid

High efficacy antifoam silicone liquid for larger dental facilities.

Reduces foam that occur through surgical treatments that have a large amount of dense liquids such as blood and mucus.

Order Number	Size	Price Inc GST
C-AS1000	Single	\$419.76



# SUCTION ACCESSORIES

## Surgical Suction Tip #9

Dental aspiration tip with a smaller diameter for precise evacuation of fluids from the oral cavity. The design makes it easy to use and ensures effective aspiration during dental procedures. Each pack contains 3 suction tips.

- Autoclavable (Type B) at 134°Celsius, 2.1 bar for 10 minutes
- Reusable for 100 sterilisation cycles
- Sterilise prior to use

Order Number	Size	Pack Qty	Price Inc GST
CT90000	Suits 11mm hose terminal	1	\$44.90



## Surgical Suction Tip #21

Dental aspiration tip with a smaller diameter for precise evacuation of fluids from the oral cavity. The design makes it easy to use and ensures effective aspiration during dental procedures. Each pack contains 3 suction tips.

- Curved design
- Autoclavable (Type B) at 134°Celsius, 2.1 bar for 10 minutes
- Reusable for 100 sterilisation cycles
- Sterilise prior to use

Order Number	Size	Pack Qty	Price Inc GST
CT21000	Suits 11mm hose terminal	1	\$44.90



## High Volume Evacuator (HVE) Tip #11

Recommended for aspirators with high airflow and low head. The design of the nebulisation and interceptor makes it easy to use and ensures effective aspiration during dental procedures where evacuation is required. Each pack contains 3 suction tips.

- Autoclavable (Type B) at 134°Celsius, 2.1 bar for 10 minutes
- Reusable for 100 sterilisation cycles
- Sterilise prior to use

Order Number	Size	Pack Qty	Price Inc GST
CT11000	Suits 16mm hose terminal	1	\$44.90



## High Volume Evacuator (HVE) Tip #17

This nebulisation and spray interceptor is ideal for aspirators with moderate airflow. The low noise design makes it easy to use and ensures effective spray interception during dental procedures where high volume evacuation is required. Each pack contains 3 suction tips.

- Quiet
- Angled design
- Autoclavable (Type B) at 134 °Celsius, 2.1 bar for 10 minutes
- Reusable for 100 sterilisation cycles
- Sterilise prior to use



Order Number	Size	Pack Qty	Price Inc GST
CT17000	Suits 11mm hose terminal	1	\$44.90

## High Volume Evacuator (HVE) Tip #20

The HVE #20 tip is recommended for aspirators with high airflow and low head. The curved design is easy to use and ensures effective aspiration and spray interception during dental procedures where high volume evacuation is required. Each pack contains 3 suction tips.

- Curved design
- Autoclavable (Type B) at 134 °Celsius, 2.1 bar for 10 minutes
- Reusable for 100 sterilisation cycles
- Sterilise prior to use



Order Number	Size	Pack Qty	Price Inc GST
CT20000	Suits 16mm hose terminal	1	\$44.90

## High Volume Evacuator (HVE) Tip #22

This nebulisation and spray interceptor is ideal for aspirators with moderate airflow. The curved it is easy to use and ensures effective aspiration and spray interception during dental procedures high volume evacuation is required. Each pack contains 3 suction tips.

- Reduces sound vibration
- Curved design
- Autoclavable (Type B) at 134 °Celsius, 2.1 bar for 10 minutes
- Reusable for 100 sterilisation cycles
- Sterilise prior to use



Order Number	Size	Pack Qty	Price Inc GST
CT22000	Suits 16mm hose terminal	1	\$44.90



## SUCTION ACCESSORIES

### Cleaning Brush for Surgical Suction Tips – Small

Designed for cleaning and scrubbing of dental suction tips. Effectively reaches all parts of the tip to clean prior to autoclaving. Suitable for small tips (11mm).

Order Number	Size	Pack Qty	Price Inc GST
C-180000	Suits 11mm suction tips	1	\$13.92

### Cleaning Brush For Surgical Suction and HVE Tips – Large

Designed for cleaning and scrubbing of dental suction tips and high volume evacuator tips. Effectively all parts of the internal surface to clean prior to autoclaving. Suitable for large tips

Order Number	Size	Pack Qty	Price Inc GST
C-190000	Suits 16mm suction tips	1	\$13.92

### Funnel Spittoon

Flexible in its design the Cattani funnel spittoon is designed to fit into both the 11mm and the 16mm hose terminals.

- Autoclavable (Type B) at 134°Celsius, 2.1 bar for 10 minutes
- Reusable for 50 sterilisation cycle

Order Number	Size	Colour	Pack Qty	Price Inc GST
C-040502	Suits 11mm and 16mm hose terminals	White	1	\$35.00

### Saliva Ejector Adaptor

Designed to suit most saliva ejector tips to fit the 11mm or 16mm hose terminal.

- Autoclavable (Type B) at 134°Celsius, 2.1 bar for 10 minutes
- Reusable for 50 sterilisation cycles

Order Number	Size	Colour	Pack Qty	Price Inc GST
C-150000	Suits 11mm and 16mm hose terminals	Grey	1	\$27.67

### Tip Reducer

The tip reducer reduces the internal diameter from 16mm to 11mm hose terminals to suit insertion of 11mm suction and high volume evacuator tips. Suitable for use with the Cattani Surgical Suction Tip #9 and High Volume Evacuator (HVE) Tip #17.

- Autoclavable (Type B) at 135°Celsius, 2.1 bar for 10 minutes

Order Number	Size	Colour	Pack Qty	Price Inc GST
C-160000	Reduces diameter from 16 to 11mm	Grey	1	\$8.20



### Hose Terminal – Small (11mm)

Designed to work with 11mm suction tips and the 11mm high volume evacuation tips.

- Autoclavable (Type B) at 134°Celsius, 2.1 bar for 10 minutes
- Reusable for 50 sterilisation cycles

Order Number	Size	Colour	Pack Qty	Price Inc GST
C-11015G	11mm	Grey	1	\$77.06
C-11015W	11mm	White	1	\$77.06



### Hose Terminal – Large (16mm)

Designed to work with 16mm suction tips and the 16mm high volume evacuation tips.

- Autoclavable (Type B) at 134°Celsius, 2.1 bar for 10 minutes
- Reusable for 50 sterilisation cycles

Order Number	Size	Colour	Pack Qty	Price Inc GST
C-11014G	16mm	Grey	1	\$77.06
C-110140	16mm	Black	1	\$77.06
C-11014W	16mm	White	1	\$77.06



## SUCTION ACCESSORIES

### Hose End Connector – Small (11mm)

Replacement hose end connector for use with the 11mm Hose Terminal. Also used to connect to manual and pneumatic tip supports.

- Autoclavable (Type B) at 134°Celsius, 2.1 bar for 10 minutes
- Reusable for 50 sterilisation cycles

Order Number	Size	Colour	Pack Qty	Price Inc GST
C-203470	11mm	Grey	1	\$28.00



### Hose End Connector – Large (16mm)

Replacement hose end connector for use with the 16mm Hose Terminal. Also used to connect to manual and pneumatic tip supports.

- Autoclavable (Type B) at 134°Celsius, 2.1 bar for 10 minutes
- Reusable for 50 sterilisation cycles

Order Number	Size	Colour	Pack Qty	Price Inc GST
C-203471	16mm	Grey	1	\$28.00



### Slide Closer

Replacement slide closer for use with the Hose Terminal.

Order Number	Size	Colour	Pack Qty	Price Inc GST
C-11015F	11mm	Grey	1	\$13.92
C-11014F	16mm	Grey	1	\$13.92



### Hose Terminal Complete Kit – Small (11mm)

The Cattani Hose Terminal Complete Kit comes ready to connect, simply select your colour and suction hose type. Each kit includes:

- A Hose Terminal
- A Suction Hose (1.5 metres)
- A Hose End Connector

Order Number	Size	Hose Type	Colour	Pack Qty	Price Inc GST
C-11018G	11mm	Spiral	Grey	1	\$205.43
C-11018S00G	11mm	Silicone	Grey	1	\$205.43

Also available in white via special order.



### Hose Terminal Complete Kit – Large (16mm)

The Cattani Hose Terminal Complete Kit comes ready to connect, simply select your colour and suction hose type. Each kit includes:

- A Hose Terminal
- A Suction Hose (1.5 metres)
- A Hose End Connector"



Order Number	Size	Hose Type	Colour	Pack Qty	Price Inc GST
C-11017G	16mm	Spiral	Grey	1	\$209.28
C-11017S00G	16mm	Silicone	Grey	1	\$209.28

*Also available in white via special order.*

### Manual Tip Support with Filter

Three hose terminal hanger with manual closure of terminals. Includes manifold with filter, 3 spiral tubes with terminals (1 x 16mm, 2 x 11mm), accessories and additional flat spiral tube. Dimensions without hoses or terminals: Height 118mm x Width 135mm x Depth 105mm.



Order Number	Entry Side	Colour	Pack Qty	Price Inc GST
C-0011BG	Rear	Grey	1	\$997.70

### Pneumatic Tip Support with Filter

Two hose terminal hanger with automatic pneumatic closure of terminals. Includes manifold with filter, 2 x silicone tubes with terminals (1 x 11mm and 1 x 16mm), fastening plate, accessories and spiral tube. Dimensions without hoses or terminals: Height 118mm x Width 135mm x Depth 105mm.



Order Number	Colour	Pack Qty	Price Inc GST
C-0055G0	Grey	1	\$1,336.50

## SUCTION ACCESSORIES

### Pneumatic Tip Support with filter and Triplex Holder

Two hose terminal hanger with automatic pneumatic closure of terminals. Also includes triplex holder. Includes an element filter. Dimensions without hoses or terminals: Height 118mm x Width 148mm x Depth 105mm.



Order Number	Colour	Pack Qty	Price Inc GST
C-5528G0	Grey	1	\$1,420.10

### Triplex Syringe Holder

Triplex syringe holder with the flexibility of being either cabinet mounted or fitted to the side of the Cattani Pneumatic Tip Support. Complete with adapters included to cater for various sized syringes.



Order Number	Colour	Pack Qty	Price Inc GST
C-A0028G	Grey	1	\$115.78

### Saliva Ejector Holder

Flexible installation options of being cabinet mounted or fitted to the side of the Cattani Pneumatic Tip Support. Complete with a micro-switch and a range of reducers to cater for various sized hoses.



Order Number	Colour	Pack Qty	Price Inc GST
C-0070G0	Grey	1	\$250.80

### Extension Segment Arm – Manual Tip Support

Extension arm suitable for use with the Cattani Manual Tip Support.

- Double segment arm extends: Minimum 50mm to Maximum 470mm
- Note: Extension lengths do not include the tip support depth.



Order Number	Arm Extension	Pack Qty	Price Inc GST
C-040050	Double	1	\$1,105.85

### Extension Segment Arm – Pneumatic Tip Support

Extension arm suitable for use with the Cattani Pneumatic Tip Support that can be wall or cabinet mounted.

- Single segment arm extends: Maximum 300mm
- Double segment arm extends: Maximum 500mm
- Maximum extension to double segment only

*Note: Extension lengths do not include the tip support depth.*



Order Number	Arm Extension	Pack Qty	Price Inc GST
C-006000	Single	1	\$587.95
C-006100	Double	1	\$920.70

### Suction Hoses – Spiral 11mm

Full roll is 25 metres.

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-11013G	11mm	Grey	\$49.67 PER M
C-110130	11mm	Black	\$49.67 PER M



### Suction Hoses – Spiral 14mm

Full roll is 25 metres.

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-H14G00	14mm	Grey	\$49.67 PER M
C-H14000	14mm	Black	\$49.67 PER M



### Suction Hoses – Spiral 16mm

Full roll is 25 metres.

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-11012G	16mm	Grey	\$49.67 PER M
C-110120	16mm	Black	\$49.67 PER M





## SUCTION ACCESSORIES

### Suction Hoses – Spiral 20mm

Full roll is 25 metres.

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-H20G00	20mm	Grey	\$70.57 PER M
C-H20000	20mm	Black	\$70.57 PER M



### Suction Hoses – Spiral 25mm

Full roll is 25 metres.

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-H25G00	25mm	Grey	\$70.57 PER M
C-H25000	25mm	Black	\$70.57 PER M



### Suction Hoses – Spiral 30mm

Full roll is 25 metres.

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-H30G00	30mm	Grey	\$70.57 PER M
C-H30000	30mm	Black	\$70.57 PER M



### Suction Hoses – Spiral 40mm

Full roll is 25 metres.

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-H40000	40mm	Black	\$87.07 PER M



## SUCTION ACCESSORIES

### Suction Hoses – Spiral 50mm

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-H50000	50mm	Grey	\$87.07 PER M



### Suction Hoses – Spiral 60mm

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-H60000	60mm	Black	\$98.01 PER M



### Suction Hoses – Spiral 80mm

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-H80000	80mm	Black	\$168.85 PER M



### Suction Hoses – Silicone 11mm

Full roll is 25 metres.

Tubing is sold in metre lengths only unless in full rolls.

Order Number	Internal Diameter	Colour	Price Inc GST
C-11013S00G	11mm	Grey	\$85.42 PER M
C-11013W	11mm	White	\$85.42 PER M



### Suction Hoses – Silicone 16mm

Full roll is 25 metres.

Tubing is sold in metre lengths only unless in full rolls.


Order Number	Internal Diameter	Colour	Price Inc GST
C-11012S-00G	16mm	Grey	\$83.88 PER M
C-11012W	16mm	White	\$83.88 PER M



Go to page 83 for heat resistant hoses for suction or exhaust air.

# SELECTION, INSTALLATION, MAINTENANCE AND SERVICE

The pursuit of 100% reliability is what drives Cattani at every level. Our Cattani Blueprints Training Program™ and Cattani After Care Training Program™ cover the essential aspects of selection, installation, ongoing preventative maintenance, and servicing of Cattani equipment for dental clinics.

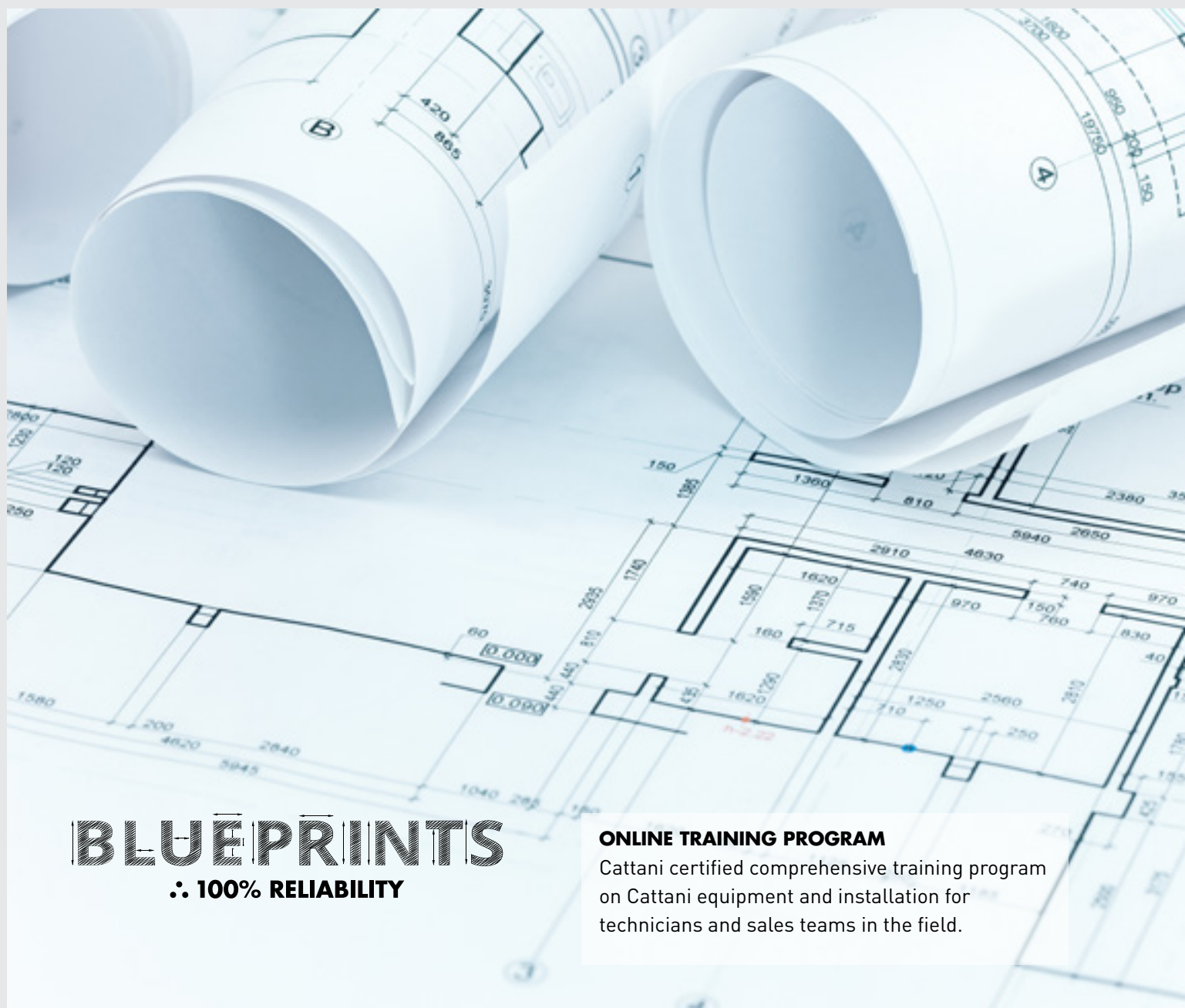

**BLUEPRINTS**  
∴ 100% RELIABILITY

#### ONLINE TRAINING PROGRAM

Cattani certified comprehensive training program on Cattani equipment and installation for technicians and sales teams in the field.

# AFTER CARE

∴ 100% RELIABILITY

## AFTER CARE TRAINING PROGRAM

Comprehensive training program for technicians on everything about servicing your equipment.

## AFTER CARE: PROTECT THE POWER BEHIND YOUR PRACTICE

Cattani Australasia wants equipment owners to get maximum benefit, reliability, and life out of their purchase. To that end, we encourage all end users to follow our daily and weekly maintenance guidelines.

We also urge owners to ensure that their equipment is serviced annually using the After Care Service Kits. With annual servicing and scheduled replacement of wear-prone parts you are protecting the power behind your practice. Your suction unit and compressor will then serve you well for its lifetime, preventing untimely and costly breakdowns; and you know the cost from the beginning.



# VENTILATION GUIDELINES – CONTROLLING THE TEMPERATURE IN YOUR PLANT ROOM

### AIR CONDITIONING

Air conditioning is the best way to maintain the stable temperatures your plant room equipment needs for optimal performance. Modern split systems are the most efficient and affordable. And with a dedicated air conditioner in your plant room, no vents are required, so noise emissions are greatly reduced.

The clinic's central air conditioning system must not include the plant room. Instead the room should have its own dedicated air conditioner. Central systems are designed for human comfort and the thermostat is usually located in reception. It cannot sense the temperature in the plant room, so in winter, for example, it would often be heating the room, even though it generates its own heat.

Recommendations for plant room air conditioning:

- 1-4 surgeries – 2.5 kW
- 4-6 surgeries – 3.5 kW
- 6-8 surgeries – 4.5 kW
- 8-10 surgeries – 5.5 kW

### NOISE REDUCTION OPTIONS

All plant room equipment generates noise, which can irritate dentists, staff and patients. The noise level may also breach council and body corporate regulations. Any noise reduction solution that will cause equipment to overheat is out of the question, so a balance between ventilation and noise reduction needs to be found:


- The plant space can be air conditioned to eliminate the need for vents
- The space may be lined with sound insulation
- Cattani supplies Acoustic Covers for certain models
- Mufflers can be fitted to the suction exhaust outlet
- Fan-forced ventilation vents can be baffled (see below)

### FAN-FORCED VENTILATION

Fan-forced ventilation systems turn over the air in the plant room. Cool air is drawn in while hot air is exhausted via the in-line fan. With this ventilation method we recommend a fan capacity of 150m<sup>3</sup>/hr per 1 kW power rating of your equipment.

There are drawbacks with fan-forced ventilation:

- Noise – the vents needed to allow air to flow through the plant room also allow more noise to escape the room, including extra noise generated by the cooling fans.
- Temperature – an efficient fan-forced ventilation system will only reduce the temperature to 10°C above the incoming air temperature. Also, temperature control is only as consistent as the incoming air temperature.

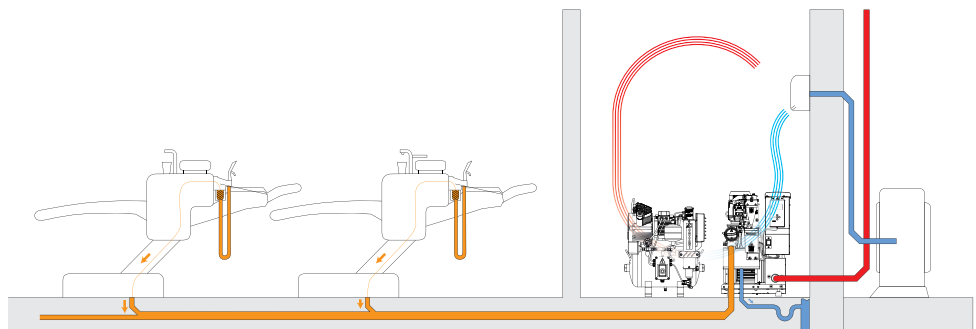

A simple baffle can reduce noise escaping from vents. Essentially, a box with one end open is placed over the vent. Air can pass through but noise is absorbed. The box can be made of materials similar to cabinet doors and ideally should be lined with sound insulation. Ensure that the open end of the box is large enough so that it does not restrict air flow. Restricting air flow will compromise the efficiency of the ventilation system.



# WHAT YOU NEED TO KNOW

Suction and air supply are the unsung heroes of the dental practice. Without their optimal performance and reliability, work suffers and can even stop, so it is essential that your Cattani equipment is installed correctly.

Your Cattani dealer will discuss with you the various installation options and recommend those that best suit your clinic and environment. The information below is a useful starting point for site requirements, while your installer has the expertise to ensure your set-up produces optimal performance and longevity.

The team at Cattani is also happy to advise you on planning your installation. Call us at +61 3 9484 1120.

### SUCTION SYSTEM ESSENTIALS

The suction piping system uses DWV PVC pipe and connects each dental surgery back to the suction unit in the plant room. The suction unit should be installed on the same floor level or below the level of the dental chairs. It should have a constant fall from the chairs towards the suction unit and cannot include 90° junctions, dips, U-shapes or rises. Termination within the dental surgery is typically at the floor junction box of the dental chair, but is site dependent. The chair manufacturer or installer should be consulted for location details.

Your suction unit will need:

- Suction inlet connection – termination of suction piping that connects to dental surgeries
- Waste connection – sewer drain pipe terminated lower than waste outlet of suction unit
- Exhaust air outlet connection – piping that directs exhaust air to atmosphere outside the building
- Power supply – dedicated electrical power circuit to run the suction unit
- Control switch wiring – switch wiring to connect the dental surgeries to the suction unit

### COMPRESSOR ESSENTIALS

The compressed air piping system is typically copper and connects each dental surgery back to the compressor unit in the plant room. Termination within the dental surgery is typically at the floor junction box of the dental chair, but is site dependent. The chair manufacturer or installer should be consulted for location details.

Your compressor will need:

- Compressed air outlet connection – termination of compressed air piping system that connects to the dental surgeries
- Power supply – dedicated electrical power circuit to run the suction unit
- Waste connection (optional) – sewer connection to allow drainage of compressor condensate

### LOCATION AND ACCESS – WHERE TO POSITION YOUR PLANT ROOM

When designing a new practice, the location of your plant room is an important consideration because proper planning will ensure the most efficient and balanced suction performance. Locating the plant room close to surgeries prevents underperformance caused by excessive pipeline lengths. The plant room must be on the same floor as the dental chairs, or below them. Fluids need to fall to the pump, so if there is a floor below the chairs, like a car park or basement that is accessible and still close to the dental suites above, that location would be ideal for the plant room. Many sites also have air-conditioned plant rooms to protect their investment.

### MAXIMISE YOUR SYSTEM'S PERFORMANCE WITH AN AIR INJECTOR VALVE

Cattani highly recommends the use of an Air Injector Valve with all suction systems. It is best to install one at the time the clinic is constructed but they can also be added later.


Air must flow through the piping in a suction system to help aspirated fluids reach the suction unit. Depending on the site scenario, it is possible that insufficient air will flow through the system. This can lead to fluctuations in performance as fluids pool at the lowest part of the system, with air flow falling below optimal levels. An Air Injector Valve, installed at the end of the pipeline, is an excellent way to compensate for any lack of air volume.



There are two types available:

- **Automatic Air Injector Valve (SMART Suction)** – an automated unit that allows the flow of large volumes of air over short periods. This is an electro-pneumatic valve that is actuated by an electrical signal. It is designed to open only when all chairs have ceased operation. With no air flow required from the chairs, the valve allows a large volume of air to be injected into the system without compromising the performance to the chair.
- **Standard Air Injector Valve (Classic Suction)** – a constantly variable unit that allows the flow of small volumes of air over long periods. It is a simple spring-loaded, vacuum-actuated valve, designed to open at the same time that the chairs are in operation.

If you are building a new clinic, we strongly recommend that you make provision for an Air Injector Valve to be fitted, even if you don't install it straight away. It is much simpler to install an Air Injector Valve to existing services rather than have to add those services later. To provide for an Air Injector Valve, you need to extend the suction pipeline past the chair furthest from the suction unit, terminating the line at a point higher than the main suction pipe line, in an accessible area like a cabinet, ceiling space, void etc. An additional switch cable between this point and the suction unit is also required. Ensure that the pipe is capped if an Air Injector Valve is not being installed immediately. If a system does not have the extra pipe-work from the start, an Air Injector Valve can be retro-fitted by adding it to the floor junction box of the dental chair. However, the noise generated by the aspirated air can be disruptive.

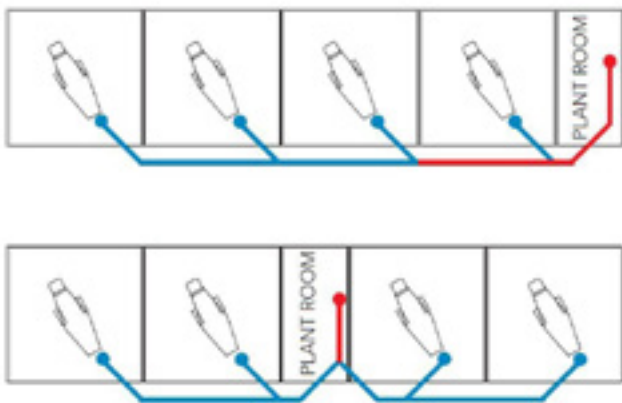

### PIPELINE DESIGN AND LAYOUT

The pipeline needs to facilitate the flow of air and fluids back to the plant room. Pipelines should be designed with a main pipe connected to branches from each dental chair. Large clinics may also require sub-branches. Where possible the piping should take the shortest and straightest route between the plant room and the last dental chair on the line. The diameter of the pipe depends on the number of chairs and the proximity to the plant room: in simple terms, the further from the plant room, the smaller the pipe.

### PIPE SIZE GUIDELINES

Pipe size plays a crucial role in ensuring an even air flow and efficient evacuation of fluids. Suction plumbing requirements based on an average air flow rate of 350l/min per chair are:

- 1-2 chairs – 40mm DWV pipe
- 3-6 chairs – 50mm DWV pipe
- 7-10 chairs – 65mm DWV pipe
- 11-15 chairs – 80mm DWV pipe

### PIPELINE CONFIGURATION GUIDELINES

Bends in the pipeline should never be greater than 45°. If a 90° bend is required, use 2 x 45° bends together in-line. All branches entering a main line must use 45° junctions. Do not use a T-junction when splitting main lines into two branches. Instead, use a 45° Y-junction for the first branch off the main line and 2 x 45° bends for the second.

### PIPELINE RISE AND FALL GUIDELINES

The pipeline should slope constantly towards the suction unit in the plant room. Strictly avoid dips and U-shapes around beams etc to prevent fluids pooling at the lowest point and restricting air flow. If the suction unit is installed on the same floor as the dental chairs, a single vertical rise should be installed just before the unit with the height kept to a minimum. The higher this rise, the greater the risk of fluids creating a wave effect as they try to reach the inlet of the suction unit. This “wave” may cause fluctuating performance at the suction tips.

# AFTER CARE DENTAL CLINIC RESOURCES

## USEFUL DOCUMENTS FOR THE PRACTICE MANAGER



**DOWNLOAD NOW**  
SMART USER MAINTENANCE GUIDE  
FOR CATTANI SMART SUCTION

Point your smartphone camera at this QR code to view.



**DOWNLOAD NOW**  
EXTENDED SHUTDOWN AND RESTART GUIDE  
FOR CATTANI EQUIPMENT


Point your smartphone camera at this QR code to view.

### COMPRESSOR

### SMART SUCTION



**DOWNLOAD NOW**  
AFTER CARE EQUIPMENT  
SERVICING SCHEDULE

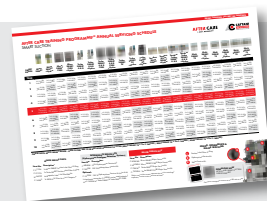

Point your smartphone camera at this QR code to view.



**DOWNLOAD NOW**  
AFTER CARE SCHEDULED  
MAINTENANCE SERVICE RECORD

Point your smartphone camera at this QR code to view.

**Sales and Technical Support:**



**Australia**  
+61 3 9484 1120  
[cattani@cattani.com.au](mailto:cattani@cattani.com.au)  
[www.cattani.com.au](http://www.cattani.com.au)



**New Zealand**  
0800 68 22 88  
[cattani@cattani.co.nz](mailto:cattani@cattani.co.nz)  
[www.cattani.co.nz](http://www.cattani.co.nz)

**Company Address:** 280 Dundas Street, Thornbury VIC 3071

